COOK®

MEDICAL

**EN
2**

Non-lithotripsy Mini Extraction Basket

**FR
4**

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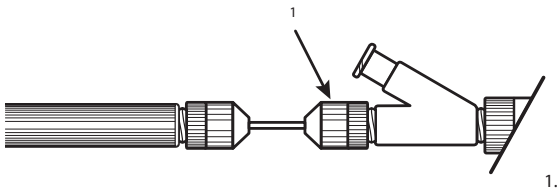
ENGLISH

INTENDED USE

This device is used for the endoscopic removal of biliary stones, pancreatic stones, and foreign bodies.

NOTES

Do not use this device for any purpose other than the stated intended use.

Store in a dry location, away from temperature extremes. The recommended long-term storage temperature is 22°C (71.6°F). The device may be exposed to temperatures up to 50°C (122°F) during transport.

Use of this device is restricted to a trained healthcare professional.

Basket extraction of a pancreatic stone or debris should only be undertaken by endoscopists who have been specifically trained to perform pancreatic ductal interventions.

CONTRAINDICATIONS

Those specific to the primary endoscopic procedure to be performed in conjunction with the removal of pancreatic stones, biliary stones, stents, or foreign bodies.

Perforation.

If this device is to be used for biliary or pancreatic stone extraction, contraindications include an ampullary or pancreatic duct opening inadequate to allow for unimpeded passage of the stone and basket.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy and ERCP include, but are not limited to: pancreatitis • cholangitis • sepsis • perforation • hemorrhage • aspiration • fever • infection • hypotension • allergic reaction • respiratory depression or arrest • cardiac arrhythmia or arrest.

Other potential complications associated with basket extraction include, but are not limited to: impaction of a pancreatic stone or other pancreatic

ductal debris • aspiration of a foreign body • localized inflammation • pressure necrosis • perforation and ductal trauma • pancreatitis and/or tearing, rupturing or laceration of the ductal system. The risk of these complications may be increased if excessive force is used to withdraw the basket with a captured pancreatic stone or other pancreatic ductal debris.

PRECAUTIONS

Refer to the package label for the minimum channel size required for this device.

If this device is to be used for the removal of biliary or pancreatic stones, assessment of the stone size and ampullary or pancreatic duct opening must be made to determine the necessity of sphincterotomy. In the event sphincterotomy is required, all appropriate cautions, warnings and contraindications must be observed.

This device should never be coiled in less than an 8-inch (20 cm) diameter.

WARNINGS

This device is **not** compatible with the Soehendra® Lithotripter or any other mechanical lithotripter. Use of this device with a mechanical lithotripter may result in device failure and/or basket impaction. Surgical intervention may be required if impaction occurs.

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

INSTRUCTIONS FOR USE

1. With the basket fully retracted into the sheath, insert the device into the endoscope accessory channel. Advance the device through the channel, in short increments, until the basket sheath exits the endoscope.
2. Fluoroscopically visualize the extraction target and advance the basket to the appropriate position. Fluoroscopic visualization may be enhanced by injecting contrast via the side arm fitting. **Note:** Tighten the connector cap on the handle before injection of contrast. (See fig. 1)
3. Place the sheath of the basket beyond the target. Confirm the desired position of the basket sheath relative to the target.
4. Advance the basket out of the sheath. **Caution:** Pulling on the sheath while advancing or retracting the basket may damage the device, rendering it inoperable.
5. Use the extended basket to retrieve the stone or foreign body from the duct. **Note:** It may be necessary to move the basket back and forth along the target to retrieve the target.

For stone removal: Withdraw the basket into the channel until the stone is removed from the duct. For multiple stones, repeat the

process until all stones have been removed. **Caution:** If withdrawal of the basket from the duct is restricted, surgical intervention may be necessary.

For foreign body removal: Withdraw the basket into the channel until the foreign body is removed from the duct. **Caution:** If withdrawal of the basket from the duct is restricted, surgical intervention may be necessary.

6. Prepare or dispose of the extracted specimen per institutional guidelines.

Upon completion of procedure, dispose of device(s) per institutional guidelines for biohazardous medical waste.

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A symbol glossary can be found at cookmedical.com/symbol-glossary

Pour un glossaire des symboles, consulter le site Web cookmedical.com/symbol-glossary • En cookmedical.com/symbol-glossary puede consultarse un glosario de símbolos • En symbolordlista.finnsp.com på cookmedical.com/symbol-glossary

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If symbol appears on product label, X = maximum insertion portion width

Si le symbole est visible sur l'étiquette du produit, X = largeur maximale de la section d'insertion • Se o símbolo aparecer no rótulo do produto, X = largura máxima da parte introduzida • Si el símbolo aparece en la etiqueta del producto, X = anchura máxima de la parte que se va a insertar

If symbol appears on product label, X = minimum accessory channel

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If symbol appears on product label, X = wire guide compatibility

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If symbol appears on product label, X = quantity per box

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**This device is intended
for single use only.**

RxOnly

STERILE

EO

Wilson-Cook Medical, Inc.

4900 Bethania Station Road
Winston-Salem, North Carolina 27105
USA

© COOK 2020
cookmedical.com