EN
2

Memory™ Soft Wire Baskets

CS
4

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DA
6

Memory™ bløde trådkurve

NL
8

Memory™ zachte-draad-baskets

FR
11

Paniers Memory™ en fil souple

DE
13

Memory™ Weichdraht-Körbchen

EL
16

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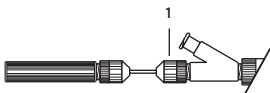
ENGLISH

INTENDED USE

This device is used for the endoscopic removal of biliary stones and foreign bodies.

NOTES

Select Memory Soft Wire Baskets are compatible with the Soehendra Lithotripter.

Do not use this device for any purpose other than the stated intended use.

Store in a dry location. Recommended long term storage temperature is 22°C (71.6°F). Device may be exposed to temperatures up to 50°C (122°F) during transportation.

Use of this device is restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to the primary endoscopic procedure to be performed in conjunction with the removal of the foreign body.

Perforation.

If this device is to be used for the removal of biliary stones, contraindications include an ampullary opening inadequate to allow for the unimpeded passage of the stone and basket.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy and ERCP include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest.

Others associated with basket extraction include, but are not limited to: impaction of the object, aspiration of a foreign body, localized inflammation, pressure necrosis.

PRECAUTIONS

Refer to the package label for the minimum channel size required for this device and to determine basket compatibility.

If this device is to be used for the removal of biliary stones, the following caution applies: Assessment of the stone size and the ampullary orifice must be made to determine the necessity of sphincterotomy. If sphincterotomy is required, all appropriate cautions, warnings and contraindications must be observed.

This device should never be coiled in less than an 8-inch (20 cm) diameter.

WARNINGS

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

This device is supplied sterile and intended for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

When retrieving a foreign body from the upper GI tract, extreme caution must be exercised to avoid occlusion of the airway.

Surgical intervention may be required if stone impaction and/or basket fragmentation occurs.

If a stone cannot be removed endoscopically with this basket, the Soehendra Lithotripter may be used with select Memory Soft Wire Baskets (see package label) to mechanically crush the stone and aid in removal. Due to the mechanical pressure generated with the Soehendra Lithotripter, basket fragmentation and/or stone impaction in the common bile duct may occur and require surgical intervention. Risk of basket fragmentation or stone impaction must be weighed against the potential benefit of using the lithotripter.

INSTRUCTIONS FOR USE

1. With the basket fully retracted into the sheath, insert the device into the endoscope accessory channel. Advance the device through the channel in short increments until the basket sheath exits the endoscope. **Note:** When using a basket that is removable from its sheath, the basket may be removed to facilitate the use of a wire guide. When the sheath is in the proper position, remove the wire guide, then reintroduce the basket through the sheath.
2. Endoscopically or fluoroscopically visualize the extraction target and advance the basket to the appropriate position. Fluoroscopic visualization may be enhanced by injecting contrast via the side arm fitting. Place the sheath of the basket beyond the target before extending the basket to entrap the extraction target. **Note:** Tighten the connector cap on the handle before injecting contrast. (See fig.)

3. Confirm the desired position of the basket sheath relative to the target. With one hand holding the side arm fitting, **gently** push forward on the pin vise handle with the other hand to advance the basket out of the sheath. **Caution:** Pulling on the sheath while advancing or retracting the basket may damage the device, rendering it inoperable.
4. Place the basket around the extraction target or the foreign body and slowly retract the handle until entrapment is achieved. **Note:** It may be necessary to move the basket back and forth along the target in order to capture the object. **Note:** The connector cap on the handle may be loosened in order to more freely manipulate the basket. (See fig.)
5. Withdraw the basket into the channel until the foreign body is supported against the endoscope tip, then withdraw the endoscope from the patient. Maintain endoscopic visualization to ensure continued entrapment of the object. While withdrawing the device from the endoscope, wipe any secretions from the outer sheath. **Caution:** If difficulty is encountered during the removal of the basket from the duct, moderate force may be applied by pulling on the handle to manually fracture the stone. If this proves unsuccessful, a compatible basket may be used with the Soehendra Lithotriptor to mechanically crush the stone. If passage is still restricted, surgical intervention may be necessary.
6. Prepare or dispose of the extracted specimen per institutional guidelines.

Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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© 2019 Cook Medical
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.