COOK®

MEDICAL

EN

2

Soehendra® Lithotripsy Cable

FR

3

Câble de lithotripsie Soehendra®

PT

5

Cabo de litotripsia Soehendra®


ES

6

Cable de litotricia Soehendra®

* 1 0 6 8 7 / 0 1 2 0 *

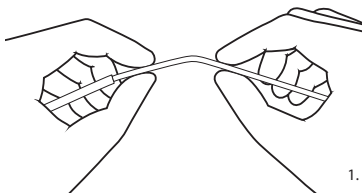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ENGLISH

INTENDED USE

This device is used to mechanically crush stones in the biliary duct when other methods of endoscopic removal have failed.

NOTES

Do not use this device for any purpose other than the stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device is restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to ERCP and mechanical lithotripsy.

Those associated with biliary stone extraction include, but are not limited to: an ampullary opening inadequate to allow for the unimpeded passage of the stone and basket.

POTENTIAL COMPLICATIONS

Those associated with ERCP include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction to contrast or medication, respiratory depression or arrest, cardiac arrhythmia or arrest.

Those associated with basket extraction include, but are not limited to: impaction of the object, localized inflammation, pressure necrosis.

PRECAUTIONS

Wire cutters must be available when using this device.

Assessment of the stone size and ampullary orifice must be made to determine the necessity of sphincterotomy prior to this procedure. The length of the intramural segment must be evaluated before determining the advisability of mechanical lithotripsy. In the event that sphincterotomy is required, all appropriate cautions, warnings and contraindications must be observed.

WARNINGS

This device is supplied sterile and intended for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Due to the mechanical pressure generated with this device, basket fragmentation in the common bile duct is a possibility that may require surgical intervention.

Only select Cook Biliary Soft Wire Baskets are recommended for use with this device.

Do not remove the sheath from the basket wire. Doing so may result in basket fragmentation and consequently require surgical intervention.

Due to the varying compositions of biliary stones, stone fracture may not be possible. If the stone cannot be fractured, continued rotation of the handle may cause the basket wire to break, requiring surgical intervention.

INSTRUCTIONS FOR USE

1. Slightly kink the basket wire and sheath below the handle and the reinforcement sheath. (See fig. 1)
2. Using wire cutters, cut the handle off the basket. Do not remove the sheath from the basket wire. (See fig. 2)
3. Fluoroscopically monitor the basket and stone to ensure continued entrapment while slowly removing the endoscope.
4. Gently feed the lithotripsy cable over the basket sheath and advance the cable, under fluoroscopic monitoring, until it reaches the basket in the duct.
5. Follow Instructions for Use for the Soehendra® Lithotripter Handle to continue with lithotripsy procedure.

Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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A symbol glossary can be found at cookmedical.com/symbol-glossary

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X mm

If symbol appears on product label, X = maximum insertion portion width

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X mm

If symbol appears on product label, X = minimum accessory channel

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X mm

If symbol appears on product label, X = wire guide compatibility

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If symbol appears on product label, X = quantity per box

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This device is intended
for single use only.

Wilson-Cook Medical, Inc.
4900 Bethania Station Road
Winston-Salem, North Carolina 27105
USA

RxOnly

STERILE

EO