
- EN**
3 **Soehendra® Lithotripter Handle**
Instructions for Use
- BG**
5 **Дръжка за литотриптер Soehendra®**
Инструкции за употреба
- HR**
7 **Drška uređaja za litotripsiju Soehendra®**
Upute za uporabu
- CS**
8 **Rukojeť litotryptoru Soehendra®**
Návod k použití
- DA**
10 **Soehendra® litotripterhåndtag**
Brugsanvisning
- NL**
12 **Soehendra® Lithotripter-handgreep**
Gebruiksaanwijzing
- ET**
14 **Litotriptori käepide Soehendra®**
Kasutusjuhend
- FI**
16 **Soehendra®-litotripsiakahva**
Käyttöohjeet
- FR**
17 **Poignée de lithotriporteur Soehendra®**
Mode d'emploi
- DE**
19 **Soehendra® Lithotripter-Griff**
Gebrauchsanweisung
- EL**
21 **Λαβή λιθοτρίπτη Soehendra®**
Οδηγίες χρήσης
- HU**
23 **Soehendra® kőzúzófogantyú**
Használati utasítás
- IT**
25 **Impugnatura per litotritore di Soehendra®**
Istruzioni per l'uso
- LV**
27 **Soehendra® litotripsijas ierīces rokturis**
Lietošanas instrukcija
- LT**
29 **„Soehendra®“ litotriptoriaus rankena**
Naudojimo instrukcija
- NO**
31 **Soehendra® litotripterhåndtak**
Bruksanvisning
- PL**
32 **Uchwyt litotryptora Soehendra®**
Instrukcja użycia
- PT**
34 **Punho do litotripter Soehendra®**
Instruções de utilização

RO 36	Mâner de litotritor Soehendra® Instrucțiuni de utilizare
SK 38	Rúčka litotryptora Soehendra® Návod na použitie
SL 40	Držalo za litotriptor Soehendra® Navodila za uporabo
ES 42	Mango de litotritor Soehendra® Instrucciones de uso
SV 43	Soehendra®-litotriptorhandtag Bruksanvisning
TR 45	Soehendra® Litotriptör Elceği Kullanma Talimatı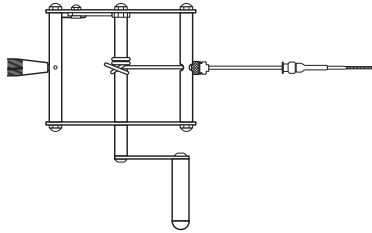

1.

ENGLISH

Soehendra® Lithotripter Handle

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

Please read all instructions before using this device.

DEVICE DESCRIPTION

This device is to be used with Cook Lithotripsy Extraction Baskets only. Look for the 'Lithotripsy Compatible' symbol on the product labels for Cook Extraction Baskets.

Patient Population

This device is indicated for adult use only.

Intended Users

Use of this device is restricted to a trained healthcare professional.

INTENDED USE

This device is used to mechanically crush stones in the biliary duct when other methods of endoscopic removal have failed.

INDICATIONS FOR USE

This device is used for treatment of biliary obstruction due to gallstones (choledocholithiasis) in adult patients.

CLINICAL BENEFITS

This device relieves jaundice and reduces the risk of infection due to obstruction.

CONTRAINDICATIONS

Those specific to ERCP and mechanical lithotripsy.

Those associated with biliary stone extraction, including: an ampullary opening inadequate to allow for the unimpeded passage of the stone and basket.

WARNINGS

- This device is supplied non-sterile and therefore, should not be used in a sterile field. The device is reusable if its integrity remains intact.
- If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks,

bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

- Due to mechanical pressure generated with this device, basket fragmentation in the common bile duct is a possibility that may require surgical intervention.
- Device is only recommended for use with the Cook Lithotripsy Extraction Baskets.
- Due to the varying compositions of biliary stones, stone fracture may not be possible. If the stone cannot be fractured, continued rotation of the handle may cause the basket wire to break, requiring surgical intervention.
- Device contains nickel which may cause an allergic reaction and other potential systemic complications related to nickel allergy.

PRECAUTIONS

- Do not use this device for any purpose other than the stated intended use.
- Wire cutters must be available when using this device.
- Assessment of the stone size and ampullary orifice must be made to determine necessity of sphincterotomy prior to this procedure. The length of the intramural segment must be evaluated before determining advisability of mechanical lithotripsy. In the event a sphincterotomy is required, all appropriate cautions, warnings and contraindications must be observed.
- Reusability of the device depends in large part on care of the device by the user. Factors involved in prolonging the life of this device include: thorough cleaning following the instructions included in this booklet.

POTENTIAL COMPLICATIONS

Those associated with ERCP, including: pancreatitis • cholangitis • sepsis • perforation • hemorrhage • aspiration • fever • infection • hypotension • allergic reaction to contrast or medication • respiratory depression or arrest • cardiac arrhythmia or arrest.

Those associated with basket extraction, including: impaction of the object • localized inflammation • pressure necrosis.

HOW SUPPLIED

This device is supplied non-sterile and may not be sterilized by any method. Store in a dark, dry, cool place. Avoid extended exposure to light and extremes of temperature and humidity.

SYSTEM PREPARATION

1. Examine the Luer lock fittings on the cable and the handle for a proper fit.
2. Ratchet on the handle must be attached securely to allow rotation in one direction.

INSTRUCTIONS FOR USE

If using SLC cable

1. Insert the basket wire and the sheath through the male luer lock fitting on the lithotripsy handle, then through the hole in the rotating rod of the handle. **Warning:** Do not remove the sheath from the basket wire. Doing so may result in basket fragmentation and consequently require surgical intervention.
2. Wrap the wire and the sheath several times around the rod to secure them to the rod. (See fig. 1)

If using TTCL cable

1. Insert the basket wires through the male luer lock fitting on the lithotripsy handle, then through the hole in the rotating rod of the handle.
2. Wrap the basket wires several times around the rod to secure them to the rod. (See fig. 1)

Stone Fracture

1. To fracture the stone, apply pressure by slowly rotating the handle. **Note:** Slow, constant pressure is often the most effective method of fracturing a stone. A sudden release of pressure may indicate fracture of the stone, fracture of the basket, or loss of the stone from the basket. **Note:** The handle will only move in one direction unless the ratchet is released.
2. After the stone has been fractured, assess the need to remove the fractured stone from the duct.
3. Release the ratchet on the handle of the lithotripter to remove the basket.

DEVICE CLEANING AND DISPOSAL

1. Upon completion of the procedure, clean and prepare the handle for future use following the cleaning instructions that follow. During cleaning, inspect the integrity and the function of the device to determine advisability of reuse. If kinks, bends or breaks exist, do not use. Dispose of the device per institutional guidelines for biohazardous medical waste.

CLEANING INSTRUCTIONS

Equipment / Supplies

- Disposable lint-free cloth
- Soft-bristled brush
- Cold tap water
- pH neutral detergent (i.e. Steris Prolystica etc.)

Cleaning

1. Immediately after use, submerge the handle in a cleaning solution using pH neutral detergent and cold tap water. Soak the handle for twenty (20) minutes.
2. Wipe the handle using a clean, lint free cloth soaked in cleaning solution for thirty (30) seconds.
3. Gently brush the handle with a soft nylon bristle brush for one (1) minute to remove all foreign matter.
4. Rinse the handle thoroughly using cold running water for one (1) minute.
5. Ultrasonically clean the handle using a cleaning solution of pH neutral detergent and cold tap water prepared in the Ultrasonic Cleaner for a minimum of 15 minutes per the ultrasonic cleaner manufacturer's instructions.
6. Remove the handle from the ultrasonic cleaner, then rinse thoroughly with cold tap water for one (1) minute.
7. Gently wipe the handle dry with a lint-free cloth.
8. Package the handle for storage. **Note:** Prior to packaging the handles for storage, follow the inspection steps listed below to determine advisability of reuse.

INSPECTION OF DEVICE

- Upon cleaning, inspect integrity and function of the device. If bends or breaks exist, do not use.
- Examine the Luer lock fittings on handle for a proper fit.
- Ratchet on the handle must be attached securely to allow rotation in one direction.

EQUIPMENT REQUIRED

- Sohendra Lithotripsy Handle
- Cook Mechanical Lithotripsy Cable Sheath (sold separately) - SLC, TTCL
- Wire Cutters (sold separately)

PATIENT COUNSELING

Please inform the patient as necessary of the relevant warnings, precautions, contraindications, measures to be taken and limitations of use that the patient should be aware of.

SERIOUS INCIDENT REPORTING

If any serious incident has occurred in relation to the device this should be reported to Cook Medical and the competent authority of the country where the device was used.

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© COOK 2021
William A. Cook Australia Pty. Ltd.
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.