



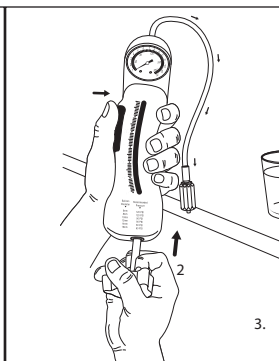
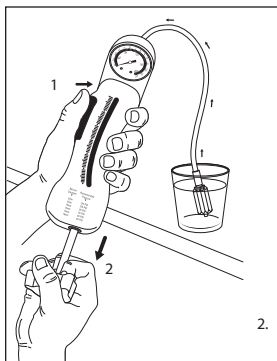
|          |   |
|----------|---|
| EN<br>4  | <b>Quantum™ Inflation Device</b>          |
| CS<br>6  | <b>Plnicí zařízení Quantum™</b>           |
| DA<br>8  | <b>Quantum™ inflationspistol</b>          |
| NL<br>11 | <b>Quantum™ inflatie-instrument</b>       |
| FR<br>13 | <b>Dispositif d'inflation Quantum™</b>    |
| DE<br>16 | <b>Quantum™ Inflationsgerät</b>           |
| EL<br>19 | <b>Συσκευή πλήρωσης Quantum™</b>          |
| HU<br>22 | <b>Quantum™ feltöltőeszköz</b>            |
| IT<br>25 | <b>Dispositivo di gonfiaggio Quantum™</b> |
| NO<br>28 | <b>Quantum™ fyllingsanordning</b>         |
| PL<br>30 | <b>Urządzenie do wypełniania Quantum™</b> |
| PT<br>33 | <b>Dispositivo de insuflação Quantum™</b> |
| ES<br>36 | <b>Dispositivo de hinchado Quantum™</b>   |
| SV<br>38 | <b>Quantum™ fyllningsanordning</b>        |

- 1 I / D button  
 Tlačítko pro plnění / vyprazdňování  
 I/D-knap  
 I / D knop  
 Bouton I/D  
 I/D-Taste  
 Κουμπί συμπίεσης/αποσυμπίεσης  
 (I / D)  
 Feltöltő/leeresztő gomb  
 Pulsante I/D  
 I/D-knapp  
 Przycisk napełniania/oprózniania  
 Botão I / D  
 Botón de hinchado/deshinchado  
 I/D-knapp

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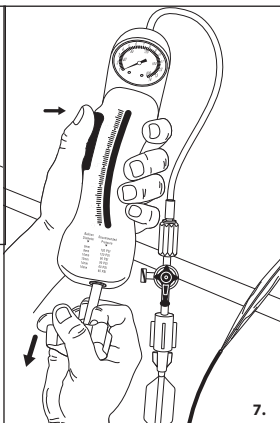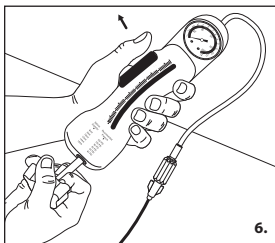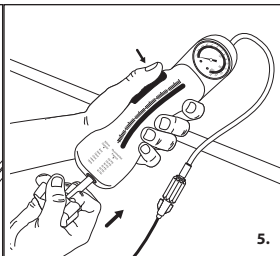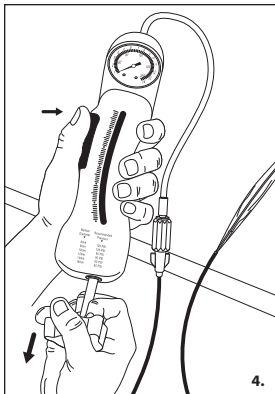**INTENDED USE**

This device is used to inflate, deflate and monitor pressure of the Cook line of balloon dilators during GI balloon dilation.

This device is indicated for adult use only.

**NOTES**

Do not use this device for any purpose other than the stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device is restricted to a trained healthcare professional.

This device is supplied non-sterile and may not be sterilized by any method.

**CONTRAINDICATIONS**

Those specific to the primary endoscopic procedure to be performed in gaining access to the desired dilation site.

Those specific to dilation include, but are not limited to: uncooperative patient; asymptomatic rings, webs, or strictures; inability to advance the balloon dilator through the strictured area; coagulopathy; known or suspected perforation; severe inflammation or scarring near the dilation site.

**POTENTIAL COMPLICATIONS**

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

**PRECAUTIONS**

The inflation device is used in conjunction with a balloon dilator. Refer to the balloon dilator instructions for use for specific inflation instructions.

**WARNINGS**

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

**SYSTEM PREPARATION**

1. Examine the features of the Q.I.D. (See fig 1). Ensure the pressurize/depressurize (P/D) knob is rotated completely counterclockwise and the plunger is pushed completely forward. To move the plunger, depress and hold the inflate/deflate (I/D) button. Release the I/D button to lock the plunger in position.

2. Submerge the connecting tube in a minimum of 60 ml of water or saline.
3. Fill the inflation device with a minimum of 40 ml and a maximum of 50 ml of fluid by depressing the I/D button while simultaneously grasping and pulling back on the P/D knob. (See fig. 2) Release the I/D button to lock the plunger in place.
4. To completely purge the inflation device of air, hold the Q.I.D. upright and depress the I/D button while grasping and slowly pushing forward on the P/D knob. (See fig. 3) Release the I/D button to lock the plunger in position.
5. Attach the Q.I.D. connecting tube to the inflation hub of the balloon dilator. Secure by rotating the fittings. **Note:** If use of the stopcock is desired, attach the appropriate fittings to the balloon dilator and the Q.I.D. connecting tube. To secure the attachment, rotate the fittings. The arm of the stopcock indicates which port is closed.
6. Aspirate all residual air from balloon.
7. Depress the I/D button and pull back the P/D knob until the vacuum is indicated on the pressure gauge (See fig. 4). While maintaining a vacuum with the P/D knob, release the I/D button. Release the P/D knob. The balloon dilator is now ready for placement through the accessory channel of the endoscope.

## **INSTRUCTIONS FOR USE**

### **Balloon Inflation**

1. For rapid inflation, depress the I/D button and push the P/D knob forward until significant resistance is met. (See fig. 5) Maintain pressure on the P/D knob while releasing the I/D button. (See fig. 6)
2. To attain specified balloon pressure for the dilator being used, rotate the P/D knob clockwise. Refer to the balloon dilator labeling for recommended inflation pressure. Pressure may be decreased by rotating the P/D knob counterclockwise.

### **Balloon Deflation**

3. To decrease the balloon pressure and deflate the balloon, hold the inflation device while rotating the P/D knob counterclockwise until pressure decreases below 20 psi. Depress the I/D button and pull back on the P/D knob, creating negative pressure as seen on the pressure gauge. (See fig. 7) Release the I/D button to lock in negative pressure.
4. Maintain negative pressure and observe the balloon for complete deflation. Negative pressure may be increased by continued counterclockwise rotation of the P/D knob. **Note:** The indicator on the pressure gauge should register in the red, vacuum portion of the gauge during deflation.

5. If the balloon is not completely deflated, disconnect the Q.I.D. connecting tube from the balloon and purge the inflation device of air. To purge the inflation device of air, hold upright and depress the I/D button while grasping and slowly pushing forward on the P/D knob. **Note:** If an optional stopcock is attached, open the stopcock to the atmosphere and purge the inflation device of air by closing the port to the balloon catheter. Depress the I/D button and push forward on the P/D knob.
6. Reattach the fittings and create negative pressure by depressing the I/D button and pulling back on the P/D knob. Repeat as necessary until the balloon is completely deflated. **Note:** If the stopcock is attached, turn the stopcock to close the open port and create negative pressure.

## DISPOSAL OF DEVICE

Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.

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© 2019 Cook Medical
**Wilson-Cook Medical, Inc.**
**Cook Medical Europe Ltd.**