EN

Pancreatic Dilation Catheters

DA

Katetre til pancreasdilatation

NL

Pancreasdilatatiekatheters

FR

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ENGLISH

INTENDED USE

This device is used to dilate pancreatic strictures or stenotic minor papilla.

NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to ERCP.

POTENTIAL COMPLICATIONS

Those associated with ERCP include, but are not limited to: pancreatitis, cholangitis, aspiration, perforation, hemorrhage, infection, sepsis, allergic reaction to contrast or medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTION

Refer to package label for minimum channel size required for this device.

INSTRUCTIONS FOR USE

1. Introduce device into accessory channel of endoscope over pre-positioned wire guide. Advance in small increments until device is endoscopically visualized exiting scope.
2. Proceed with cannulation.
3. Under fluoroscopic monitoring, advance catheter to appropriate position to dilate stricture.
4. Proceed with dilation.

Upon completion of procedure, dispose of device(s) per institutional guidelines for biohazardous medical waste.

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© 2013 Cook Medical
Wilson-Cook Medical, Inc.
Cook Ireland Ltd.