EN	Wire Guide Locking Device
CS	Aretátor vodícího drátu
DA	Låseanordning til kateterleder
NL	Voerdraadvergrendelinstrument
FR	Dispositif de verrouillage de guide
DE	Draht-Fixierungssystem
EL	Συσκευή ασφάλισης συρμάτινου οδηγού
HU	Vezetődrót rögzítő eszköz
IT	Dispositivo di blocco della guida
NO	Låseutstyr på ledevaieren
PL	Nakładka blokująca przewadnik
PT	Dispositivo de fixação de fio guia
ES	Dispositivo para fijación de guías
SV	Ledarens låsanordning

18601/1013

ENGLISH

INTENDED USE

This device is an accessory to be used with endoscopic biliary devices to lock the wire guide(s) in place during ERCP procedures.

NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Notify Cook for return authorization.

Do not use this device for any purpose other than the stated intended use.

Store in a dry location, away from temperature extremes.

Do not reuse, reprocess or resterilize this device.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

None known.

POTENTIAL COMPLICATIONS

This device is used with Cook biliary devices. Read appropriate Instructions for Use for complications associated with the procedure and device being used.

INSTRUCTIONS FOR USE

Apply water-soluble lubricant to top of cap as needed to ease passage of accessories.

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Wilson-Cook Medical, Inc.
Cook Ireland Ltd.