COOK

MEDICAL

EN
2

Savary-Gilliard® Dilator

Instructions for Use

FR
5

Dilatateur Savary-Gilliard®

Mode d'emploi

PT
9

Dilatadores Savary-Gilliard®

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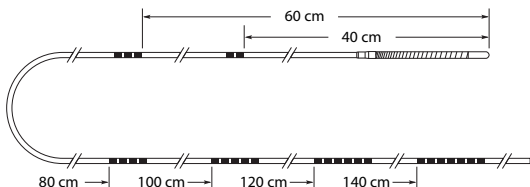
ES
12

Dilatador Savary-Gilliard®

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* 1 8 8 5 1 / 0 3 2 1 *

1.

ENGLISH

INTENDED USE

This device is used over a pre-positioned Savary-Gilliard wire guide for dilation of esophageal strictures.

NOTES

Do not use this device for any purpose other than stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Contraindications include those specific to upper GI endoscopy.

Contraindications to dilation include, but are not limited to: uncooperative patient • asymptomatic strictures • inability to advance the dilator through the strictured area • coagulopathy • known or suspected perforation • severe inflammation or scarring near the dilation site • recent myocardial infarction • active ulcer and severe cervical arthritis.

WARNINGS

This device is supplied non-sterile and is reusable if the device integrity is intact.

If the package is opened or damaged when received, do not use.

Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

The maximum service temperature for the dilator is 50°C. Exposing the dilator to temperatures greater than 50°C may compromise the performance of the device.

POTENTIAL COMPLICATIONS

Potential complications associated with upper gastrointestinal endoscopy and esophageal dilation include, but are not limited to: perforation • hemorrhage • aspiration • fever • infection • allergic reaction to medication • hypotension • respiratory depression or arrest • cardiac arrhythmia or arrest.

PRECAUTIONS

Reusability of a device depends, in large part, on care of device by user. Factors involved in prolonging life of this device include, but are not limited to: thorough cleaning following instructions included in this booklet.

During cleaning, inspect the integrity and the function of the device to determine advisability of reuse. If kinks, bends or breaks exist, do not use.

INSTRUCTIONS FOR USE

Caution: These devices are supplied non-sterile and have not been high level disinfected prior to packaging.

1. Prior to and after each use, visually inspect the dilator with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. If no abnormality is detected, proceed to the next step.
2. Perform screening endoscopy to identify strictured area.
3. Introduce a Savary-Gilliard Wire Guide (sold separately), floppy tip first, into accessory channel of endoscope and advance until it is endoscopically visualized well beyond tip of scope.
4. When wire guide is in position well beyond strictured area, slowly begin to withdraw endoscope in 5-10 cm increments while simultaneously advancing wire guide in 5-10 cm increments to ensure wire guide remains in position. **Caution:** Continuous fluoroscopic monitoring of wire guide is essential in order to ensure proper placement and positioning of wire guide.
5. When endoscope is removed completely, fluoroscopically confirm that wire guide has not been displaced. **Note:** Markings on distal portion of wire guide may also be utilized to determine wire guide position. Markings occur in 20 cm increments from 40 to 140 cm. (*See fig. 1*)
6. Generously lubricate dilator and advance it over pre-positioned wire guide to strictured area.
7. Proceed with esophageal dilation. **Note:** Successive dilations can be accomplished by repeating steps 5-7.
8. Upon completion of esophageal dilation, remove dilator and wire guide from patient.
9. Clean and prepare dilator for future use following cleaning instructions below. During cleaning, inspect integrity and function of device to determine advisability of reuse. If kinks, bends or breaks exist, do not use. Dispose of device per institutional guidelines for biohazardous medical waste.

CLEANING INSTRUCTIONS

Caution: Savary-Gilliard Dilators cannot be autoclaved, EO sterilized, dry heat sterilized, or gamma irradiated.

Note: Betadine and acetone are not recommended for use for cleaning or disinfection of Savary-Gilliard Dilators.

Note: The following cleaning instructions have been validated using the Steris Prolystica Enzymatic Cleaner®.

1. Immediately after use, soak dilator in a solution of warm tap water and enzymatic cleaning solution. **Note:** Refer to enzymatic cleaning solution instructions for recommended duration of cleaning, temperature, and concentration of cleaning solution.
2. Scrub external surfaces of dilator with warm water and enzymatic solution. Using a Savary Cleaning Brush (sold separately), brush wire guide lumen of dilator.
3. Rinse dilator thoroughly with clean, running water. Attach suction to tapered end of dilator or utilize Savary-Gilliard Dilator Washer flushing adapter (sold separately) to suction water through wire guide lumen.
4. Wipe external portions of dilator dry with a lint-free cloth. Use medical grade (oil-free) forced air to dry wire guide lumen of dilator.
5. Proceed with high-level disinfection of dilator.

HIGH LEVEL DISINFECTION

Note: A list of disinfectant solutions validated for use with this device is as follows:

- Metricide 28® (Glutaraldehyde based solution)
 - Cidex OPA® (Ortho-phthalaldehyde base solution)
 - Revital-Ox® Resert (Hydrogen peroxide based solution)
1. Completely immerse dilator in a disinfectant. **Note:** Refer to disinfectant solution instructions for recommended duration of immersion, temperature, and concentration required for high level disinfection. If difficulty is encountered with solution penetrating lumen, we recommend attaching suction to tapered end of dilator or utilize a Savary-Gilliard Dilator Washer flushing adapter (sold separately). **Note:** Discoloration of dilator may occur following use of these solutions. This is not harmful to device.
 2. Rinse dilator thoroughly with clean, running water, utilizing above method for thoroughly rinsing lumen. Attach suction to tapered end of dilator or utilize Savary-Gilliard Dilator Washer flushing adapter (sold separately) to suction water through wire guide lumen. **Note:** Proper rinsing with clean water and drying of dilator is necessary following use of these solutions.
 3. Dry external portions of dilator completely with a lint-free cloth. Use medical grade (oil-free) forced air to dry wire guide lumen. Ensure that both the external and internal surfaces of the dilator are dry prior to packaging and/or storing the devices.

4. Upon cleaning, inspect integrity and function of device. If kinks, bends or breaks exist, do not use.

PACKAGING / STORAGE AFTER INITIAL USE

1. Following cleaning and disinfection, package the dilator per institutional guidelines and best practices.
2. Store the dilator in a dry environment, away from temperature extremes, referring to institutional guidelines and best practices.

TRANSPORTATION

1. Secure the dilator in their packaging and ensure that they do not come into contact with any contaminated surfaces while transporting them, referring to institutional guidelines and best practices.
2. Visually inspect the dilators with particular attention to kinks, bends and breaks after the dilators have been brought to their new location. If an abnormality is detected that would prohibit proper working condition, do not use.

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A symbol glossary can be found at cookmedical.com/symbol-glossary

FR Pour un glossaire des symboles, consulter le site Web cookmedical.com/symbol-glossary
• PT Existe um glossário de símbolos em cookmedical.com/symbol-glossary • ES En cookmedical.com/symbol-glossary puede consultarse un glosario de símbolos

If symbol appears on product label, X = maximum insertion portion width

FR Si le symbole est visible sur l'étiquette du produit, X = largeur maximale de la section d'insertion • PT Se o símbolo aparecer no rótulo do produto, X = largura máxima da parte introduzida • ES Si el símbolo aparece en la etiqueta del producto, X = anchura máxima de la parte que se va a insertar

If symbol appears on product label, X = minimum accessory channel

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If symbol appears on product label, X = wire guide compatibility

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If symbol appears on product label, X = quantity per box

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Savary-Gilliard is a registered trademark of Bredam Medical Distribution SA.

Wilson-Cook Medical, Inc.
4900 Bethania Station Road
Winston-Salem, North Carolina
27105
USA

RxOnly