COOK®

MEDICAL

CE

0123

EN

2

The Web® Extraction Basket

CS

3

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DA

4

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NL

5

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FR

6

Panier d'extraction The Web®

DE

8

The Web®-Extraktionskörbchen

EL

9

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* 1 8 8 7 7 / 0 6 1 9 *

INTENDED USE

This device is used for the endoscopic removal of biliary stones and foreign bodies.

NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Contraindications include those specific to the primary endoscopic procedure to be performed in conjunction with the removal of a foreign body.

Perforation.

If device is to be used for removal of biliary stones, contraindications include an ampullary opening inadequate to allow for unimpeded passage of stone and basket.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy and ERCP include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest.

Others associated with use of an extraction basket include, but are not limited to: impaction of object, aspiration of foreign body, localized inflammation, pressure necrosis.

PRECAUTIONS

Refer to package label for minimum channel size required for this device.

This device is compatible with the Soehendra® Lithotripter.

Device should never be coiled in less than an 8 inch (20 cm) diameter.

Pulling on the sheath while advancing or retracting basket may damage device, rendering it inoperable.

WARNINGS

When retrieving a foreign body from upper GI tract, extreme caution must be exercised to avoid occlusion of airway.

Surgical intervention may be required if impaction occurs.

INSTRUCTIONS FOR USE

1. With basket fully retracted into sheath, insert device into endoscope accessory channel and advance in short increments until basket sheath exits endoscope.
2. Endoscopically or fluoroscopically visualize extraction target and advance basket to appropriate position. Fluoroscopic visualization may be enhanced by injecting contrast via side arm fitting. Place sheath of basket beyond target before extending basket to entrap extraction target.
3. After confirming desired position of basket sheath relative to target, advance basket out of sheath. Place basket around extraction target and slowly retract handle until entrapment is achieved. **Note:** It may be necessary to move basket back and forth along target in order to capture object.

4. Withdraw basket into channel until foreign body is supported against endoscope tip, then withdraw endoscope from patient. Maintain endoscopic visualization to ensure continued entrapment of object. **Caution:** If difficulty is encountered when removing basket from duct, moderate force may be applied by pulling on handle to manually fracture stone. If passage is still restricted, surgical intervention may be necessary.
5. Prepare or dispose of extracted specimen per institutional guidelines.

Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.

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© 2019 Cook Medical
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.