EN	Cytology Brush
CS	Cytologický kartáček
DA	Cytologibørste
NL	Cytologieborstel
FR	Brosse de cytologie
DE	Zytologiebürste
EL	Ψήκτρα κυτταρολογίας
HU	Citológiai kefe
IT	Spazzolino citologico
NO	Cytologibørste
PL	Szczoteczka cytologiczna
PT	Escova de citologia
ES	Cepillo de citología
SV	Cytologiborste

ENGLISH

INTENDED USE

This device is used to collect cells from the bronchi and upper and lower gastrointestinal tracts.

NOTES

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to the primary endoscopic procedure to be performed in gaining access to desired cytology site.

POTENTIAL COMPLICATIONS

Those associated with pulmonary or gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTION

Refer to package label for minimum channel size required for this device.

INSTRUCTIONS FOR USE

1. Uncoil catheter and straighten completely, then extend and retract brush.
2. Endoscopically inspect area to be sampled.
3. With cytology brush retracted, insert device into accessory channel of endoscope.
4. Advance in short increments until tip of device is visualized exiting endoscope.
5. Advance device to desired sampling site, then advance brush out of sheath by pushing forward on handle.
6. Brush area to be sampled several times to obtain adequate cellular material.
7. Pull brush back into sheath and withdraw device from endoscope.
8. Prepare specimen for examination per institutional guidelines.

Upon completion of procedure, dispose of device(s) per institutional guidelines for biohazardous medical waste.

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Wilson-Cook Medical, Inc.
Cook Ireland Ltd.