COOK®

MEDICAL

CE

0123

EN

2

Spray Catheter

CS

2

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DA

3

Spraykateter

NL

4

Spraykatheter

FR

5

Cathéter spray

DE

6

Sprühkatheter

EL

7

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ENGLISH

INTENDED USE

This device is used in conjunction with an endoscope to spray the gastrointestinal mucosa.

NOTES

Do not use this device for any purpose other than the stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device is restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to ERCP or primary endoscopic procedure to be performed in gaining access to the desired procedure site.

WARNINGS

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to spray solution/media, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTION

Refer to the package label for the minimum channel size required for this device.

INSTRUCTIONS FOR USE

1. Gently advance the stylet completely into the spray catheter and lock the stylet cap into the catheter fitting.
2. Introduce the spray catheter into the accessory channel of the endoscope and advance in short increments until the device is endoscopically visualized exiting the scope.
3. Advance the spray catheter to the procedure site.
4. Using a syringe, inject into the catheter to spray the surrounding gastrointestinal mucosa.

Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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© 2019 Cook Medical
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.