EN Spray Catheter

DA Spraykateter

NL Spraykatheter

FR Cathéter spray

DE Sprühkatheter

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ENGLISH

INTENDED USE

This device is used in conjunction with an endoscope to spray the gastrointestinal mucosa.

NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to ERCP or primary endoscopic procedure to be performed in gaining access to desired procedure site.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Those associated with ERCP include, but are not limited to: pancreatitis, cholangitis, aspiration, perforation, hemorrhage, infection, sepsis, allergic reaction to contrast or medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTION

Refer to package label for minimum channel size required for this device.

INSTRUCTIONS FOR USE

1. Gently advance stylet completely into spray catheter and lock stylet cap into catheter fitting.
2. Introduce spray catheter into accessory channel of endoscope and advance in short increments until device is endoscopically visualized exiting scope.
3. Advance spray catheter to procedure site.
4. Using a syringe, inject into catheter to spray surrounding gastrointestinal mucosa. **Note:** This procedure may be repeated as necessary.

Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.

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© 2013 Cook Medical
Wilson-Cook Medical, Inc.
Cook Ireland Ltd.