**COOK®**

**MEDICAL**

**CE**

**0123**

**EN**

**2**

**Grasping Forceps**

**CS**

**2**

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**DA**

**3**

**Gribetang**

**NL**

**4**

**Grijptang**

**FR**

**5**

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**DE**

**6**

**Greifzange**

**EL**

**6**

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\* 1 8 9 3 0 / 0 6 1 9 \*

**INTENDED USE**

This device is used for the endoscopic removal of polyps or foreign bodies in the GI tract.

**NOTES**

Do not use this device for any purpose other than stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

**CONTRAINDICATIONS**

Those specific to the primary endoscopic procedure to be performed in conjunction with the removal of polyps or foreign bodies.

**WARNINGS**

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

**POTENTIAL COMPLICATIONS**

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction to medication, respiratory depression or arrest, cardiac arrhythmia or arrest.

**PRECAUTION**

Refer to package label for minimum channel size required for this device.

**INSTRUCTIONS FOR USE**

1. With forceps fully retracted into sheath, insert device into accessory channel. Advance device through channel until forceps sheath exits endoscope.
2. After confirming desired position of forceps sheath relative to target, advance forceps by pushing forward on handle. Place forceps around object and slowly retract handle until entrapment is achieved.
3. Withdraw forceps into sheath while maintaining entrapment. Support object against endoscope tip, and withdraw endoscope from patient. Maintain endoscopic visualization to ensure continued entrapment of object. While withdrawing device, wipe secretions from outer sheath.
4. Prepare extracted specimen per institutional guidelines.

**Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.**

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© 2019 Cook Medical
**Wilson-Cook Medical, Inc.**
**Cook Medical Europe Ltd.**