EN

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ENGLISH

INTENDED USE

This device is used for the endoscopic removal of biliary stones.

NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to ERCP and any procedures to be performed in conjunction with balloon stone extraction.

Use of this natural latex rubber balloon is contraindicated in patients with a known hypersensitivity to latex.

POTENTIAL COMPLICATIONS

Those associated with ERCP include, but are not limited to: pancreatitis, cholangitis, aspiration, perforation, hemorrhage, infection, sepsis, allergic reaction to contrast or medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Those that can occur during endoscopic balloon stone extraction include, but are not limited to: stone impaction, localized inflammation, pressure necrosis.

PRECAUTIONS

Refer to package label for minimum channel size required for this device.

Assessment of size of stone and ampullary orifice must be made to determine necessity of sphincterotomy.

INSTRUCTIONS FOR USE

1. Ensure all hubs are secure. Verify balloon integrity prior to use by opening stopcock attached to **balloon inflation port**, attaching enclosed **premeasured syringe** to stopcock, then inflating balloon with **AIR ONLY**. If any leakage is detected, do not use. **Note:** The stopcock is in open position, allowing access to balloon, when two arms are aligned with catheter and syringe. To maintain balloon inflation, turn stopcock arm 90°.
2. With elevator open, advance deflated balloon over a prepositioned wire guide, through accessory channel in short increments until it is visualized exiting endoscope. **Note:** Prior to advancing device over a pre-positioned wire guide, close Tuohy-Borst adapter and flush wire guide lumen with sterile water to facilitate advancement.
3. Position deflated balloon. **Note:** Two radiopaque bands on either side of balloon provide fluoroscopic visualization of balloon position.
4. Inflate balloon. Maintain inflation by turning balloon port stopcock 90° to closed position.
5. Advance .025 inch wire guide through blue port. **Note:** The floppy tip of wire guide may deflect off inflated balloon into desired position. **Note:** If injecting contrast, close Tuohy-Borst adapter and attach syringe to extension tube after removing protective cap.
6. If necessary, stone extraction may now be performed. **Note:** If more than one stone is to be removed, extract one stone at a time.
7. Keeping elevator open, gently withdraw inflated balloon and stone toward papilla under fluoroscopic monitoring. A black 1 cm band precedes balloon by 2 cm allowing endoscopic awareness of balloon approaching papilla. **Warning:** Do not exert excessive pressure on ampulla while extracting stones. If stone does not pass easily, reassess need for sphincterotomy.
8. Repeat extraction process until duct is clear.
9. Once balloon is visualized in duodenum, turn stopcock 90° to open position and deflate balloon.
10. Withdraw deflated balloon from accessory channel. **Note:** When removing balloon from endoscope, wire guides may be left in place to facilitate introduction of other compatible wire-guided devices.

Upon completion of procedure, dispose of device(s) per institutional guidelines for biohazardous medical waste.

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© 2013 Cook Medical
Wilson-Cook Medical, Inc.
Cook Ireland Ltd.