EN 5 **Conquest TTC™ Through-the-Channel Lithotripter Cable**

Instructions for Use

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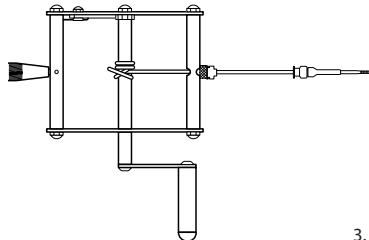
Conquest TTC™ Through-the-Channel Lithotripter Cable

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

Please read all instructions before using this device.

DEVICE DESCRIPTION

The device is to be used with Cook Lithotripsy Extraction Baskets only. Look for the 'Lithotripsy Compatible' symbol on the product labels for Cook Extraction Baskets. The minimum endoscope channel size for the lithotripter cable is provided on the product label.

Patient Population

This device is indicated for adult use only.

Intended Users

Use of this device restricted to a trained healthcare professional.

INTENDED USE

This device is used to mechanically crush stones in the biliary duct when other methods of endoscopic removal have failed.

INDICATIONS FOR USE

This device is used for treatment of biliary obstruction due to gallstones (choledocholithiasis) in adult patients.

CLINICAL BENEFITS

This device relieves jaundice and reduces the risk of infection due to obstruction.

CONTRAINDICATIONS

Contraindications include those specific to ERCP and mechanical lithotripsy.

Contraindications associated with biliary stone extraction, including: an ampullary opening inadequate to allow for the unimpeded passage of the stone and basket.

WARNINGS

- This device is designed for single use only. Attempts to reprocess, sterilize, and/or reuse may lead to contamination with biological or chemical agents and/or mechanical integrity failure of device.
- This device is supplied non-sterile and therefore, should not be used in a sterile field.
- If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.
- Due to the mechanical pressure generated with this device, basket fragmentation in the common bile duct is a possibility that may require surgical intervention.

- Only specified Cook Endoscopy Soft lithotripsy-compatible baskets are recommended for use with the Conquest TTC Lithotripter Cable. Refer to basket label.
- Due to the varying compositions of biliary stones, stone fracture may not be possible. If the stone cannot be fractured, continued rotation of the handle may cause the basket wire to break, thus requiring surgical intervention.
- Device contains nickel which may cause an allergic reaction and other potential systemic complications related to nickel allergy.

PRECAUTIONS

- Do not use this device for any purpose other than stated intended use.
- This device is intended for use with a side-viewing endoscope only.
- Assessment of the size of the stone and ampullary orifice must be made to determine the necessity of sphincterotomy prior to this procedure. The length of the intramural segment must be evaluated before determining the advisability of mechanical lithotripsy. In the event sphincterotomy is required, all necessary cautions, precautions and contraindications must be observed.
- Cook Endoscopy wire guides are recommended for use with this product.
- Injection of contrast during ERCP should be monitored fluoroscopically. Overfilling of the pancreatic duct may cause pancreatitis.
- Wire cutters must be available when using this device.

POTENTIAL COMPLICATIONS

Potential complications associated with ERCP, including: pancreatitis • cholangitis • sepsis • perforation • hemorrhage • aspiration • fever • infection • hypotension • allergic reaction to contrast or medication • respiratory depression or arrest • cardiac arrhythmia or arrest.

Potential complications associated with basket extraction include: impaction of the object • localized inflammation • pressure necrosis.

HOW SUPPLIED

This device is supplied non-sterile and may not be sterilized by any method. Store in a dark, dry, cool place. Avoid extended exposure to light and extremes of temperature and humidity.

INSTRUCTIONS FOR USE

1a. For baskets with removable handle (See fig. 1)

Loosen the sliding handle from the basket wires by holding the thumb grip in one hand and turning the basket wire lock in the direction of the arrow. (See fig. 1) Hold the basket sheath firmly at the accessory channel part of the endoscope and slowly remove handle. Dispose of basket handle per institutional guidelines for

biohazardous medical waste. **Note:** If desired, remove protective coil to expose basket wires. The basket wires can be used to facilitate basket control.

1b. For other lithotripsy compatible baskets (See fig. 2)

Once the stone is completely captured in the basket, make a complete cut, using wire cutters, at the junction of the 10 French sheath and the 7 French sheath (See fig. 2) and dispose of basket handle per institutional guidelines for biohazardous medical waste.

- Keeping the endoscope as straight as possible, remove the outer sheath of the basket slowly. As distal tip of the basket sheath clears endoscope elevator, gently lift the elevator to lock the basket wires in place. Then, continue removing the outer sheath. Dispose of outer sheath per institutional guidelines for biohazardous medical waste.
- Introduce the lithotripsy cable over the basket wires until the wires exit the proximal end of the cable.
- Grasp the basket wires exiting the cable. Release the elevator and continue to advance the cable until the distal tip reaches the basket within the duct.
- If contrast injection is desired, attach adapter provided. **Note:** To attach adapter, insert the basket wires through the male Luer lock of the Tuohy-Borst adapter. Continue to slide the adapter over the basket wires until the male Luer lock engages the female Luer lock of the cable. Tighten the male Luer lock to prevent leakage during injection. The Tuohy-Borst adapter need not be attached if additional contrast is not required for the procedure.
- Pass the basket wires through the male Luer lock on the lithotripsy handle; continue to pass the wires through the rotating-rod opening of the handle.
- Connect the male Luer lock of the lithotripter handle to the female Luer lock of the adapter (if applicable). Tighten female Luer lock to prevent back flow during injection. Wrap the basket wires around the rotating rod to secure. (See fig. 3)
- To fracture the stone, apply pressure by slowly rotating the handle. **Note:** Slow, constant pressure is often the most effective method of fracturing a stone. Sudden release of pressure may indicate the fracture of the stone, fracture of the basket, or loss of the stone from the basket. **Warning:** If basket fragmentation should occur, surgical intervention may be necessary. **Note:** The handle will only move in one direction unless the ratchet is released.
- After the stone has been fractured, assess the need to remove the fractured stone from the duct.
- Release the ratchet on the lithotripter handle to remove the basket. Examine the basket to confirm that it is not broken or damaged in any way, then dispose of the basket per institutional guidelines for biohazardous medical waste.

EQUIPMENT REQUIRED

- Conquest TTC Lithotripsy Cable
- Sohendra Lithotripsy Handle (sold separately)
- Wire Cutters (sold separately)
- Tuohy-Borst adapter - if contrast injection is desired

DISPOSAL OF DEVICE

Dispose of the lithotripsy cable per institutional guidelines for biohazardous medical waste.

PATIENT COUNSELING

Please inform the patient as necessary of the relevant warnings, precautions, contraindications, measures to be taken and limitations of use that the patient should be aware of.

SERIOUS INCIDENT REPORTING

If any serious incident has occurred in relation to the device this should be reported to Cook Medical and the competent authority of the country where the device was used.

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© COOK 2021
William A. Cook Australia Pty. Ltd.
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.