18968/1013

ENGLISH

INTENDED USE

This device is used for the endoscopic removal of biliary stones and foreign bodies.

NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Those specific to primary endoscopic procedure to be performed in conjunction with removal of a stent (if applicable) or foreign body.

Perforation.

If device is to be used for biliary stone extraction, contraindications include an ampullary opening inadequate to allow for unimpeded passage of the stone and basket.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy and ERCP include, but are not limited to: pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest.

Others associated with basket extraction include, but are not limited to: impaction of object, aspiration of foreign body, localized inflammation, pressure necrosis.

PRECAUTIONS

Refer to the package label for the minimum channel size required for this device.

If device is to be used for removal of biliary stones, assessment of stone size and ampullary orifice must be made to determine necessity of sphincterotomy.

This device should never be coiled in less than an 8-inch (20 cm) diameter.

WARNINGS

This device is **not** compatible with the Soehendra® Lithotriptor or any other mechanical lithotriptor.

When retrieving foreign body from upper GI tract, extreme caution must be exercised to avoid occlusion of airway.

Surgical intervention may be required if impaction occurs.

INSTRUCTIONS FOR USE

Note, if applicable: Flush wire guide port with sterile water to facilitate easy passage of wire guide.

1. With basket fully retracted into sheath, insert device into endoscope accessory channel. Advance device through channel, in short increments, until basket sheath exits endoscope.
2. Endoscopically or fluoroscopically visualize extraction target and advance basket to appropriate position. If applicable, fluoroscopic visualization may be enhanced by injecting contrast via side arm fitting. Place sheath of basket beyond target before extending basket to entrap extraction target. **Note, if applicable:** Tighten connector cap on handle before injection of contrast. (See fig. 1)
3. Confirm desired position of basket sheath relative to target. Advance basket out of sheath. **Caution:** Pulling on sheath while advancing or retracting basket may damage device, rendering it inoperable.
4. Place basket around extraction target or foreign body and slowly retract handle until entrapment is achieved. **Note:** It may be necessary to move basket back and forth along target in order to capture target.
5. **Stone removal:** Withdraw basket into the channel until stone is supported against endoscope tip. For multiple stones, repeat process until all stones have been removed, then withdraw endoscope from patient. **Caution:** If difficulty is encountered when removing basket from duct, moderate force may be applied by pulling on handle to manually fracture stone. If passage is still restricted, surgical intervention may be necessary.
6. **Foreign body removal:** Withdraw basket into channel while maintaining entrapment. Support object against endoscope tip and withdraw endoscope from patient. Maintain endoscopic visualization to ensure continued entrapment of object.
7. Prepare or dispose of extracted specimen per institutional guidelines.

Upon completion of procedure, dispose of device(s) per institutional guidelines for biohazardous medical waste.

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© 2013 Cook Medical
Wilson-Cook Medical, Inc.
Cook Ireland Ltd.