- EN**
4 Cook Airway Exchange Catheters with Rapi-Fit™ Adapters
Instructions for Use
- CS**
5 Výměnné katetry Cook pro dýchací cesty s adaptéry Rapi-Fit™
Návod k použití
- DA**
7 Cook luftvejsudskiftningskatetre med Rapi-Fit™ adaptere
Brugsanvisning
- DE**
8 Cook-Umintubationskatheter mit Rapi-Fit™-Adapter
Gebrauchsanweisung
- EL**
10 Καθετήρες αλλαγής αεραγωγών Cook με προσαρμογείς Rapi-Fit™
Οδηγίες χρήσης
- ES**
12 Catéteres de intercambio para vía aérea de Cook con adaptadores Rapi-Fit™
Instrucciones de uso
- FR**
13 Cathéters Cook pour échange endotrachéal avec adaptateur Rapi-Fit™
Mode d'emploi
- IT**
15 Cateteri Cook Airway Exchange con adattatori Rapi-Fit™
Istruzioni per l'uso
- NL**
17 Cook wisselkatheters voor tubes in de luchtwegen, met Rapi-Fit™ adapters
Gebruiksaanwijzing
- NO**
18 Cook luftveisutskiftingskatetre med Rapi-Fit™ adaptere
Bruksanvisning
- PT**
20 Cateteres de substituição para vias aéreas da Cook com adaptadores Rapi-Fit™
Instruções de utilização
- SV**
22 Cook-utbyteskatetrar med Rapi-Fit™-adapter
Bruksanvisning

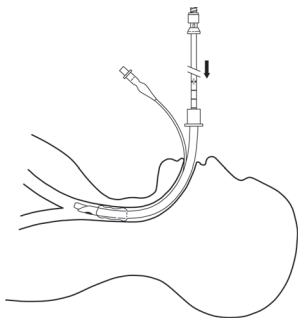
Fig. 1

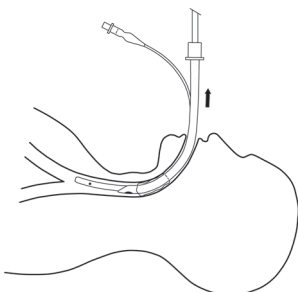
Fig. 2

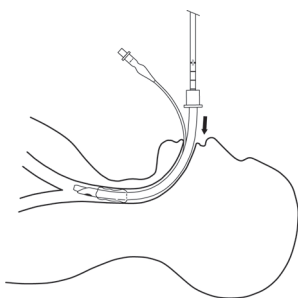
Fig. 3

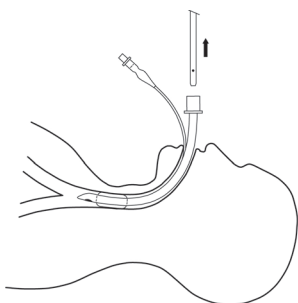
Fig. 4

COOK AIRWAY EXCHANGE CATHETERS WITH RAPI-FIT™ ADAPTERS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician or properly licensed practitioner.

DEVICE DESCRIPTION

The Cook Airway Exchange Catheter is a blunt-tipped radiopaque catheter with centimeter markings to facilitate placement. The catheter is supplied with a Rapi-Fit Adapter (with either a 15 mm connector or a Luer lock connector), permitting use of ventilatory devices during exchange procedure. Additional product specifics are outlined in the table below.

Order Number	Used for:
C-CAE-8.0-45 (inner diameter: 1.6 mm)	For use in replacement of endotracheal tubes whose inner diameter (ID) is 3 mm or larger.
C-CAE-11.0-83 (inner diameter: 2.3 mm)	For use in replacement of endotracheal tubes whose inner diameter (ID) is 4 mm or larger.
C-CAE-11.0-100-DLT-EF C-CAE-11.0-100-DLT-EF-ST (inner diameter: 2.3 mm)	For use in replacement of double-lumen endotracheal tubes or endotracheal tubes whose inner diameter (ID) is 4 mm or larger.
C-CAE-14.0-83 (inner diameter: 3 mm)	For use in replacement of endotracheal tubes whose inner diameter (ID) is 5 mm or larger.
C-CAE-14.0-100-DLT-EF C-CAE-14.0-100-DLT-EF-ST (inner diameter: 3 mm)	For use in replacement of double-lumen endotracheal tubes or endotracheal tubes whose inner diameter (ID) is 5 mm or larger.
C-CAE-19.0-83 (inner diameter: 3.4 mm)	For use in replacement of endotracheal tubes whose inner diameter (ID) is 7 mm or larger.

INTENDED USE

The Cook Airway Exchange Catheter is intended for uncomplicated, atraumatic endotracheal tube exchange.

CONTRAINDICATIONS

None known

WARNINGS

- Attention should be paid to insertion depth of catheter into patient's airway and correct tracheal position of replacement endotracheal tube. Markers on the Cook Airway Exchange Catheter refer to distance from tip of catheter. Catheter and endotracheal tube should not be advanced beyond the carina.
- Ensure proper sizing of the Cook Airway Exchange Catheter within a **double-lumen endotracheal tube**. Failure to do so may cause small fragments to be shaved off during removal of the Cook Airway Exchange Catheter.
- To avoid barotrauma, ensure that the tip of the CAE catheter is always above the carina, preferably 2-3 cm.

PRECAUTIONS

- The product is intended for use by physicians trained and experienced in airway management techniques. Standard techniques for placement and exchange of endotracheal tubes should be employed.
- This product is not intended for intravascular use.

POTENTIAL ADVERSE EVENTS

- Barotrauma
- Perforation of the bronchi or lung parenchyma

INSTRUCTIONS FOR USE

Use of Rapi-Fit™ Adapter

A ventilatory device may be used at any time during the endotracheal tube exchange procedure by utilizing the attached Rapi-Fit Adapter.

1. To attach Rapi-Fit Adapter, position the adapter on catheter, then push the white collar forward and lock into position.

NOTE: Use of a high-pressure oxygen source should only be considered if the patient has sufficient egression of the insufflated gas volume.

2. To remove the adapter, pull the white collar back to release, then remove from catheter.

Endotracheal Tube Exchange

1. Before advancing the Cook Airway Exchange (CAE) Catheter into the endotracheal tube to be replaced, confirm correct endotracheal tube position.

- Using the outer margin of the patient's mouth or nasal orifice as a landmark, note the marking on the endotracheal tube. A piece of tape or other marker may be placed on the CAE catheter at the corresponding distance from the tip to aid in correct placement within the endotracheal tube.
- Advance the CAE catheter, sideported end first, into the endotracheal tube to be replaced. **(Fig. 1)**
NOTE: It is recommended that a sterile lubricant be applied to the orifice of the endotracheal tube prior to introduction of CAE catheter.
CAUTION: To avoid barotrauma, ensure that the tip of the CAE catheter is always above the carina, preferably 2-3 cm.
NOTE: If a high-pressure oxygen source is used for insufflation (e.g., jet ventilator), begin at lower pressure and work up gradually. Rising chest wall, pulse oximetry and oral air flow should be carefully monitored.
- Properly position the CAE catheter within the endotracheal tube by aligning the appropriate centimeter mark on the CAE catheter with the corresponding centimeter mark on the endotracheal tube. This placement is determined by visualizing the indicated centimeter length of the endotracheal tube, in place, as shown on its surface scale. (For example, an endotracheal tube that has been shortened to 24 cm should have the 24 cm marker of the CAE catheter aligned at the 24 cm mark of the endotracheal tube.)
- Fully deflate the cuff of the endotracheal tube and remove the Rapi-Fit Adapter, while maintaining the position of the CAE catheter. Remove the endotracheal tube, leaving the CAE catheter in place. **(Fig. 2)**
- Maintaining position of the CAE catheter, using patient's mouth or nasal orifice (depending on approach) as a landmark, advance the new endotracheal tube over the CAE catheter and position appropriately. **(Fig. 3)**
NOTE: It is recommended that a sterile lubricant be applied to the tip of the endotracheal tube prior to advancing the tube.
- Remove the CAE catheter and inflate the balloon cuff of the new endotracheal tube. Re-establish ventilatory device use and secure the new endotracheal tube in place. Confirm its position using standard methods (e.g., capnography, breath sounds and chest x-ray). **(Fig. 4)**

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

This product is protected by U.S. patent 5,052,386.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd