COOK

MEDICAL

CE
0123

EN

2

Cook Chest Drain Valve

Instructions for Use

DA

3

Cook-thoraxdræn ventil

Brugsanvisning

DE

4

Cook Thoraxdrainageventil

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EL

5

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C _ T _ C D V _ R E V 4

COOK CHEST DRAIN VALVE

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Cook Chest Drain Valve is a hard, clear, plastic tube with a soft, polyurethane, one-way valve inside. Multi-purpose adapters are affixed on each end of the tube. The one-way valve is sensitive to pressure changes. It pushes open, allowing air to flow out of the pleural space, and then closes to prevent air from reentering.

INTENDED USE

The Cook Chest Drain Valve is intended for use with a pleural drainage catheter to help remove air from the pleural cavity. The Cook Chest Drain Valve can be used for transport of patients who need to have pleural drainage catheters in place for longer durations or for outpatient procedures.

The product is intended for use by physicians trained and experienced in proper positioning and utilization of drainage catheters. Standard techniques for placement of drainage catheters should be employed.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Keep the connecting tube as straight as possible in order to avoid kinking or blocking of the tube and valve.
- Confirm the connecting tube with drainage bag is secured tightly around the Cook Chest Drain Valve to avoid air leakage into the pleural space.
- Secure connecting tube to valve end with direction arrow pointing away from the chest, or the device will not function properly.

INSTRUCTIONS FOR USE

1. Insert the pleural drainage catheter and place in position using standard hospital protocol.
2. Attach and firmly tighten the connecting tube's luer lock directly onto the catheter. This luer fitting may also be attached to an appropriate stopcock.
3. Attach the funnel portion of the connecting tube with the blue end of the Cook Chest Drain Valve. Make sure the fit is firm and airtight.
NOTE: Correct placement and operation is achieved when the arrow on the valve points away from the patient's chest.
4. Attach connection tubing to the clear plastic end of the Cook Chest Drain Valve.
5. Insert the proximal end of connection tubing into a plastic drainage bag.
NOTE: Flapping or pulsating of the valve leaflet indicates increasing and decreasing intrapleural pressure.
6. The drainage catheter or connecting tube may now be firmly taped to the valve to prevent dislodging.
7. Utilize standard hospital protocol to keep the valve system free of clots. If clotting does begin to occur within the Cook Chest Drain Valve, clamp the chest drain tubing, remove the valve and replace it with an alternate sterile one-way drain valve.
8. Suspend the drainage bag at the patient's bedside. When transporting the patient, the drainage bag may be carried or attached to the patient's clothing.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd