EN
3

Cook Esophageal Dilator Set

Instructions for Use

DA
4

Cook oesophagus-dilatatorsæt

Brugsanvisning

DE
5

Cook-ösophagusdilator-Set

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EL
6

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Bruksanvisning

COOK ESOPHAGEAL DILATOR SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Cook Esophageal Dilator Set consists of an esophageal-gastric catheter and an esophageal dilator, both with markings to aid in placement. The sets are available in a range of lengths and French sizes, for both adult and pediatric use.

INTENDED USE

The Cook Esophageal Dilator Set is intended for placement of an esophageal bougie or stent during laparoscopic hiatal hernia repair. The set permits a dilator to be passed over an esophageal-gastric catheter, which serves to guide the dilator into the esophagus, avoiding blind, unguarded passage.

CONTRAINDICATIONS

- Esophageal stricture that does not allow passage of the Cook Esophageal Dilator Set through the mouth and esophagus.
- Abnormal coagulation profile or current anticoagulation therapy.
- Esophageal diverticulae, unless surgical excision is planned at the time of operation.
- Previous history of inability to pass an orogastric tube.

WARNINGS

None known

PRECAUTIONS

- The product is intended for use by physicians trained and experienced in esophageal dilation techniques. Standard techniques for placement of esophageal catheters and dilators should be employed.
- Protect patient's teeth and/or dental work with adequate protection or dental guards.
- Protect patient's lip from pressure of the dilator against teeth.
- Use special caution when using the Cook Esophageal Dilator Set in patients who have had previous esophageal or stomach surgery, esophageal varices, esophageal stricture or coagulopathy.
- **Generously lubricate both the esophageal-gastric catheter and the esophageal dilator prior to use in patient.**
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

POTENTIAL ADVERSE EVENTS

Use of the Cook Esophageal Dilator Set involves potential risks normally associated with placement of esophageal dilators, including, but not limited to:

- Perforation of the esophagus or stomach
- Induction or recurrence of stricture after dilation
- Injury to structures in the mouth, oropharynx, esophagus or stomach during device introduction

INSTRUCTIONS FOR USE

NOTE: If desired, the esophageal dilator may be soaked in warm, sterile solution prior to use.

1. **Generously lubricate the esophageal-gastric catheter with sterile lubricant.**
2. Introduce the lubricated esophageal-gastric catheter through the mouth and advance into the stomach. **NOTE:** A sufficient length of catheter should be advanced to prevent the dilator from advancing beyond the tip of the catheter.
NOTE: The esophageal-gastric catheter may be used for gastric suctioning, if needed.
3. **Generously lubricate the dilator with sterile lubricant.**
4. Advance the dilator down the esophagus over the esophageal-gastric catheter with a gentle twisting motion, while **maintaining** position of the esophageal-gastric catheter.
5. Verify passage of the dilator with the operating physician, to properly monitor depth. The operating physician should observe the dilator advancing into the stomach.
NOTE: If desired, the esophageal-gastric catheter may be removed and a fiberoptic scope positioned inside the dilator. This allows the operating physician to observe for light during placement of the dilator.
NOTE: If surgical palpation of the esophagus is necessary during placement, the dilator may be pulled back while keeping the esophageal-gastric catheter in place.
6. Once the dilator is in place, perform visual inspection to rule out perforation.

7. At the completion of the procedure, remove the esophageal dilator and esophageal-gastric catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.