Barone Jejunostomy Catheter Set	EN
Instructions for Use	3
Barone jenunostomi-katetersæt	DA
Brugsanvisning	4
Barone Jejunostomie-Katheter-Set	DE
Gebrauchsanweisung	5
ΣΕΤ καθετήρα νηστιδοστομίας Barone	EL
Οδηγίες χρήσης	6
Equipo de catéter de yeyunostomía Barone	ES
Instrucciones de uso	7
Set de sonde de Barone pour jéjunostomie	FR
Mode d'emploi	9
Set con catetere per digiunostomia Barone	IT
Istruzioni per l'uso	10
Barone jejunostomiekatheterset	NL
Gebruiksaanwijzing	11
Conjunto de cateter de jejunostomia Barone	PT
Instruções de utilização	12
Barone jejunostomikateterset	SV
Bruksanvisning	14
Barone空肠造瘘导管套装	ZH
使用说明	15

C _ T _ C J C _ R E V . 1

BARONE JEJUNOSTOMY CATHETER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Barone Jejunostomy Catheter Set consists of a jejunostomy catheter with leakproof adapter, 18 gage introducer needle, 15 gage catheter introducer needle, wire guide and dilator.

INTENDED USE

The Barone Jejunostomy Catheter Set is intended for intraoperative feeding jejunostomy.

The product is intended for use by physicians trained and experienced in jejunostomy techniques. Standard techniques for percutaneous placement of wire guides, dilators, and jejunostomy catheters should be employed.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

None known

INSTRUCTIONS FOR USE

1. Place a Babcock clamp or stay suture on the bowel wall at a point 8-14 inches (20-36 cm) beyond the ligament of Treitz, on the antimesenteric border of the jejunum.
2. Select a site distal to the clamp (or suture) for needle introduction. Using a traction/countertraction technique between the clamp (or suture) proximally and a point on the bowel wall distal to the needle site, stretch the jejunal segment taut across the site selected for needle introduction.
3. Advance the 18 gage introducer needle through the serosa and muscularis of the small bowel. **NOTE:** The needle tip should initially be directed parallel to the bowel wall and tunneled through the muscularis and submucosal tissues for 1-2 inches (3-5 cm) before being directed through the mucosal tissue into the bowel lumen.
4. Introduce the wire guide through the introducer needle into the bowel lumen.
5. Remove the introducer needle, leaving the wire guide in place.
6. Manipulate the wire guide into the distal bowel for a distance of 18-24 inches (46-61 cm).
7. Advance the 15 gage catheter introducer needle through the abdominal wall, usually in the left upper quadrant or in a location where the bowel wall catheter entry site can be easily attached to the abdominal wall at the end of the procedure.
8. Place the proximal end of the wire guide through the distal end of the catheter introducer needle and out the hub of the needle from inside to outside of the abdominal wall.
9. Remove the catheter introducer needle.
10. Withdraw the wire guide from its intra-abdominal position through the abdominal wall to an extra-abdominal site, leaving 12-18 inches (30-46 cm) within the bowel lumen.
11. Make a small skin incision at the abdominal wall puncture site and dilate using the 10.0 French dilator.
12. Advance the catheter and internal obturator over the wire guide. Ensure that the wire guide protrudes beyond the proximal end of the catheter when the catheter tip and obturator are at the level of the skin entry site.
13. Introduce the catheter and internal obturator through the abdominal wall using firm pressure at a point 2-3 inches (5-8 cm) from the catheter tip. Ensuring that the end of the wire guide continues to protrude beyond the proximal end of the catheter, advance the catheter beyond the parietal peritoneum for a distance of 8-10 inches (20-25 cm).
14. Again, grasp the catheter 2-3 inches (5-8 cm) from its end and gently advance it over the wire guide through the bowel wall and into the bowel lumen. Continue advancing until almost the entire catheter is seated in the lumen and the suture wing is almost at the level of the skin entry site.
15. Remove the wire guide, leaving the catheter in place.
16. Place a purse-string suture (absorbable or non-absorbable) where the catheter enters the bowel wall. Alternatively, a plication of the bowel wall around the catheter can be performed. In each instance, the bowel wall should be attached to the abdominal wall with several interrupted silk sutures.
17. Secure the catheter to the skin through the holes in the suture wings using several interrupted silk sutures.
18. Remove the internal obturator.
CAUTION: It is imperative that the internal obturator be removed. If left in place, the obturator's pointed end may perforate the bowel wall.
19. Attach the injection port to the end of the catheter. Confirm intraluminal position of the catheter by occluding the bowel distal and proximal to the tip of the catheter within the bowel lumen and injecting saline through the catheter with a 10 ml syringe.
20. Close the abdominal incision in standard fashion.

21. Postoperativt, flush the catheter with 10 ml of saline every 6-8 hours until tube feedings are initiated.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK INCORPORATED 2007
Bloomington, IN 47404 U.S.A.