**EN**  
3

**Fanelli Cholangiography Catheter**

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## FANELLI CHOLANGIOGRAPHY CATHETER

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

### DEVICE DESCRIPTION

The Fanelli Cholangiography Catheter has a flexible distal tip with smooth, atraumatic edges, which facilitate the passage of the catheter into the cystic duct. The integrated wire guide allows passage of the catheter into difficult, small or tortuous ducts. The centimeter markings provide for accurate positioning. The three-way adapter allows saline for flush, contrast for cholangiography, and a wire guide all to be used simultaneously and seamlessly, reducing the risk of introduction of air.

### INTENDED USE

The Fanelli Cholangiography Catheter is intended for diagnostic evaluation of the bile ducts during laparoscopic cholecystectomy procedures.

### CONTRAINDICATIONS

None known

### PRECAUTIONS

- This product is intended for use by physicians trained and experienced in placement of cholangiography catheters. Standard cholangiography techniques should be employed.
- It is recommended that all wire exchanges be performed using fluoroscopic guidance, to maximize patient safety.
- The surgeon should anatomically verify the cystic duct location.
- The cystic duct incision should be of sufficient size to allow introduction of the catheter tip, and should demonstrate the egress of a small amount of bile, confirming appropriate location.
- Check the tightness of all screw connections and stopcocks.
- Note the labeling on the three-way adapter top, indicating placement of syringes with saline and contrast.
- The syringes should be held with the plunger slightly elevated to prevent the injection of air that might be present in the syringe barrel during cholangiography.
- 20 ml syringes with Luer lock adapters are recommended.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

### WARNINGS

- It is vital that appropriate placement be confirmed before catheter is introduced.
- Care should be taken when advancing the catheter, as dissection in a false plane might occur, and perforation or tearing of the duct could occur if excessive force is used.
- Care should be taken when introducing the wire guide; use of excessive force may result in perforation of the duct.
- The wire passage port of the cholangiography catheter should not be pointed toward the face, in case the injection pressure overcomes the pressure relief feature of the silicone valve.

### INSTRUCTIONS FOR USE

The catheter can be used through a laparoscopic cholangiogram clamp or introduced through a separate site and anchored with a free tie or partially occluding clip.

1. Fill a 20 ml Luer lock syringe with saline solution and attach it to the saline port of the cholangiography catheter.
2. Close the contrast stopcock and flush the catheter with approximately 5-10 ml of saline.
3. Fill a 20 ml Luer lock syringe with radiographic contrast material and attach it to the contrast port of the cholangiography catheter.
4. Open both stopcocks, and inject a small amount (1 ml) of saline, which will flow back through the three-way adapter top to enter the contrast syringe. This removes additional air from the system.
5. Close the contrast stopcock, leaving the saline stopcock open for flushing during catheter insertion if necessary.
6. Position the wire guide so that its tip is within the flexible tip of the cholangiography catheter.
7. When using the Fanelli Cholangiography Catheter through a laparoscopic cholangiogram clamp, load the catheter, positioning its tip within the anchoring jaws of the clamp to prevent bending of the catheter's flexible tip during advancement through the laparoscopic trocar valve.
8. Introduce the cholangiography catheter into the opening created in the cystic duct, flushing with saline as necessary to confirm placement.

**NOTE:** If necessary, the wire guide can be advanced into the cystic duct first and the cholangiography catheter then advanced over the wire guide.

**NOTE:** When using the cholangiography catheter through a separate site, introduce through a small trocar device.

9. Appropriate catheter positioning for cholangiography depends on the clinical scenario, but it is generally advisable to have at least 1 cm of the catheter tip within the cystic duct. The catheter is marked every 1 cm, with 5 cm increments indicated by broader markings.
10. If a cholangiogram clamp is not used, a partially occluding clip or tie can be used. Care must be taken not to occlude the soft distal portion of the catheter.
11. Perform cholangiogram by injecting contrast. The catheter is well suited for both static and dynamic cholangiography.
12. If necessary, the cholangiography catheter can be easily advanced into the common bile duct, so that small stones can be flushed free.
13. If common bile duct stones are identified and transcystic common bile duct exploration planned, a 2.0 mm helical stone basket can be introduced through the wire passage port of the cholangiography catheter three-way adapter top, after removing the included wire guide.
14. If laparoscopic endobiliary stenting or common bile duct exploration is to be performed, remove the integrated wire guide once the cholangiography catheter has been directed into the distal common bile duct.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

## REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

## ACKNOWLEDGEMENT

Robert D. Fanelli, M.D., F.A.C.S, Assistant Professor of Surgery, University of Massachusetts Medical School at Berkshire Medical Center, Pittsfield, Massachusetts.

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Robert D. Dr. Fanelli, F.A.C.S, Assistente di chirurgia, University of Massachusetts Medical School at Berkshire Medical Center, Pittsfield, Massachusetts.

This symbol on the label indicates that this device contains phthalates. Specific phthalates contained in the device are identified beside or below the symbol by the following acronyms:

- BBP: Benzyl butyl phthalate
- DBP: Di-n-butyl phthalate
- DEHP: Di(2-ethylhexyl) phthalate
- DIDP: Diisodecyl phthalate
- DINP: Diisononyl phthalate
- DIPP: Diisopentyl phthalate
- DMEP: Di(methoxyethyl) phthalate
- DNOP: Di-n-Octyl phthalate
- DNPP: Di-n-pentyl phthalate

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**MANUFACTURER**

COOK INCORPORATED  
750 Daniels Way  
Bloomington, IN 47404 U.S.A.

[www.cookmedical.com](http://www.cookmedical.com)  
© COOK 2019

<b>EC</b>	<b>REP</b>
-----------	------------

**EC REPRESENTATIVE**

Cook Medical Europe Ltd  
O'Halloran Road  
National Technology Park  
Limerick, Ireland

2019-07  
C\_T\_FCC\_REV5