EN
2

Gastric Sizing Balloon Catheter (GSB)

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C _ T _ G S B _ R E V 1

ENGLISH

GASTRIC SIZING BALLOON CATHETER (GSB)

CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The GSB is a flexible gastric tube designed to be used in gastric and bariatric surgical procedures. The catheter provides visible and tactile delineation of the cardia of the stomach along with the ability to decompress the stomach, drain and remove gastric fluid and size a gastric pouch. The silicone catheter is 18 French and 55 cm long with centimeter markings every centimeter from 10 cm to 50 cm.

INTENDED USE

The GSB is indicated for use in gastric and bariatric surgical procedures to size the gastric pouch and drain and remove gastric fluid. The product is intended for use by physicians trained and experienced in gastric and bariatric surgical techniques.

CONTRAINDICATIONS

- Esophageal stricture which does not allow passage of the GSB through the mouth and esophagus.
- Previous history of inability to pass an orogastric tube.

PRECAUTIONS

Inspect the GSB prior to use for damage. If damaged, do not use.

POTENTIAL ADVERSE EVENTS

Complications which may result from the use of this product include the risks associated with the medications and methods utilized in the surgical procedure, the risks associated with any surgical procedure and the patient's degree of intolerance to a gastric tube.

INSTRUCTIONS FOR USE

1. The GSB tube is inserted into the esophageal passage and advanced into the stomach.
2. The balloon attached to the catheter is inflated with 10-20 cc of air or saline and drawn up to the cardia.
3. The bulge created by the balloon can be seen and a visible boundary is established for gastric dissection, band placement or staple resection.
4. Before gastric dissection, band placement or staple resection, the balloon is deflated and the catheter is removed.
5. The surgical procedure is then completed using standard techniques.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.