COOK

MEDICAL

CE
0088

EN
3

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C _ T _ I R D H C _ R E V 4

Cook TPN® Multilumen Catheter Repair Sets

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Cook TPN Double-Lumen Catheter Repair Set includes a double-lumen catheter assembly, one large lumen catheter end and one small lumen catheter end. **NOTE:** The repair set for the 9.0 and 12.0 French contain an A and B version. Refer to Choosing the Correct Repair Set to determine which set to use.

The Cook TPN Triple-Lumen Catheter Repair Set includes a triple-lumen catheter assembly, one large lumen and two small lumen catheter ends. **NOTE:** The repair set for the 8.0 French contains an A and B version. Refer to Choosing the Correct Repair Set to determine which set to use.

All sets also contain adhesive, Luer needle stub, syringe and an appropriate number of protective caps, Luer lock plugs, leakproof adapters and silicone sleeves. The adhesive is Silastic® Medical Adhesive Silicone Type A made by Dow Corning Corporation. For further details, refer to instructions for use supplied with the adhesive.

INTENDED USE

The Cook TPN Multilumen Catheter Repair Sets are intended to repair Cook TPN double- and triple-lumen catheters and Redo TPN double- and triple-lumen catheters.

The products are intended for use by physicians trained and experienced in the use of central venous catheters. Standard techniques for catheter handling and repair should be employed.

CONTRAINDICATIONS

None known

WARNINGS

Possible allergic reactions should be considered.

PRECAUTIONS

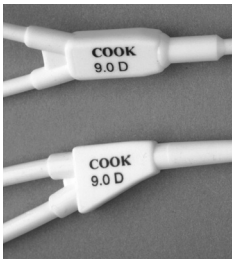
Care should be taken when using a wire guide through a Cook TPN catheter repair set to prevent damage within the catheter.

PRODUCT RECOMMENDATIONS

Choosing the Correct Repair Set

Look carefully at the shape of the manifold on the catheter you are repairing. This shape determines which repair set (A or B) you should use.

9.0 and 12.0 French Double Lumen


Example A

Example B

If the manifold has the more rectangular shape of example A, then use repair set A. If the manifold on a double-lumen has an angled edge like example B, then use repair set B.

ENGLISH

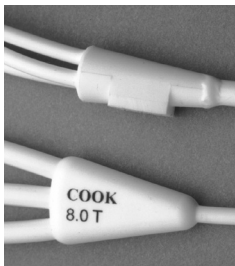8.0 French Triple Lumen

Example A

Example B

If the manifold has the more cone-like shape of example A, then use repair set A. If the manifold on a triple-lumen has a triangular shape like example B, then use repair set B.

INSTRUCTIONS FOR USE

Repairing Breaks Distal to the Y-Manifold

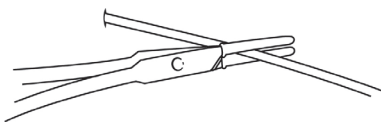
Fig. 1

1. Prepare the area. Firmly clamp the catheter near its entrance to the skin with rubber-shod forceps. (Fig. 1)

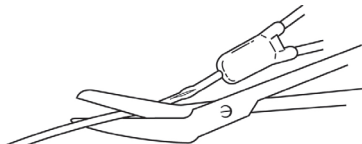
Fig. 2

2. Cut off the damaged portion of the catheter. (Fig. 2)

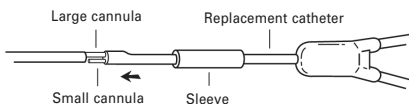
Fig. 3

3. The replacement catheter end has preloaded small (short end) and large (long end) metal cannulae. Slide the large metal cannula slightly into the large lumen of the catheter. (Fig. 3)

NOTE: 5.0 and 10.0 French catheters have equal sized cannulae.

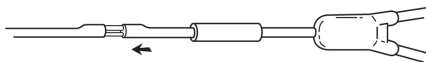
Fig. 4

4. Insert the small metal cannula(s) into the small lumen(s) of the catheter end. (Fig. 4)

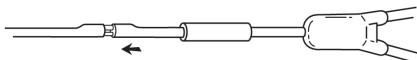
Fig. 5

5. Slide the small and large metal cannulae into the catheter lumens until the replacement catheter end is about 3 mm from the catheter end. (Fig. 5)

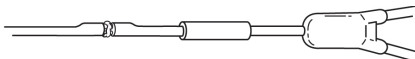
Fig. 6

6. Apply adhesive to junction. (Fig. 6)

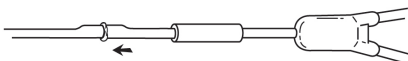
Fig. 7

7. Slide the metal cannulae of the replacement catheter end completely into the catheter. (Fig. 7)

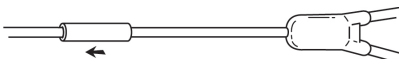
Fig. 8

8. Slide the sleeve over the junction. (Fig. 8)

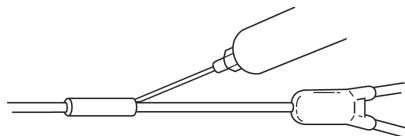
Fig. 9

9. Inject adhesive into both ends of the sleeve. (Fig. 9) Wipe off excess adhesive.

NOTE: Follow nursing protocol and adhesive manufacturer's recommendations (found on the adhesive package) to determine when the catheter is ready for use.

Repairing Breaks Proximal to the Y-Manifold

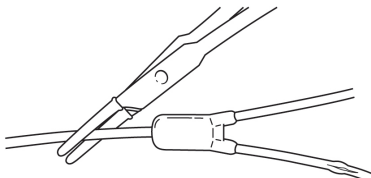
Fig. 10

1. Prepare the area. Firmly clamp the catheter distal to the Y-manifold with rubber-shod forceps. (Fig. 10)

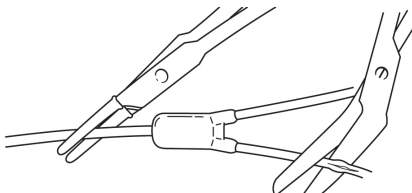
Fig. 11

2. Cut off the damaged portion of the extension tubing, leaving enough of the tubing so that it can be repaired. (Fig. 11)

ENGLISH

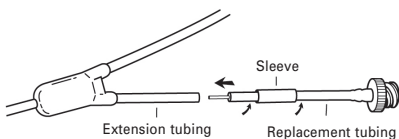
Fig. 12

3. The replacement tubing end has a preloaded metal cannula. With a twisting motion, slide the metal cannula into the extension tubing until the replacement tubing is approximately 2-3 mm from the extension tubing. (Fig. 12)

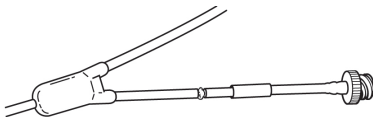
Fig. 13

4. Apply adhesive to the junction. (Fig. 13)

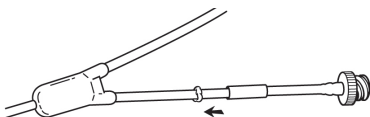
Fig. 14

5. Slide the metal cannula of the replacement tubing end completely into the extension tubing. (Fig. 14)

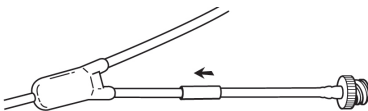
Fig. 15

6. Slide the sleeve over the junction. (Fig. 15)

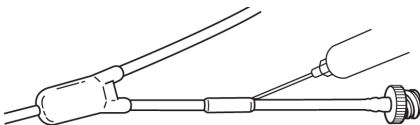
Fig. 16

7. Inject adhesive into both ends of the sleeve. (Fig. 16) Wipe off excess adhesive.

NOTE: Follow nursing protocol and adhesive manufacturer's recommendations (found on the adhesive package) to determine when the catheter is ready for use.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.