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ZH
22

C - T - I R H C - R E V . 1

Cook TPN® Single-Lumen Catheter Repair Sets

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Cook TPN Single-Lumen Catheter Repair Sets include one single-lumen catheter end. All sets consist of adhesive, Luer needle stub, syringe and protective cap, Luer lock plug, leak-proof adapter and silicone sleeve. The adhesive is Silastic® Medical Adhesive Silicone Type A, made by Dow Corning Corporation. For further details, refer to instructions for use supplied with the adhesive. The catheter is intended for use by physicians trained and experienced in the placement of central venous catheters.

INTENDED USE

The product is intended to repair Cook TPN and Redo TPN single-lumen catheters. The product is intended for use by physicians trained and experienced in the use of central venous catheters. Standard techniques for catheter handling and repair should be employed.

CONTRAINDICATIONS

None known.

WARNINGS

- Possible allergic reactions should be considered.

PRECAUTIONS

- Care should be taken when using a wire guide through a Cook TPN catheter repair set to prevent damage within the catheter.

INSTRUCTIONS FOR USE

1. Prepare the area. Firmly clamp the catheter near its entrance to the skin with rubber-shod forceps.
2. Cut off the damaged portion of the catheter.

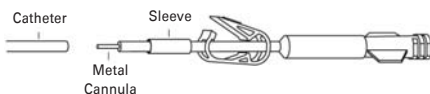
Fig. 1

3. The replacement catheter end has a preloaded metal cannula. (Fig. 1) Apply adhesive to the metal cannula.

Fig. 2

NOTE FOR 3.0 FRENCH SINGLE-LUMEN REPAIR SET

ONLY: To facilitate passage of the metal cannula of the replacement catheter into the existing catheter lumen, the included stylet should be passed through the replacement catheter and into the lumen of the existing catheter. The stylet acts as a guide for insertion of the metal cannula. After insertion of the metal cannula into the lumen of the existing catheter, the stylet should be removed. (Fig. 2)

Fig. 3

4. Insert the metal cannula into the existing catheter lumen with a twisting motion. (Fig. 3)

Fig. 4

5. Slide the existing catheter over the metal cannula until it is flush with the replacement catheter tubing. (Fig. 4)

Fig. 5

6. Apply adhesive to the junction. (Fig. 5)

Fig. 6

7. Slide the sleeve over the junction. (Fig. 6)

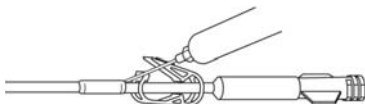
Fig. 7

8. Inject adhesive into both ends of the sleeve (Fig. 7) and wipe off the excess adhesive.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.