EN
4

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
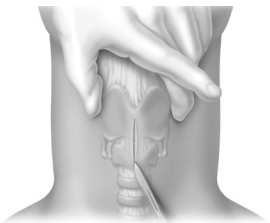
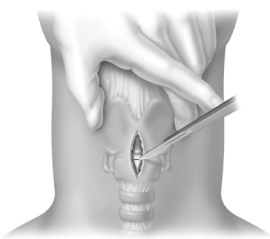
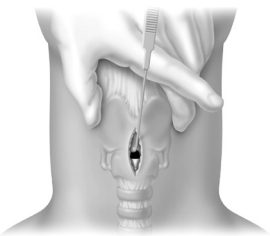
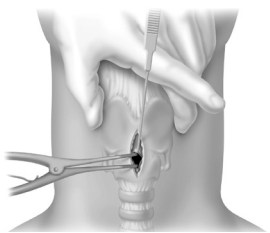
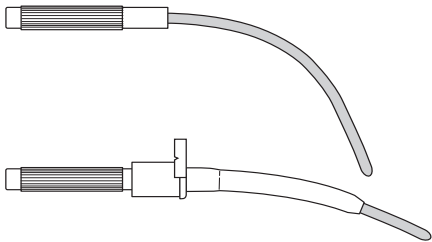
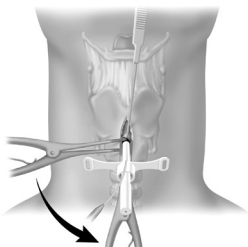
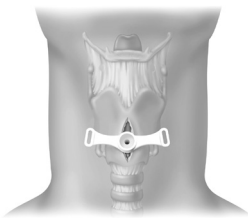
7

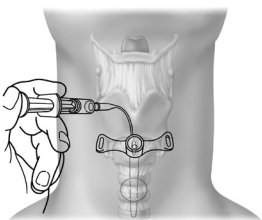8

MELKER CUFFED EMERGENCY CRICOTHYROTOMY CATHETER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Melker Cuffed Emergency Cricothyrotomy Catheter Set consists of an airway catheter, scalpel, syringe, curved dilator, Trousseau dilator, tracheostomy tube tape and tracheal hook.

INTENDED USE

The Melker Cuffed Emergency Cricothyrotomy Catheter Set is intended to establish emergency airway access when endotracheal intubation cannot be performed. Airway access is achieved utilizing the surgical technique via the cricothyroid membrane.

CONTRAINDICATIONS

None known.

WARNINGS

The Melker Cuffed Emergency Cricothyrotomy Catheter Set is not specifically designed for pediatric applications. Use with pediatric patients should be determined by the attending physician. Consideration should be given to the following medical and anatomic conditions and/or therapies:

- Distorted anatomy
- Subcutaneous abscess
- Hematoma
- Scarring/radiation
- Coagulopathies or systemic thrombolytic therapy

PRECAUTIONS

- This product is intended for use by physicians trained and experienced in proper emergency airway techniques. Standard emergency techniques for a surgical cricothyrotomy should be employed.
- Patients in need of cricothyrotomy may have significant spinal injury. In patients who have sustained significant trauma, the cervical spine should be immobilized throughout the procedure, if possible.
- Whenever possible and appropriate, utilize aseptic technique and local anesthetic for the procedure.

INSTRUCTIONS FOR USE

Surgical Technique

1. Identify the cricothyroid membrane between the cricoid and thyroid cartilages.
2. Firmly immobilize the larynx and make a vertical, midline skin incision down to the depth of the laryngeal structures. (**Fig. 1**)
3. Make a horizontal membrane incision near the inferior edge of the cricothyroid membrane. (**Fig. 2**) **NOTE:** The index finger may be moved aside or may remain in the wound, palpating the inferior edge of the thyroid cartilage, to "guide" the scalpel to the membrane. **NOTE:** A low cricothyroid incision avoids the superior cricothyroid vessels, which run transversely near the top of the membrane.
4. Insert the tracheal hook, oriented transversely. After insertion, apply cephalad traction to the inferior margin of the thyroid cartilage. (**Fig. 3**)
5. Insert the Trousseau dilator a short distance into the incision and enlarge the opening vertically. (**Fig. 4**)
6. Advance the handled dilator with the blunt tip, blunt end first, into the connector end of the airway catheter until the handle stops against the connector. (**Fig. 5**)

NOTE: This step may be performed prior to beginning the procedure. Use of lubrication on the surface of the dilator may enhance fit and placement of the emergency airway catheter.

7. Insert the cuffed emergency airway access assembly between the blades of the Trousseau dilator, into the airway. As the assembly is passed between the blades, rotate the Trousseau dilator 90 degrees counterclockwise, orienting the blades longitudinally in the airway to facilitate passage of the airway assembly. (**Fig. 6**)
8. Fully insert the cuffed emergency airway access assembly.
9. Remove dilator and instruments. (**Fig. 7**)
10. Inflate the cuff using a syringe; an 8-10 mL volume in the cuff will yield a cuff diameter of 22-29 mm. (**Fig. 8**) The inflation and deflation procedure is at the discretion of the physician.

WARNING: Inflation of the cuff with more than 20 mL is not recommended.

11. Fix the emergency airway catheter in place with the cloth tracheostomy tape strip in a standard fashion.
12. Connect the emergency airway catheter, using its standard 15 mm connector, to an appropriate ventilatory device.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd