- EN**
2 **Fuhrman Pleural Drainage Set**
Instructions for Use
- DA**
3 **Fuhrman pleuradrænagesæt**
Brugsanvisning
- DE**
4 **Fuhrman Pleuradrainage-Set**
Gebrauchsanweisung
- EL**
5 **ΣΕΤ παροχέτευσης υπεζωκοτικής κοιλότητας Fuhrman**
Οδηγίες χρήσης
- ES**
7 **Equipo de drenaje pleural Fuhrman**
Instrucciones de uso
- FR**
8 **Set de drainage pleural Fuhrman**
Mode d'emploi
- IT**
9 **Set per drenaggio pleurico di Fuhrman**
Istruzioni per l'uso
- NL**
10 **Fuhrman-pleuradrainageset**
Gebruiksaanwijzing
- NO**
12 **Fuhrman pleuralt drenasjesett**
Bruksanvisning
- PT**
13 **Conjunto de drenagem pleural Fuhrman**
Instruções de utilização
- SV**
14 **Fuhrman-set för pleuradränage**
Bruksanvisning

FUHRMAN PLEURAL DRAINAGE SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The intended purpose of the Fuhrman Pleural Drainage Set is to secure percutaneous access and provide a conduit for the removal of air or fluid by gravity or by suction. The Fuhrman Pleural Drainage Set (PPD-) consists of pigtail catheter with a Luer fitting on the proximal hub for connectivity, access needle, dilator, three-way stopcock, multipurpose tubing adapter, and wire guide. Refer to label for further information. The standard Luer connection may be connected to a suction source following hospital protocols.

INTENDED USE/INDICATIONS FOR USE

The Fuhrman Pleural Drainage Set (PPD-) is intended for drainage of air or fluid from the pleural space. The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard Seldinger technique for placement of needles, wire guides, dilators and catheters should be employed.

CONTRAINDICATIONS

None known

WARNINGS

- Over-insertion of the access needle, wire guide, dilator, or chest tube may result in serious or life-threatening injury. Clinical judgment and consideration of patient size and anatomy should guide appropriate chest tube selection (e.g., French size and length) and insertion.
- The wire guide should always advance without impedance to avoid perforation of any surrounding structures.
- The wire guide should always extend beyond the catheter tip to avoid perforation of any surrounding structures.

PRECAUTIONS

- Possible allergic reactions should be considered.
- A TFE-coated wire guide is recommended for a smoother catheter introduction.

POTENTIAL ADVERSE EVENTS

Adverse events associated with the use of the Fuhrman Pleural Drainage Set may include, but are not limited to:

- Hemorrhage/hemothorax
- Infection
- Organ laceration/perforation
- Re-expansion pulmonary edema
- Transient hypoxemia
- Subcutaneous emphysema

INSTRUCTIONS FOR USE

1. Prepare the chest using standard aseptic technique and infiltrate the insertion area with local anesthesia.
2. If needed make a small incision with a scalpel.
3. Direct the tip of the needle towards the desired catheter placement and insert the needle under the skin.
4. Gently advance the needle into the positive pressure area.
WARNING: Over-insertion of the needle, wire guide, dilators, and/or chest tubes may result in serious harm to the patient.
NOTE: Due to anatomical variations in chest wall thickness, insertion depth of access needle, wire guide, dilator, and chest tube will vary from patient to patient.
5. Attach a syringe to the needle. Verify placement into the pleural space by aspiration of air or fluid into syringe. Remove syringe from the needle once placement is verified.
6. Introduce the wire guide through the needle and gently advance it into the selected cavity.
NOTE: The wire guide should advance without impedance.
7. Remove the needle, leaving the wire guide in place, then advance the supplied dilator over the wire guide and dilate to facilitate catheter introduction.
8. Remove the dilator, leaving the wire guide in place, then introduce the catheter over the wire guide.
NOTE: The wire guide should always extend beyond the catheter tip.
9. Advance the catheter into position. Remove the wire guide and attach the multipurpose adapter to the Luer fitting on the catheter hub.
NOTE: If the use of three-way stopcock is elected, the three-way stopcock may be attached directly to the Luer fitting on the catheter hub and the multipurpose adapter may then be attached to the three-way stopcock.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Keep dry and out of sunlight. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2022
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd