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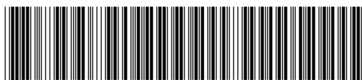
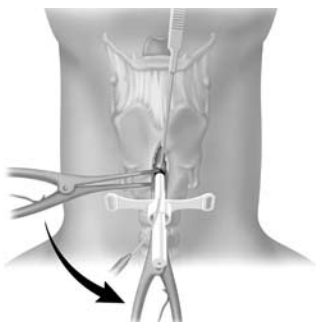
Fig. 12

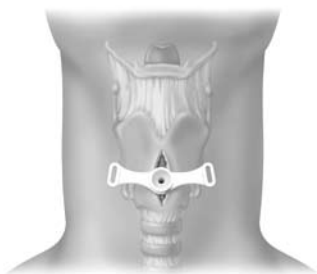
Fig. 13

MELKER UNIVERSAL CRICOTHYROTOMY CATHETER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Melker Universal Cricothyrotomy Catheter Set consists of components and accessories used for both Seldinger and surgical placement of a cricothyrotomy tube.

INTENDED USE

The Melker Universal Cricothyrotomy Catheter Set is intended to establish emergency airway access when endotracheal intubation cannot be performed. Airway access is achieved utilizing either percutaneous entry (Seldinger) technique or surgical technique via the cricothyroid membrane.

CONTRAINDICATIONS

No absolute contraindications known.

WARNINGS

- The Melker Universal Cricothyrotomy Catheter Set is not specifically designed for pediatric applications. Use with pediatric patients should be determined by attending physician.
- Consideration should be given to the following medical and anatomic conditions:
 - Distorted anatomy
 - Subcutaneous abscess
 - Hematoma
 - Post-operative scarring/radiation
 - Coagulopathies or systemic thrombolytic therapy

PRECAUTIONS

- This product is intended for use by physicians trained and experienced in proper emergency airway techniques. Standard emergency techniques for the placement of a Seldinger or surgical cricothyrotomy should be employed.
- Patients in need of cricothyrotomy may have significant spinal injury. In patients who have sustained significant trauma, the cervical spine should be immobilized throughout the procedure, if possible.
- Whenever possible and appropriate, utilize aseptic technique and local anesthetic for the procedure.

INSTRUCTIONS FOR USE

Illustrations

Seldinger Technique

1. Identify the cricothyroid membrane between the cricoid and thyroid cartilages.
2. Carefully palpate the cricothyroid membrane and, while stabilizing the cartilage, make a vertical incision in the midline. **(Fig. 1) NOTE:** Ensure that the incision is sufficiently large to allow passage of the dilator and airway catheter.
3. Attach the supplied syringe to either the introducer needle or the catheter introducer needle and advance the needle through the incision into the airway at a 45 degree angle to the frontal plane in a caudad direction, in the midline. **(Fig. 2) NOTE:** Entrance into the airway can be confirmed by aspiration on the syringe, resulting in free air return.
4. If using the catheter introducer needle, remove the syringe and needle, leaving the catheter in place. If using the introducer needle, remove only the syringe, leaving the needle in place.
5. Advance the soft, flexible end of the wire guide through the catheter or needle and into the airway several centimeters. **(Fig. 3)**
6. Remove the catheter or needle, leaving the wire guide in place. **(Fig. 4)**
7. Advance the handled dilator with the tapered tip, tapered end first, into the connector end of the airway catheter until the handle stops against the connector. **(Fig. 5)**
NOTE: This step may be performed prior to beginning the procedure. Use of lubrication on the surface of the dilator may enhance fit and placement of the airway catheter.
8. Advance the airway catheter/dilator assembly over the wire guide until the proximal stiff end of the wire guide is completely through and visible at the handle end of the dilator. **(Fig. 6) It is important to continually visualize the proximal end of the wire guide during the airway insertion procedure to prevent its inadvertent loss into the trachea.**
9. Maintaining wire guide position, continue to advance the airway catheter/dilator assembly over the wire guide with a reciprocating motion completely into the trachea.
Take care not to advance the tip of the dilator beyond the tip of the wire guide within the trachea.
10. Remove the wire guide and dilator simultaneously.
11. If placing a standard (uncuffed) airway catheter, proceed to step #12. If placing a cuffed airway catheter, inflate the cuff with air using a syringe. **(Fig. 7) NOTE:** An 8-10 ml volume in the cuff will yield a cuff diameter of 22-29 mm. The inflation and deflation procedure is at the discretion of the physician.

WARNING: Inflation of the cuff with more than 20 ml is not recommended.

12. Fix the airway catheter in place with tracheostomy tape strip in standard fashion.
13. Connect the airway catheter, using its standard 15 mm connector, to an appropriate ventilatory device.

Surgical Technique

1. Identify the cricothyroid membrane between the cricoid and thyroid cartilages.
2. Firmly immobilize the larynx and make a vertical, midline skin incision down to the depth of the laryngeal structures. (Fig. 1)
3. Make a horizontal membrane incision near the inferior edge of the cricothyroid membrane. (Fig. 8) The index finger may be moved aside or may remain in the wound, palpating the inferior edge of the thyroid cartilage, to "guide" the scalpel to the membrane. A low cricothyroid incision avoids the superior cricothyroid vessels, which run transversely near the top of the membrane.
4. Insert the tracheal hook, oriented transversely. After insertion, apply cephalad traction to the inferior margin of the thyroid cartilage. (Fig. 9)
5. Insert the Trousseau dilator a short distance into the incision and enlarge the opening vertically. (Fig. 10)
6. Advance the handled dilator with the blunt tip, blunt end first, into the connector end of the airway catheter until the handle stops against the connector. (Fig. 11)

CAUTION: Use only blunt-tip dilator with the surgical procedure.

NOTE: This step may be performed prior to beginning the procedure. Use of lubrication on the surface of the dilator may enhance fit and placement of the emergency airway catheter.

7. Insert the cuffed emergency airway access assembly between the blades of the Trousseau dilator, into the airway. As the assembly is passed between the blades, rotate the Trousseau dilator 90 degrees counterclockwise, orienting the blades longitudinally in the airway to facilitate passage of the airway assembly. (Fig. 12)
 8. Fully insert the cuffed emergency airway access assembly. Remove dilator and instruments. (Fig. 13)
 9. Inflate the cuff using a syringe; 8-10 ml volume in the cuff will yield a cuff diameter of 22-29 mm. (Fig. 7) The inflation and deflation procedure is at the discretion of the physician.
- WARNING: Inflation of the cuff with more than 20 ml is not recommended.**
10. Fix the emergency airway catheter in place with the cloth tracheostomy tape strip in a standard fashion.
 11. Connect the emergency airway catheter, using its standard 15 mm connector to an appropriate ventilatory device.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.