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ZH
14

C - T - W M N - R E V - 2

WEINMANN TRACHEOSTOMY EXCHANGE SET

CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Weinmann Tracheostomy Exchange Set consists of a Ciaglia Blue Rhino® Tracheostomy Dilator, Cook Airway Exchange Catheter, Rapi-Fit adapter, and two loading dilators (26 and 28 French).

INTENDED USE

Used to exchange adult tracheostomy tubes. Supplied sterile in peel-open packages. Intended for one-time use.

CONTRAINDICATIONS

Pediatric applications

WARNINGS

- Tracheostomy tube position should be verified by a chest x-ray.
- Care must be taken not to advance the Ciaglia Blue Rhino dilator past the black level mark.
- In small or short patients (50 kilograms or less and/or individuals 4 feet or less in height) care must be taken not to injure the airway at the level of the carina.

PRECAUTIONS

- Tracheostomy tube should fit snugly to the loading dilator for insertion.
- Activate hydrophilic coating by wetting surface of Ciaglia Blue Rhino dilator with sterile water or saline. For best results, maintain wetted condition of device during placement.
- Generous lubrication to the surface of the loading dilators will enhance fit and placement of the tracheostomy tube.
- Loading dilators are designed to be inserted within the tracheostomy tube only. Loading dilators should not be used for the creation of a stoma.

INSTRUCTIONS FOR USE

1. Insert the appropriate size loading dilator into inner lumen of the replacement tracheostomy tube.
2. Attach a Cook Airway Exchange Catheter via an adapter to an oxygen source.
3. Upon disconnection from mechanical ventilatory support, insert the Cook Airway Exchange Catheter down the lumen of the existing tracheostomy tube.
4. Depending on the patient's spontaneous ventilatory capacity, oxygenation and ventilation may be supplemented utilizing the manual resuscitator.
5. Deflate the existing tracheostomy tube balloon; temporarily detach the manual resuscitator; and withdraw the tracheostomy tube over the Cook Airway Exchange Catheter.
6. Position the replacement tracheostomy tube with loading dilator over the Cook Airway Exchange Catheter and reattach the manual resuscitator. Using the Cook Airway Exchange Catheter as a guide into the tracheal lumen, insert the replacement tracheostomy tube with loading dilator over the airway exchange catheter.
7. Inflate the cuff and secure the replacement tracheostomy tube.
8. Remove the Cook Airway Exchange Catheter and loading dilator. Re-institute mechanical ventilatory support.
9. Clinically and radiologically confirm position of tracheostomy tube.

Stomal Re-Dilation

Stomal re-dilation may be performed either routinely or if resistance is encountered at the time of airway-exchange.

1. Detach the manual resuscitator and Rapi-Fit adapter from the Cook Airway Exchange Catheter.
2. Perform re-dilation using the Ciaglia Blue Rhino dilator. For a 6 mm or 7 mm tracheostomy tube, dilate up to the 32 French black marking on the Ciaglia Blue Rhino dilator. For 7 mm or 8 mm tracheostomy tube exchange, use the 38 French black marking on the Ciaglia Blue Rhino dilator.
3. Once re-dilation of the stoma is complete, mount the new tracheostomy tube on the appropriate loading dilator and insert over the airway exchange catheter into the tracheal lumen.
4. Upon successful re-dilation and replacement of the tracheostomy tube, re-inflate the cuff and secure the tube. Withdraw the catheter and restore mechanical ventilatory support.
5. Clinically and radiologically confirm position. Suction airway to aspirate any blood due to re-dilation and tracheostomy tube re-insertion.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

Weinmann, Maxwell and Bander Joseph MD. *The Journal of Trauma: Injury, Infection and Critical Care*. "Introduction of a New Tracheostomy Exchange Device After Percutaneous Tracheostomy In a Patient with Coagulopathy". P317-319 vol 40 #2.

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Bloomington, IN 47404 U.S.A.