- EN 5 **Charger for Doppler Blood Flow Monitor**  
Instructions for Use
- CS 8 **Nabíječka pro dopplerovský monitor průtoku krve**  
Návod k použití
- DA 11 **Oplader til doppler blodflowmonitor**  
Brugsanvisning
- DE 14 **Ladegerät für Doppler-Blutfluss-Monitor**  
Gebrauchsanweisung
- EL 17 **Φορτιστής για συσκευή παρακολούθησης ροής αίματος Doppler**  
Οδηγίες χρήσης
- ES 20 **Cargador del monitor de flujo sanguíneo Doppler**  
Instrucciones de uso
- FI 23 **Doppler-verenvirtausmonitorin laturi**  
Käyttöohjeet
- FR 26 **Chargeur du moniteur de débit sanguin Doppler**  
Mode d'emploi
- IT 29 **Caricabatteria per monitor Doppler per flusso ematico**  
Istruzioni per l'uso
- NL 32 **Oplader voor doppler-bloedstroommonitor**  
Gebruiksaanwijzing
- NO 35 **Lader for dopplerblodstrømsmonitor**  
Bruksanvisning
- PL 38 **Ładowarka do dopplerowskiego monitora przepływu krwi**  
Instrukcja użycia
- PT 41 **Carregador para monitor de fluxo sanguíneo Doppler**  
Instruções de utilização
- SV 44 **Laddare för dopplerblodflödesmätare**  
Bruksanvisning
- TR 47 **Doppler Kan Akışı Monitörü Şarj Cihazı**  
Kullanma Talimatı

0 0 0 0 7 8 6 7 4 - 0 0 4

- 1 **EN** Charger • **CS** Nabíječka • **DA** Oplader • **DE** Ladegerät • **EL** Φορτιστής • **ES** Cargador • **FI** Laturi • **FR** Chargeur • **IT** Caricabatterie • **NL** Oplader • **NO** Lader • **PL** Ładowarka • **PT** Carregador • **SV** Laddare • **TR** Şarj Cihazı

- 2 **EN** Cable Manager • **CS** Organizér kabelů • **DA** Kabelholder • **DE** Kabelmanager • **EL** Διαχειριστής καλωδίων • **ES** Organizador de cable • **FI** Johtopidike • **FR** Enrouleur de câble • **IT** Avvolgicavo • **NL** Kabelhouder • **NO** Kabelordner • **PL** Menedżer kabli • **PT** Organizador de cabos • **SV** Kabelhanterare • **TR** Kablo Yöneticisi

- 3 **EN** Adapter Plugs • **CS** Zásuvkové adaptéry • **DA** Adapterstik • **DE** Adapterstecker • **EL** Βύσματα προσαρμογέα • **ES** Enchufes adaptadores • **FI** Sovitinpistokkeet • **FR** Fiches d'adaptation • **IT** Adattatori per spina • **NL** Adapterstekkers • **NO** Adapterplugger • **PL** Wtyczki adaptera • **PT** Fichas adaptadoras • **SV** Kontaktadapterar • **TR** Adaptör Fişleri

Figure 1a

Figure 1b


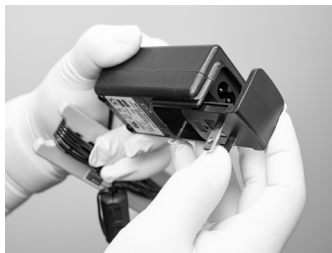
Figure 2

Figure 3

Figure 4

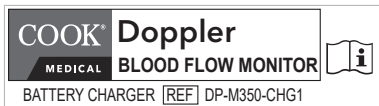**EN** Doppler Blood Flow Monitor Battery Charger • **CS** Nabíječka pro dopplerovský monitor průtoku krve • **DA** Batterioplader til doppler blodflowmonitor • **DE** Akkuladegerät für Doppler-Blutfluss-Monitor • **EL** Φορτιστής μπαταριών για συσκευή παρακολούθησης ροής αίματος Doppler • **ES** Cargador de baterías del monitor de flujo sanguíneo Doppler • **FI** Doppler-verenvirtausmonitorin akkulaturi • **FR** Chargeur de batterie du moniteur de débit sanguin Doppler • **IT** Caricabatterie per monitor Doppler per flusso ematico • **NL** Batterijoplader voor doppler-bloedstroommonitor • **NO** Batterilader for dopplerblodstrømsmonitor • **PL** Ładowarka baterii do dopplerowskiego monitora przepływu krwi • **PT** Carregador de bateria para monitor de fluxo sanguíneo Doppler • **SV** Batteriladdare för dopplerblodflödesmätare • **TR** Doppler Kan Akış Monitörü için Batarya Şarj Cihazı

**EN** Doppler Blood Flow Monitor • **CS** Dopplerovský monitor průtoku krve • **DA** Doppler blodflowmonitor • **DE** Doppler-Blutfluss-Monitor • **EL** Συσκευή παρακολούθησης ροής αίματος Doppler • **ES** Monitor de flujo sanguíneo Doppler • **FI** Doppler-verenvirtausmonitori • **FR** Moniteur de débit sanguin Doppler • **IT** Monitor Doppler per flusso ematico • **NL** Doppler-bloedstroommonitor • **NO** Dopplerblodstrømsmonitor • **PL** Dopplerowski monitor przepływu krwi • **PT** Monitor de fluxo sanguíneo Doppler • **SV** Dopplerblodflödesmätare • **TR** Doppler Kan Akış Monitörü

## Charger for Blood Doppler Blood Flow Monitor

**CAUTION: US federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).**

### DEVICE DESCRIPTION

The Charger for Doppler Blood Flow Monitor (DP-M350-CHG1) charges the internal battery pack of the monitor and is supplied with interchangeable plug adapters that are compatible with regional (US, UK, EU, AUS) mains input connections. The unit is double insulated (in insulation class II).

### Performance Characteristics

100-240 VAC, 50-60 Hz, 0.35 A max input

12.6 VDC @ 1.2 A output

### Device Compatibility

Doppler Blood Flow Monitor (DP-M350)

### Intended User

The device is intended for use by physicians who are licensed providers, clinically trained, and experienced in microsurgical vascular procedures and monitoring. In addition, monitoring and care of the device may be performed by licensed healthcare providers.

### Operating Principle

The Charger for Doppler Blood Flow Monitor charges the internal lithium-ion battery of the monitor with DC electricity by using circuitry tailored to lithium-ion cell chemistry.

### INTENDED USE

To charge the internal battery of the Doppler Blood Flow Monitor.

### CONTRAINDICATIONS

None.

### WARNINGS

1. Do not connect the charger to the mains before it is connected to the Doppler Blood Flow Monitor.
2. Remove from wall power source (mains) prior to unplugging the charger from the Doppler Blood Flow Monitor.
3. Do not attempt to sterilize by any means.
4. To isolate equipment from mains, remove the battery charger from the AC mains socket. Portable RF communications equipment should not be used within 30 cm (12 in) of any part of the system, including cables. Degradation of performance may result.
5. If any components of the charger are damaged, the monitor may be subject to signal interference.

### PRECAUTIONS

1. Ensure the charger is dry before plugging in.
2. This product is designed for indoor use only and should not come into contact with water or dust. To prevent overheating the product should not be covered during use.

3. The mains socket should be easily accessible. In the event of operational error, the plug should be immediately removed from the socket.
4. The product contains dangerous voltages, and the cover should not be removed. All service or maintenance work should be carried out by qualified personnel who can get assistance by contacting the manufacturer's agent.
5. Not for use with any other electrical equipment.

## HOW SUPPLIED

The charger is supplied non-sterile. It is a reusable device.

## STORAGE

Recommended storage and shipping temperature and humidity ranges: -40°C to 60°C and 30% to 90%.

## INSPECTION OF DEVICE

Inspect the charger thoroughly, including packaging, to verify that there is no damage prior to use.

## INSTRUCTIONS FOR USE

### Battery Charging Instructions

1. Select the correct regional plug adapter and confirm that the mains voltage is compatible with the charger.
2. If the appropriate plug adapter is not connected to the primary charger unit, remove the existing plug adapter by pressing on the grated area directly beneath its output plugs (Figure 1a), or by placing fingers directly below the elevated plug adapter (Figure 1b), and pushing upward and away from the cable. The attached plug adapter will detach and slide upward and off the primary charger unit.
3. Orient the appropriate plug adapter so that it is appropriately in-line with the receiving channel of the primary charger unit (Figure 2) and slide the plug adapter fully onto the primary charger unit (Figure 3).
4. Connect the supplied charger to the rear port of the Blood Flow Monitor. Do not connect the charger to the mains before it is connected to the monitor.
5. Connect the charger to the mains.
6. Charge the internal battery until fully charged as indicated by the light on the top of the charger illuminating green light. When charging is complete, disconnect from the mains before removing monitor connections, or the charger may be left attached and plugged in.
7. Battery indicator light on the front left of the monitor illuminates red when the monitor battery needs to be charged. If the battery light is not illuminated, the monitor is adequately charged.

### Charge Indicator (on charger):

Red – Indicates battery pack is being charged

Green – Indicates battery pack is fully charged

### Cable / Cord Manager

The Cable / Cord Manager (as supplied) can be used to organize the charger's cable when the unit is not in use (Figure 4).

## CLEANING AND DISINFECTION

**WARNING:** After cleaning and disinfecting, ensure the device is completely dry before reattaching connectors and before connecting it to a power source. To avoid electrical shock, always unplug the charger from the wall outlet before cleaning the device.

## **Cleaning the Exterior**

Clean the exterior surfaces of the Charger between patients: Use 1 teaspoon (5 mL) of pH neutral detergent per 1 gallon (4 liters) of potable water.

1. Disconnect the charger from the power source and the Monitor.
2. Use a lint-free cloth dampened with a pH neutral detergent solution to clean the exterior surfaces. Remove all visible soil. Use a soft bristle brush on hard to clean areas. When using a soft bristle brush, avoid excessive force.
3. Use a lint-free cloth dampened with clear water to remove all detergent residue.
4. Use a lint-free cloth to dry the exterior surface.
5. Inspect the device for any visible soil or detergent residue.
6. Repeat all cleaning steps until the surfaces are visibly clean.
7. Inspect the device for damage after cleaning. If any parts are damaged, return the device for repair or replace the damaged part.

## **Low Level Disinfection of the Exterior**

Disinfect the Charger exterior surfaces between patients. Use one of the following methods.

**Prerequisite:** Before disinfecting the Charger exterior, ensure you have performed the cleaning as instructed in the previous section.

### **Isopropyl Alcohol or Commercially Available Wipes**

Use 70% isopropyl alcohol by volume or off the shelf 70% isopropyl alcohol with a lint-free cloth.

1. Use a lint-free cloth to wipe the alcohol onto the exterior, thoroughly wetting the surfaces.
2. Keep wet 5 minutes.
3. Allow to air dry.
4. Inspect the device for damage after disinfection. If any parts are damaged, return the device for repair or replace the damaged part.

## **DISPOSAL OF DEVICE**

To dispose of the charger, please return to Cook or to your Cook Medical products distributor

## **SERIOUS INCIDENT REPORTING**

If any serious incident has occurred in relation to the device this should be reported to Cook Medical and the competent authority and/or regulatory authority of the country where the device was used.

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**Cook Vandergrift Inc.**
**Cook Medical Europe LTD**
William A. Cook Australia Pty Ltd