EN 2	Flexipet Pipettes Instructions for Use
CS 3	Pipety Flexipet Návod k použití
DA 4	Flexipet-pipetter Brugsanvisning
DE 5	Flexipet Pipetten Gebrauchsanweisung
EL 6	Πιπέτες Flexipet Οδηγίες χρήσης
ES 7	Pipetas Flexipet Instrucciones de uso
FR 8	Pipettes Flexipet Mode d'emploi
IT 9	Pipette Flexipet Istruzioni per l'uso
NL 10	Flexipet pipetten Gebruiksaanwijzing
PT 11	Pipetas Flexipet Instruções de utilização
RU 12	Пипетки Flexipet Инструкция по применению
SV 13	Flexipet-pipetter Bruksanvisning

NOTE: One cell mouse embryo tested and passed with 80% or greater blastocyst within 96 hours. USP endotoxin (LAL) tested and passed with 20 EU's or less per device. Testing is conducted on a lot-to-lot (batch) basis.

CAUTION: Federal (U. S. A.) law restricts this device to sale by or on the order of a physician.

PRECAUTIONS:

Assisted reproduction microtools (and any other accessories used during these procedures having direct contact with gametes, zygotes, pre-embryos and/or embryos) should be comprised of embryo compatible materials.

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population.

CONTRAINDICATIONS: There are no known contraindications for the use of this device.

• **FLEXIPET® DENUING PIPETTES**

Flexipet® Denuding Pipettes are used to remove cumulus cell layers. Supplied sterile in peel-open packages. Intended for one-time use.

SUGGESTED INSTRUCTIONS FOR USING FLEXIPET® DENUING PIPETTES:

1. Attach Flexipet® to the Cook® Adjustable Handle (Order Number: K-MPH-1000).
2. Aspirate oocyte in and out of pipette until the cumulus cell layers are removed.
3. Gently withdraw pipette tip from dish.
4. Remove pipette from holder and discard.

• **FLEXIPET® MANIPULATION PIPETTES**

Flexipet® Manipulation Pipettes are used to manipulate the cumulus oocyte complex. Supplied sterile in peel-open packages. Intended for one-time use.

SUGGESTED INSTRUCTIONS FOR USING FLEXIPET® MANIPULATION PIPETTES:

1. Attach Flexipet® to the Cook® Adjustable Handle (Order Number: K-MPH-1000).
2. Aspirate oocyte into pipette for manipulation, transfer, etc.
3. Expel oocyte and gently withdraw pipette tip from dish.
4. Remove pipette from holder and discard.

HOW SUPPLIED

Supplied sterilized by gamma irradiation in peel-open pouches. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

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Vandergrift, PA 15690 U.S.A.
Bloomington, IN 47402-4195 U.S.A.
COOK MEDICAL EUROPE LTD