



EN

AcuTrap™ 4-Chamber Polyp Trap
Instructions for Use



190802 Rev. B





ENGLISH

INDICATIONS

The single use polyp trap is designed for in-line suction retrieval of polyps during endoscopic procedures.

CONTRAINDICATIONS



None known within the scope of the indicated use.

INSTRUCTIONS

1. Connect suction tubing to center port on trap and outside tubing on lid to endoscope.
2. Rotate arrow on lid to position tubing over chamber 1.
3. Retrieve and collect specimen.
4. Rotate lid clockwise to position tubing over chamber 2 after first specimen is collected.
5. Retrieve and collect specimen.
6. Repeat step 4 and 5 by positioning tubing over chamber 3 & 4.
7. To detach, disconnect suction tubing from suction system and trap tubing from endoscope.
8. After all specimens are transferred to a transport vial, discard remaining portion of trap immediately.

DISPOSAL

After use, this product may be a potential biohazard. Dispose of it in accordance with accepted medical practice and applicable local, state and federal laws and regulations.



A symbol glossary can be found at cookmedical.com/symbol-glossary



X mm

If symbol appears on product label, X = maximum insertion portion width



X mm

If symbol appears on product label, X = minimum accessory channel



Consult Instruction for Use



Single use – Do not reuse



Keep dry



Non Sterile



Catalog Number / Product Code



Lot Number



Use by Date

RX ONLY

Federal (USA) Law restricts this device to sale by or on the order of a physician



Manufacturer



Quantity of devices in pouch



Quantity of devices in box



Do not use if package is opened or damaged



Distributed By:
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

cookmedical.com



Manufactured By:
Endo-Therapeutics, Inc.
15251 Roosevelt Blvd. Suite 204
Clearwater, FL 33760 USA

©2025 Endo-Therapeutics, Inc. All rights reserved. 190802 Rev. B Sept 2025