



EN

## **RotoNet™ Rotatable Retrieval Net**



## ENGLISH

### INDICATIONS

RotoNet™ is used in the endoscopic retrieval of foreign body, food bolus and excised tissue such as polyps.

### CONTRAINDICATIONS

None known within the scope of the indicated use.

### WARNING AND PRECAUTIONS

Patient clinical conditions and type of foreign objects may result in using various retrieval techniques. The technique used is at the discretion of the physician.

Do not kink tubing when loading into endoscope.

Avoid blindly passing the device past any foreign object.

Avoid inadvertently grasping tissues or organs not intended for retrieval.

Do not use to retrieve sharp objects.

Do not deploy handle quickly, in an acute position, when the endoscope is in an extremely articulated or coiled position.

Maintain moderate pressure when passing the trachea to avoid losing the foreign object.

Use caution when removing the foreign object after retrieval. Doing so may result in damage to the net.

### INSTRUCTIONS

1. Verify the endoscope accessory channel size is compatible with the device.
2. Peel open the pouch and remove the device. Visually inspect the device for kinks, loose or broken parts.
3. Form the device into a "U" shape and deploy and retract to verify proper actuation.
4. Retract the device into the outer sheath before inserting the device into the working channel of the endoscope. If resistance is met when the bending tip of the endoscope is angulated, release the bending angle slightly until the instrument passes smoothly. Do not force the device into an angulated endoscope as this could damage the endoscope.
5. Deploy, position and retract the net to retrieve the foreign object.
6. After the foreign object is collected, retract the device and endoscope while maintaining moderate pressure when passing the trachea to avoid losing the foreign object.

### DISPOSAL

After use, this product may be a potential biohazard. Dispose of it in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

### STORAGE

Store sterile instruments at room temperature. Do not expose to solvents, radiation, UV light or any other substance or environment that could compromise the sterility of the unit.

A symbol glossary can be found at [cookmedical.com/symbol-glossary](http://cookmedical.com/symbol-glossary)



X mm

If symbol appears on product label, X = maximum insertion portion width



X mm

If symbol appears on product label, X = minimum accessory channel



Attention, see IFU



Consult Instruction for Use



Single use - Do not reuse



Do not resterilize



Keep dry



Ethylene Oxide Sterilization



Catalog Number / Product Code



Lot Number



Use by Date



Date of manufacture

**RX ONLY**

**Federal (USA) Law restricts this device to sale by or on the order of a physician**



Manufacturer



Quantity of devices in pouch



Quantity of devices in box



**Do not use if package is opened or damaged**



Distributed By:  
**Cook Medical, LLC**  
400 N. Daniels Way  
Bloomington, IN 47404 USA

[cookmedical.com](http://cookmedical.com)



Manufactured By:  
**Endo-Therapeutics, Inc.**  
15251 Roosevelt Blvd. Suite 204  
Clearwater, FL 33760 USA

©2019 Endo-Therapeutics, Inc. All rights reserved. 190803 Rev. A  
Patent Pending