

COOK®

MEDICAL

EN

Sentrus™ Auxiliary Water Jet Connector

Instructions for Use

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

This unit is intended to provide irrigation via sterile water supply during GI endoscopic procedures when used in conjunction with an irrigation pump.

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

MATERIAL

This device does not contain latex, phthalates, or animal derivatives.

WARNINGS

Please read all instructions before using this device.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify Cook Medical for return authorization.

These devices are intended to be used with Olympus, and Fujifilm Endoscopes, and follow the Olympus or Fujifilm guidelines for air/water/suction use in the endoscopes.

Never attach or reattach the Auxiliary Water Jet Connector to an endoscope that is/was inside the patient and has not been reprocessed.

Exposure to bodily fluids may occur during the connection or disconnection of this device. The User is responsible for adhering to Body Substance Isolation protocols.

Do not insert this device into the patient. The device should be attached to the auxiliary water port of a flexible gastrointestinal endoscope and an irrigation source. Use proper techniques to avoid risk of aspiration of a patient.

Always prime the auxiliary water channel or biopsy channel prior to inserting the endoscope into the patient.

USER QUALIFICATIONS

This device should only be used by or under the supervision of physicians who have been trained in endoscopic procedures.

PRECAUTIONS

Check the expiration date; do not use if expired.

Sterility is guaranteed unless the sterile pouch is damaged, or the seal is broken.

Do not store in direct sunlight, at extreme temperature, or in high humidity.

Operate this device gently and don't exert too much force.

PACKAGING AND STERILIZATION

This device is packaged and sterilized with ethylene oxide. It cannot be re-sterilized.

VALIDITY PERIOD

3 years

INSTRUCTIONS FOR USE

1. Remove the device from the tray.

2. Attach the Auxiliary Water Jet Connector to the endoscope by screwing it onto the auxiliary water port.

WARNINGS

Take care to ensure that the scope connector is aligned to the port and avoid excessive tightening. Misalignment or excessive tightening of the scope connector could lead to the possibility of leakage.

3. Once the Auxiliary Water Jet Connector is attached to the auxiliary water port of the endoscope, attach irrigation tubing to the Auxiliary Water Jet Connector. The Auxiliary Water Jet Connector is manufactured with a one-way valve to minimize the risk of cross-contamination of the irrigation system.

4. Place distal tip of endoscope into water filled receptacle, press and hold the Suction Valve for 2-3 seconds and then release to confirm the endoscope is suctioning water.

5. With distal tip of scope still under water, rest finger over hole in Air/Water Valve to confirm that air is exiting the tip of scope, which is evident by the production of air bubbles.

6. Remove distal tip of endoscope from water filled receptacle.

7. Press the Air/Water Valve and hold for 5-10 seconds to confirm that water exits the air/water nozzle on the end of the endoscope.

STORAGE AND TRANSPORTATION

Store at room temperature between 50°F and 104°F (10°C and 40°C), away from sunlight, non-corrosive gases, and in a well-ventilated area.

DEVICE DISPOSAL

After use, this device may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Country of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Single sterile barrier system		Sterilized using ethylene oxide
	Do not use product if package is damaged		Not made with natural rubber latex
	Do not re-sterilize		Consult instructions for use
	Number of devices in package		Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	DEHP free		Model Number
	Catalogue number		Distributor
	Unique device identifier		

COOK[®]

MEDICAL



Distributed By:
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co., Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803
Fax: 0514-87268752
Website: www.fartley.com
E-mail: sales@fartley.com