

EN



Sentrus™ Bite Block

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

The bite block is used to protect the endoscope insertion tube from being bitten by the patient. The bite block includes a retaining strap.

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

VALIDITY PERIOD

5 years

WARNINGS

Please read all instructions before using this device.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify Cook Medical for return authorization.

When placing the bite block in the patient's mouth, care should be exercised to avoid placing fingers between the bite block and the patient's teeth.

The bite block is intended for single patient use only. Any institution, practitioner, or third party who

reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses this disposable device must bear full responsibility for its safety and effectiveness.

USER QUALIFICATIONS

This device should only be used by or under the supervision of physicians who have been trained in endoscopic procedures.

PRECAUTIONS

Check the expiration date; do not use if expired.

Do not store in direct sunlight, at extreme temperature, or in high humidity.

Place and remove this device gently and don't exert too much force.

PACKAGING AND STERILIZATION

This device is packaged non-sterile.

INSTRUCTIONS FOR USE

1. Place the bite block in the patient's mouth. If the retaining strap is used, secure the strap on the bite block.
2. To secure retaining strap, insert the retaining strap through posterior aspect of side port and thread over cleat.

STORAGE AND TRANSPORTATION

Store at room temperature between -4°F and 140°F (-20°C and 60°C), away from sunlight, non-corrosive gases, and in a well-ventilated area.


DEVICE DISPOSAL


After use, this device may be a potential biohazard. Handle and

dispose of in accordance with
accepted medical practice and

applicable local, state and federal
laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

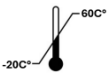	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Country of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Not made with natural rubber latex		Catalogue number
	Do not use product if package is damaged		Number of devices in package
	Non-sterile		Consult instructions for use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Unique identifier device
	Distributor		Model Number

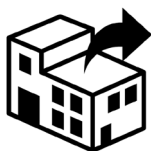

Distributed By:
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co.,
Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803 Fax:
0514-87268752
Website: www.fartley.com
E-mail: sales@fartley.com