



EN

Sentrus™ Dual-End Cleaning Brush

Instructions for Use

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

This device is intended for mechanical pre-cleaning of the endoscope channels, valves and valve shafts.

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

VALIDITY PERIOD

2 years

MATERIAL

This device does not contain latex, phthalates, or animal derivatives.

WARNINGS

Please read all instructions before using this device.

Follow society guidelines and original equipment manufacturer recommendations to properly pre-clean the endoscope after an endoscopy procedure.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify

Cook Medical for return authorization.

Original equipment manufacturer's guidelines regarding the care and cleaning of individual endoscope features must be reviewed prior to the use of any cleaning brush.

Dispose of the Dual-End Cleaning Brush after initial use.

Ensure the brush is intact before use, do not use if it is damaged or has loose bristles.

Do not apply excessive force with the bristles to avoid damaging the surface of the endoscope.

Take care to prevent the bristles from falling out or loosening during cleaning to avoid contaminating the cleaning equipment.

If the brush bristles fall out, deform, or suffer other damage during use, stop using the brush immediately and replace it with a new one.

Never transfer a used cleaning brush from one endoscope to another endoscope, since cross-contamination could occur and result in patient complications. Dual-End Cleaning Brushes are designed to safely remove debris, some of which can be retained in the bristle structure.

Do not use the Dual-End Cleaning Brush in a specific channel if use is

not recommended by the endoscope's Original Equipment Manufacturer.

Check the expiration date; do not use if expired.

USER QUALIFICATIONS

This device should only be used by or under the supervision of physicians who have been trained in endoscopic procedures.

PACKAGING AND STERILIZATION

This device is packaged non-sterile.

INSTRUCTIONS FOR USE

Cleaning endoscope channel:

1. Check the cleaning brush before use. Brush tube and head should have no folds, breakage, or damage.
2. Insert the Channel cleaning brush (small brush) into the endoscope channel and push it slowly to clean the channel until the cleaning brush is pushed out from the other end of the channel. Do not force too hard and rotate the cleaning brush at will when pushing to prevent damage to the channel.
3. Slowly pull the cleaning brush out of the channel.

Note: Suitable cleaning medium can be used during cleaning.

Cleaning endoscope valve and valve shafts:

1. Check the cleaning brush before use. Brush head should have no damage.
2. Grip the handle and insert the Channel-opening cleaning brush (large brush) into the endoscope valve; push and pull it slowly to clean the valve. Do not force too much and rotate the cleaning brush at will when pushing and pulling to prevent damage to the valve.

Note: Suitable cleaning medium can be used during cleaning.

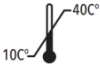
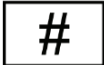
STORAGE AND TRANSPORTATION

Store at room temperature between 50 and 104 °F (10°C and 40°C), away from sunlight, non-corrosive gases, and in a well-ventilated area.

DEVICE DISPOSAL

After use, this device may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Country of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Not made with natural rubber latex		Catalogue number
	Do not use product if package is damaged		Number of devices in package
	Non-sterile		Consult instructions for use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Model Number
	Distributor		Unique identifier device



Distributed By:
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co.,
Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803 Fax:
0514-87268752
Website: www.fartley.com
E-mail: sales@fartley.com