

COOK®

MEDICAL

EN

Sentrus™ Cleaning Sponge

Instructions for Use

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

This product is used at the bedside for endoscope cleaning, then placed over the scope tip to protect the tip during transportation.

MATERIAL

This device does not contain latex, phthalate, or animal derivatives.

3E Cleaner contains: Amylase, alpha – Lipase (Triacylglycerol), Subtilisin. May produce an allergic reaction.

CONTRAINDICATIONS

None.

WARNINGS

Please read all instructions before using this device.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

Cleaning Sponge must be used exclusively by physicians and nurses, under the supervision of physicians, properly trained for pre-cleaning of endoscope with Cleaning Sponge.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify Cook Medical for return authorization.

3E cleaner is harmful if swallowed and irritating to eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Avoid contact with skin.

Wear suitable gloves and eye/face protection.

Check the expiration date; do not use if expired.

PACKAGING AND STERILIZATION

This device is packaged non-sterile.

INSTRUCTIONS FOR USE

Before using the Cleaning Sponge, have the following available:

- Personal Protective Equipment (PPE) - When handling cleaning agents and/or disinfectants, PPE is always required. PPE includes gloves, eye protection, impervious gown, face shield or simple surgical mask.



- A plastic container for detergent (if product does not include a plastic container).

1. Open package to remove sponge.
2. Put the sponge in the plastic container and fill with 500ml of water. Squeeze the sponge 5-7 times to prepare the 3E Cleaner Detergent.
3. Place sponge around the insertion tube of scope or tubular instrument.
4. Run sponge down length of scope or instrument to remove gross soil.
5. Rinse sponge and repeat until surface is clean.
6. Put sponge on the distal tip of endoscope during transportation.
7. Discard sponge and plastic container.

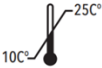
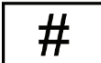
STORAGE AND TRANSPORTATION

Store at room temperature between 50°F and 77 °F (10°C and 25°C), humidity between 10% and 85%, away from sunlight, non-corrosive gases, and in a well-ventilated area.

DEVICE DISPOSAL

After use, this device may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Country of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Not made with natural rubber latex		Catalogue number
	Do not use product if package is damaged		Number of devices in package
	Non-sterile		Consult instructions for use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		DEHP free
	Model Number		Distributor
	Unique identifier		device



Distributed By:
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co., Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803 Fax:
0514-87268752

Website: www.fartley.com

E-mail: sales@fartley.com