

COOK®

MEDICAL

EN

Sentrus™ Medical Lubricating Jelly

Instructions for Use

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

This device is a sterile, water-soluble, latex-free gel intended for the smooth insertion of endoscope, reducing the risk of trauma during procedures.

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

MATERIAL

This device does not contain latex, phthalates, or animal derivatives.

WARNINGS

Please read all instructions before using this device.

Check the expiration date; do not use if expired.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

Follow society guidelines and original equipment manufacturer recommendations to properly prepare for an endoscopy procedure.

This device is limited to entry through the orifices of the human body.

This device is used with other endoscopic medical devices such as an endoscope.

Sterile Lubricant may affect sample detection accuracy.

Do not use if the lubricant has lumps.

Use one device at a time; more than one device per procedure may result in unclear risks and adverse side effects.

This device is only suitable for intact mucosa and cannot be used on damaged mucosa or injured skin.

If irritation or discomfort occurs, discontinue use and consult a physician. Keep out of eyes and ears. Keep out of the reach of children.

If contact with eyes occurs, flush eyes with plenty of 0.9% physiological saline solution. If no saline is available, flush with copious amounts of water.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify Cook Medical for return authorization.

USER QUALIFICATIONS

This device should only be used by or under the supervision of physicians who have been trained in endoscopic procedures.

PRECAUTIONS

Do not store in direct sunlight, at extreme temperature, or in high humidity.

PACKAGING AND STERILIZATION

This device is irradiation sterilized. It cannot be re-sterilized.

INSTRUCTIONS FOR USE

It is recommended to wash or disinfect your hands before and after use.

1. Visually inspect the pouch or tube to detect any abnormality. Do not use if pouch is damaged, or opened.
2. Tear sachet at one end and squeeze Lubricating Jelly on the Gauze as needed and then lubricate

the endoscope.


STORAGE AND TRANSPORTATION


Store at room temperature between 50 °F and 104 °F (15°C and 30°C), away from sunlight, non-corrosive gases, and in a well-ventilated area.

DEVICE DISPOSAL

After use, this device may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

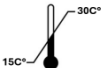	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Country of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Single sterile barrier system		Sterilized using irradiation
	Do not use product if package is damaged		Number of devices in package
	Do not re-sterilize		Consult instructions for use
	Not made with natural rubber latex		Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	DEHP free		Model Number
	Distributor		Catalogue number
	Unique device identifier		

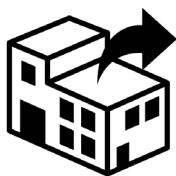

Distributed By :
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co.,
Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803 Fax:
0514-87268752
Website: www.fartley.com
E-mail: sales@fartley.com