

COOK®

MEDICAL

EN

Sentrus™ Suction Tubing

Instructions for Use

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

This device is intended to connect the endoscope with a suction pump (e.g. Ardo Senator 30 suction pump) to provide suction during endoscopic procedures. This device is designed for 24-hour, multi-patient use to minimize infection control risks associated with reuse.

MATERIAL

This device does not contain latex, phthalates, or animal derivatives.

This device contains polyvinyl chloride (PVC).

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

WARNINGS

Please read all instructions before using this device.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify Cook Medical for return authorization.

Follow society guidelines and original equipment manufacturer recommendations to properly prepare for an endoscopy procedure.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

PACKAGING AND STERILIZATION

This device is packaged non-sterile.

USER QUALIFICATIONS

This device should only be used by or under the supervision of physicians who have been trained in endoscopic procedures.

FREQUENCY OF USE

This device is rated for 24-hour, multi-patient use. Discard daily.

INSTRUCTIONS FOR USE

1. Open the package and remove the Sentrus Suction Tubing.
2. Connect the Sentrus Suction Tubing to the endoscope and to the disposable secretion bag or the reusable collection container provided for this purpose.

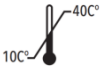
STORAGE AND TRANSPORTATION

Store at room temperature between 50 °F and 104 °F (10°C and 40°C), away from sunlight, non-corrosive gases, and in a well-ventilated area.

PRODUCT DISPOSAL

After use, this device may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Date of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Not made with natural rubber latex		Catalogue number
	Do not use product if package is damaged		DEHP free
	Non-sterile		Consult instructions for use
	Number of devices in package		Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Model Number		Distributor



Unique device identifier



Distributed By :
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co., Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803 Fax:
0514-87268752
Website: www.fartley.com
E-mail: sales@fartley.com