

COOK®

MEDICAL

EN

Sentrus™ Endoscope Transport Bag

Instructions for Use

ENGLISH

Caution: US federal law restricts use of this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

This device is intended to provide safe and easy transportation of an endoscope from the procedure room to the reprocessing room after the endoscopy.

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

WARNINGS

Please read all instructions before using this device.

This device is designed for single use only. Attempts to reprocess and/or reuse may lead to device failure and/or transmission of disease.

Inspect the package for shipping or handling damage. If damage is evident, do not use. Please notify Cook Medical for return authorization.

Follow society guidelines and original equipment manufacturer recommendations to properly prepare for an endoscopy procedure and pre-clean the endoscope after an endoscopy procedure.

Check the expiration date; do not use if expired.

USER QUALIFICATIONS

This device should only be used by or under the supervision of physicians who have been trained in endoscopic procedures.

PACKAGING AND STERILIZATION

This device is packaged non-sterile.

INSTRUCTIONS FOR USE

1. When finished pre-cleaning and disconnecting the endoscope per manufacturer's recommendations, gently lay the endoscope, centered on the Endoscope Transport Bag.
2. Carefully fold the sides (left and right) of the Endoscope Transport Bag over the endoscope and pull gently on the drawstrings to close the Endoscope Transport Bag, fully covering the endoscope.
3. Place the Biohazard Label on the exterior of the Endoscope Transport Bag by removing its paper backing and adhering it directly to the Endoscope Transport Bag.

STORAGE AND TRANSPORTATION

Store at room temperature between 50 °F and 77 °F (10°C and 40°C), away from sunlight, non-corrosive gases, and in a well-ventilated area.

DEVICE DISPOSAL

After use, this device may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A symbol glossary can be found at cookmedical.com/symbol-glossary

	Manufacturer		Keep dry
	Temperature limit		Keep away from sunlight
	Country of manufacture		Medical device
	Use by date		For Single use only.
	Batch number		Caution
	Not made with natural rubber latex		Catalogue number
	Do not use product if package is damaged		Number of devices in package
	Non-sterile		Consult instructions for use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Model Number
	Distributor		Unique device identifier



Distributed By:
Cook Medical, LLC
400 N. Daniels Way
Bloomington, IN 47404 USA

Cookmedical.com



Yangzhou Fartley Medical
Instrument Technology Co.,
Ltd.
Beizhou Road, Lidian Town,
Guangling District 225106
Yangzhou, Jiangsu China
Tel.: 0514-87572788-803 Fax:
0514-87268752
Website: www.fartley.com
E-mail: sales@fartley.com