English

FLEXIPET® MICRO-MANIPULATION PIPETTES

NOTE: One cell mouse embryo tested and passed with 75% or greater blastocyst rate. USP endotoxin (LAL) tested and passed with 20 EU's or less per device. Testing is conducted on a lot-to-lot (batch) basis.

CAUTION: Federal (U. S. A.) law restricts this device to sale by or on the order of a physician.

PRECAUTIONS:

Assisted reproduction microtools (and any other accessories used during these procedures having direct contact with gametes, zygotes, pre-embryos and/or embryos) should be comprised of embryo compatible materials.

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population.

CONTRAINDICATIONS: There are no known contraindications for the use of this device.

• FLEXIPET® DENUING PIPETTES

Flexipet® Denuding Pipettes are used to remove cumulus cell layers. Supplied sterile in peel-open packages. Intended for one-time use.

SUGGESTED INSTRUCTIONS FOR USING FLEXIPET® DENUING PIPETTES:

1. Attach Flexipet® to the Cook® Adjustable Handle (Order Number: K-MPH-1000) or the Microcise™ Fleximate™ (Order Number: K-FMDT-1000).
2. Aspirate oocyte in and out of pipette until the cumulus cell layers are removed.
3. Gently withdraw pipette tip from dish.
4. Remove pipette from holder and discard.

• FLEXIPET® MANIPULATION PIPETTES

Flexipet® Manipulation Pipettes are used to manipulate the cumulus oocyte complex. Supplied sterile in peel-open packages. Intended for one-time use.

SUGGESTED INSTRUCTIONS FOR USING FLEXIPET® MANIPULATION PIPETTES:

1. Attach Flexipet® to the Cook® Adjustable Handle (Order Number: K-MPH-1000) or the Microcise™ Fleximate™ (Order Number: K-FMDT-1000).
2. Aspirate oocyte into pipette for manipulation, transfer, etc.
3. Expel oocyte and gently withdraw pipette tip from dish.
4. Remove pipette from holder and discard.

HOW SUPPLIED

Supplied sterilized by gamma irradiation in peel-open pouches. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

North American Representative

COOK MEDICAL INCORPORATED
P. O. Box 4195
Bloomington, IN 47402-4195 USA
Phone: 812 339-2235
Toll Free: 800 457-4500
Toll Free Fax: 800 554-8335

COOK IRELAND LTD.
O'Halloran Road
National Technology Park
Limerick, Ireland
Phone: +353 61 334440
Fax: +353 61 334441

Cook Incorporated
750 Daniels Way
Bloomington, IN 47404 USA

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© COOK 2014