TEST TUBE HEATER

K-FTH-1012

Instructions for Use

CE 0123

www.cookmedical.com

Language Index

English	Section 1
Česky	Kapitola 2
Dansk.....	Afsnit 3
Deutsch	Abschnitt 4
Español.....	Apartado 5
Français	Chapitre 6
Ελληνικά.....	Ενότητα 7
Italiano.....	Sezione 8
Nederlands	Gedeelte 9
Polski	Rozdział 10
Português.....	Secção 11
Русский	Раздел 12
Slovenčina.....	Čast' 13
Suomi	Luku 14
Svenska	Avsnitt 15
Türkçe.....	Bölüm 16

General information

Copyright

This manual contains information that is subject to copyright. All rights reserved. This manual should not be photocopied, duplicated on microfilm or otherwise copied or distributed, completely or in part, without the approval of William A. Cook Australia Pty. Ltd.

WARNING: Any adjustment, modification or repairs to the equipment should be carried out by persons authorised to perform them.

WARNING: The equipment is to be used only with electrical systems complying with all IEC, CEC and NEC requirements.

Some of the parts and equipment referred to in this manual bear registered trademarks but are not identified as such. It should therefore not be assumed that the absence of the trademark indicates that any given designation is not subject to trademark protection.

Users of William A. Cook Australia Pty. Ltd. products should not hesitate to contact us if there are any unclear points or ambiguities in this manual.

Disposal of this product must be undertaken with regard to the WEEE directive (2002/96/EC).

This symbol indicates that this product may not be treated as municipal waste. Please ensure that this product is properly disposed of as inappropriate waste handling of this product may cause potential hazards to the environment and human health. For more detailed information about disposal of this product, please contact your local city office or Cook Medical Representative.

Cook fulfills its legal obligations with regards to WEEE and Waste Packaging compliance through our own takeback initiatives and through national takeback schemes.

Please see http://www.cookmedical.com/businessPractice.do?id=Environmental_Recycling for details on how to properly recycle WEEE or waste packaging in your country.

© Cook 2014

Document No: IFU-KFTH/6

Service Address:

Please refer to your local Cook Medical distributor for details of your nearest authorised service agent.

Table of contents

1

EXPLANATION OF PICTOGRAMS	1-3
HOW TO USE THIS MANUAL	1-3
1. SAFETY INSTRUCTIONS	1-4
1.1 Warnings	1-5
2. ABOUT THE FTH TEST TUBE HEATER	1-6
2.1 Intended use	1-6
2.2 Device description	1-6
2.3 Precautions for device use	1-6
3. INITIAL USE OF THE DEVICE	1-6
4. OPERATION OF THE DEVICE	1-7
4.1 Front of the device	1-7
4.2 Rear of the device	1-7
4.3 Turning the unit on	1-8
4.4 Initial heating of the unit	1-8
4.5 Placement of test tubes	1-8
4.6 Turning the unit off	1-8
5. ALARM CONDITIONS	1-8
5.1 Unit insufficiently heated	1-8
5.2 Unit at operating temperature	1-8
5.3 Overheating Alarm	1-9
5.4 Disconnection Alarm	1-9
5.5 Insufficient Power Alarm	1-9
6. SERVICE AND MAINTENANCE	1-9
6.1 Cleaning the device	1-9
6.2 Periodic inspection	1-9
6.3 Inspection by an authorised service agent	1-10
6.4 Returning the device	1-10
7. TECHNICAL DATA	1-11
8. TROUBLESHOOTING	1-11
9. LIMITED WARRANTY	1-12
9.1 Liability	1-12
9.2 Life of Product	1-12
10. INDEX	1-13

Explanation of pictograms

The following pictograms appear on the FTH Test Tube Heater

1

	Before connection, read the manual
	Consult the operating instructions for information needed for the proper use of the device
	Direct current
	Test Tube Heater
	CE Mark Approval
IP64	Degree of enclosure protection from dust and water sprayed from all directions
	Dispose of in accordance with WEEE directive (2002/96/EC)
	Manufacturer
	EC Representative
	Catalogue Code
	Serial Number

How to use this manual

Please read this manual completely and follow its instructions carefully. The words **WARNING**, **CAUTION** and **NOTE** carry special meanings and they should be read carefully.

WARNING:

The personal safety of the patient may be involved. Disregarding this information could result in injury to the patient or operator, or damage to the device or contents.

CAUTION:

These instructions point out special service procedures or pre-cautions that you must follow to avoid damaging the device.

NOTE:

This provides special information that facilitates maintenance or clarifies important instructions. Please pay particular attention to the Safety Instructions.

1. Safety instructions

1

Read this manual

Please familiarise yourself with the contents of the manual before using the device. Failure to follow these instructions may result in injuries to both the patient and the user. Medical accessories and devices may be used only by physicians or medical assistants under the direction of a physician with the appropriate technical qualification.

Who should use this manual?

The manual is addressed to physicians or medical assistants with an appropriate qualification. These persons are authorised to operate this device.

Federal law

Federal Law (U.S.A.) restricts this device to use by or on the order of a physician.

Care and maintenance

To guarantee safe operation, it is absolutely necessary to carry out proper care and maintenance of the device and accessories. For the protection of the patient and the operating team, check that the device is complete and functional before each use.

Brand new products as well as repaired products must be prepared and tested according to the manual instructions before you use them for the first time.

Service agent

The manufacturer has the exclusive right to train and certify authorised service agents.

Biohazard

All devices or accessory parts that are sent in to be repaired must be prepared for transportation as described in the manual, for the protection of the service personnel and for safety during transportation. If this is not possible, the contaminated product must be clearly marked with a contamination warning and should be doubly sealed in safety foil.

Electromagnetic compatibility

The device has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices as specified by IEC/EN60601-1-2:2007. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

Medical electrical equipment requires special precautions regarding EMC and must be installed and operated according to these instructions. It is possible that high levels of radiated or conducted radio-frequency electromagnetic interference (EMI) from portable and mobile RF communications equipment or other strong or nearby radio-frequency sources could result in performance disruption of the device. Evidence of disruption may include erratic readings, equipment ceasing to operate, or other incorrect functioning. If this occurs cease using the device and contact your Cook Medical authorised service agent.

1.1 Warnings

Reading the manual

This manual describes the operation and the intended use of the device and the accessories.

It is essential to use this manual to familiarise yourself with the functions and the operation of the device before using it in the operating room.

Not following the manual instructions can result in serious injury to the patient or the operating team and can lead to damage or breakdown of the device and accessories.

Professional qualification

This manual does not provide a detailed description of operation technologies, nor is it suitable for introducing a beginner to this operating technique. Medical accessories and devices may be used only by physicians and medical assistants under the direction of a physician with the appropriate technical qualification.

Replacement device and accessories

In case the device fails during an operation, a replacement device and replacement accessories should be kept within reach so that the operation may be completed.

Available voltage

Determine if the available voltage corresponds to your device. Working with the wrong voltage will cause malfunction or destruction of the device.

Not explosion proof

Electrical components are not explosion proof. Do not use in an area where flammable gases are present.

Risk of electrical shock

To reduce the risk of electrical shock, do not remove cover (or back). Refer servicing to qualified service personnel.

No user serviceable parts inside.

Internal temperature setting

The operating temperature of the device is factory pre-set to a user specified value, prior to delivery. The set temperature may not be that required by a doctor during an operation, for which the doctor is responsible.

Cleaning the device

Do not sterilise the device. Do not immerse the device.

Power supply

Electrical shock hazard present. Do not use in the presence of flammable anaesthetics. No attempt should be made to open the unit casing. Should any electrical or mechanical fault occur while using the K-FTH-1012, please contact your nearest authorised service agent or Cook Distributor.

2. About the FTH Test Tube Heater

2.1 Intended use

The K-FTH-1012 Test Tube Heater is intended to hold test tubes containing aspirated follicular fluid at a specified temperature.

2.2 Device description

The test tube heater is an electronically controlled heating unit powered by a 12 V DC power supply and is designed to accept up to six 14 mL test tubes.

2.3 Precautions for device use

If a device is suspected or confirmed of being deficient, stop using the device until it has been checked by authorised service personnel.

Use only with Cook K-MAR-5100 Vacuum Pump or with Cook APS series power supply.

3. Initial use of the device

Initial inspection

Check the device and all accessories immediately upon receipt to make sure the contents are complete and that nothing is damaged. The manufacturer will only honour claims for compensation which are forwarded immediately to the sales representative or the authorised service company.

Device placement

Place the device on a level surface in a dry place. The ambient temperature should be between +15 to +30°C, and the relative air humidity should be between 10 – 90% for the device to function correctly. Position the device such that quick and easy disconnection of the power supply plug is not impeded.

Power supply connection

The K-FTH-1012 is powered from either the rear outlet socket on the K-MAR-5100 or the K-APS-200 (220 – 240 Vac), K-APS-100 (100 – 120 Vac), or K-APS-300 (90 – 264 Vac) power supply. To avoid the risk of electric shock, equipment must only be connected to a supply mains with protective earth.

CAUTION: Determine if the available voltage corresponds to your device. Working with the wrong voltage will cause the device to malfunction or may permanently damage the device.

The electrical components are not explosion proof. Do not use in an area where flammable gases are present.

4. Operation of the device

4.1 Front of the device

Representative view only.

1

1. Removable acrylic cover (re-order code K-FTH-1012-CP)
2. Test Tube heating compartments
3. Heater On indicator light
4. Removable acrylic K-FTH-1012 stand
5. Power cable entry

4.2 Rear of the device

Representative view only.

4.3 Turning the unit on

Connect the power cable of the unit to the socket outlet on the rear of the K-MAR-5100 vacuum pump or to the APS series power supply connector.

The amber indicator light (3) will glow and a short single tone will sound when the power supply is turned on.

4.4 Initial heating of the unit

1. When the power supply has been turned on, the amber indicator light (3) will glow continuously, if the unit is not already heated to the operating temperature.

NOTE: *If beginning at ambient room temperature the unit takes approximately five minutes to heat up to the optimal operating temperature. During this time the amber indicator light (3) will glow continuously.*

2. When the unit has reached the set operating temperature the amber indicator light (3) will begin to flash slowly. The unit is now ready to be used.

3. Throughout the use of the unit the operating temperature will be thermostatically maintained.

4. If the power supply is less than approximately 10 V DC an audible tone will sound indicating not enough heat is generated to maintain the set temperature.

5. The temperature of the K-FTH-1012 is factory pre-set to 36.9°C at room temperature. Alternatively different temperatures may be nominated and set prior to delivery.

4.5 Placement of test tubes

1. The K-FTH-1012 has been designed to accept Falcon® 2001 Series Test Tubes. Any number, up to six can be used at the same time.

2. All test tubes should be placed in the test tube heating compartments (2) through the openings in the top of the unit. The test tube should fit snug in the appropriate test tube heating compartment (2) without using excessive force.

3. No lubricant should be used on the outside of the test tube or in the heating compartment (2).

4.6 Turning the unit off

1. When the power supply to the K-FTH-1012 is disconnected, the amber indicator light (3) will turn off. A single audible tone will also sound to alert the user to power disconnection.

5. Alarm conditions

The K-FTH-1012 uses both audible and visual indicators for a variety of conditions.

5.1 Unit insufficiently heated

When the unit is "ON" but not at the operating temperature the amber indicator light (3) will glow continuously. The unit should not be used when in this condition.

5.2 Unit at operating temperature

When the unit has reached its operating temperature (within approx. 1°C) the amber indicator light (3) will begin to flash slowly. The unit should only be used when in this condition.

5.3 Overheating Alarm

If the temperature of the unit exceeds the preset temperature a secondary safety circuit will be activated preventing overheating of the test tube contents. In this alarm condition:

- The amber indicator light will turn off and no further heating of the unit will occur.
- A continuous audible tone will sound until the temperature drops to preset temperature.

Note: If the ambient temperature of the environment is high, the overheating alarm may activate. The unit will not commence heating until the ambient temperature falls below the preset temperature. The overheating alarm will activate again when the temperature of the unit exceeds the preset temperature. This cycle will be repeated as long as the ambient temperature remains high.

5.4 Disconnection Alarm

Each time the power supply to the unit is disconnected or turned off a single audible tone will sound to alert the user.

5.5 Insufficient Power Alarm

If the power supply drops below approximately 10 V DC the audible tone will sound.

6. Service and maintenance

To preserve the device and ensure its proper functioning, proper service, maintenance and storage must be provided for. To protect the patient from infections, all accessory items that come into contact with human tissue (example test tubes and tubing) must be sterile. Accessory items must be discarded after each patient use.

6.1 Cleaning the device

WARNING: *Electric shock hazard.*

CAUTION: *Do not sterilise the device. Do not immerse the device.*

After each use, remove the power cable from the connection at the rear of the unit. Use a hydrous disinfecting solution or a 70% isopropyl alcohol solution to clean the exterior of the device. Follow instructions provided by the disinfectant manufacturer to determine the correct fluid concentration. Moisten a cloth with the disinfectant solution and wipe the surface of the device. The front acrylic panel may be removed in order to facilitate cleaning. Do not submerge the unit in the disinfectant solution.

6.2 Periodic inspection

To maintain the device integrity and functionality, it is recommended that the device be inspected regularly by the device owner or user for any sign of wear or deterioration, e.g. if the cable sheath is cracked or broken, or damage to the enclosure which may allow liquids to enter the device.

Regular inspections will assist in early detection of possible malfunctions. This helps preserve the device and maintains its safety.

It is recommended that the temperature set point is tested on a biannual basis using the following method:

Turn the unit on as described in §4.3.

To measure the temperature:-

1. Insert a capped test tube into each of the wells except the second left.
2. Insert a modified test tube in the second left well.
3. Pour 6.5 ml of water into the modified test tube and insert the glass thermometer through the hole in the cap.

4. Ensure that the water level now covers the entire bulb and that the bulb is centred in the test tube, ie not touching tube wall.
5. Ensure the device is not in the presence of any draughts
6. Allow enough time for the thermometer to thermally equilibrate.

1

1. Mercury Thermometer
2. Modified Test Tube Cap to allow a thermometer to be inserted
3. 6.5 mL of water

The temperatures measured will be the set temperature $\pm 1.0^{\circ}\text{C}$. (Standard factory set point is 36.9°C)

6.3 Inspection by an authorised service agent

Inspections at least once a year

For ongoing operational safety of the device, the device must be maintained at regular intervals by an authorised service agent as per SMM30004. The service agent will verify the performance of the device. Depending on the duration and frequency of use, the device should be maintained at least once a year. Failure to adhere to this maintenance schedule will result in the manufacturer refusing liability for the operational safety of the device.

Authorised service agents

All services such as alterations, repairs, calibrations etc., may only be performed by the manufacturer or by service agents who are authorised by the manufacturer as per SMM30004.

Liability

The manufacturer is free from all liability for the operational safety of the device if the device has been wilfully opened and unauthorised persons have performed repairs or alterations on it.

Certification

The device owner should receive a certificate from the service technician for all inspections or repairs. This certificate states the type and scope of the services rendered, the service date and the name of the service company. The certificate should be signed.

Technical documentation

If the manufacturer provides you with technical documentation, this does not authorise the user to perform repairs, adjustments or alterations to the device or accessories.

6.4 Returning the device

If it becomes necessary to return the device, use of the original packaging is required. The manufacturer does not take responsibility for damage that has occurred during transportation if the damage was caused by inadequate transport packaging. Please enclose the following information with the device:

- owner's name
- owner's address
- model type
- serial number of the equipment (see identification plate)
- description of the damage or fault

The manufacturer has the right to refuse to carry out repairs if the products it receives are contaminated.

7. Technical data

Classification according to IEC 60601-1

Degree of protection against harmful ingress of solids and water:	IP64
---	------

General specifications

Power supply (voltage):	12 V DC
Maximum current:	1.5 A
Maximum power consumption:	18 W
Environmental Operating Conditions:	+15 to +30°C 10 to 90% RH 700 hPa to 1060 hPa
Storage and transport directions:	-10 to +50°C 5 to 90% RH
Manufactured and tested to the following standards:	IEC 60601-1: 1988+A1:1991+A2:1995 IEC60601-1: 2005 IEC/EN60601-1-2: 2007
Temperature accuracy:	Set temperature +/- 1.0°C over recommended operational range
Dimensions:	(Width, Height, Depth) 150 x 100 x 28 mm
Weight:	0.7 kg
Cord length:	1.5 m

8. Troubleshooting

Error and alarm indicator	Source of error	Elimination of error
Indicator light does not illuminate.	Power supply is not connected.	Check the power supply connection.
	Incorrect voltage.	Contact your nearest service agent.
Indicator light does not illuminate and a single continuous tone sounds.	Factors in the environment may have warmed the unit above the preset temperature.	Check the ambient temperature and remove the unit to cooler environment. Allow the unit to cool for 10 minutes and then turn on. If the unit heats up but then sounds the overheating alarm again, contact your nearest service agent.
A continuous audible tone sounds.	Power supply inadequate.	Check that the power supply lead is connected firmly into the outlet socket at the rear of the K-MAR-5100 unit or APS series power supply.

9. Limited warranty

1

William A. Cook Australia Pty. Ltd. warrants to the purchasers of this device that at time of manufacture, the product was prepared and tested in accordance with good manufacturing practices and guidelines specified by the relevant competent authority.

In the event of product failure under normal use, due to defects in material or workmanship, within a period of one (1) year from the date of purchase, the product will be repaired, or at Cook's option, replaced, at no charge. This limited warranty does not apply to products subjected to abnormal use, conditions, improper storage or damaged by accident, misuse, improper line voltage or to products altered or serviced by anyone other than William A. Cook Australia Pty. Ltd. or its authorised agent.

The foregoing limited warranty is exclusive and in lieu of all other warranties whether written, oral, expressed or implied. In particular, William A. Cook Australia Pty. Ltd. does not warrant that the product is suitable for the needs of the purchaser and there are no warranties given as to merchantability or fitness for a particular purpose. William A. Cook Australia Pty. Ltd. representations concerning fitness for purpose or suitability for use by any purchaser does not extend beyond those representations set out in the William A. Cook Australia Pty. Ltd. literature that accompanies the product. William A. Cook Australia Pty. Ltd. assumes that the purchaser is experienced in the use of this device and is able to judge from his/her own expertise the suitability or otherwise of the product for the intended use. William A. Cook Australia Pty. Ltd. conducts a technical advisory service which can be consulted by a purchaser or intended purchaser on an advisory basis.

After one (1) year from the date of purchase, this device will be repaired for a repair charge equal to the cost of parts, labour and transport.

Before returning a product for any reason, please contact your nearest Cook distributor for assistance and instructions.

William A. Cook Australia Pty. Ltd. reserves the right to change or discontinue this product without notice.

9.1 Liability

Because William A. Cook Australia Pty. Ltd. has no control or influence over the conditions under which this device is used, over its method of use or administration, or on handling of the product after it leaves its possession, William A. Cook Australia Pty. Ltd. takes no responsibility for the results, use and/or performance of the product. William A. Cook Australia Pty. Ltd. expects that use of the product will be confined to trained and expert users.

In no event will William A. Cook Australia Pty. Ltd. be liable for any direct or indirect damages including incidental, consequential or special damages, arising out of or in connection with the use or performance of the product.

If the manufacturer provides you with technical documentation, this does not authorise you to perform repairs, adjustments or alterations on the device or disposables.

No representative of William A. Cook Australia Pty. Ltd. and no vendor or lessor of the product is authorised to change any of the foregoing terms and conditions, and the purchaser accepts the product subject to all terms and conditions herein, subject always to any contrary provisions which are necessarily implied by statute or law notwithstanding the within terms and conditions.

9.2 Life of Product

The life of this product is deemed to be seven (7) years. After this time William A. Cook Australia Pty. Ltd. will no longer be responsible for this product.

10. Index

A

Activating the device	8
After the operation	8
Alarm conditions	8
Authorised service agents	10

B

Biohazard	4
-----------------	---

C

Care and maintenance	4
Certification	10
Cleaning the device	5, 9

D

Device description	6
Dimensions	11

E

Electrical shock	5
EMC	4
Explosion warning	5

F

Federal law	4
Front of the device	7

I

Initial inspection	6
Initial use of the device	6
Inspection by the authorised service agent ..	10
Inspection intervals	10
Intended use	6
Internal temperature settings	5

L

Liability	12
-----------------	----

O

Operation of device	7
---------------------------	---

P

Power consumption	11
Power supply	5, 11
Power supply connection	6
Professional qualification	5

R

Rear of the device	7
Replacement device	5
Returning the device	10
Risk of electrical shock	5

S

Safety instructions	4
Service and maintenance	9
Setting up the device	6

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WILLIAM A. COOK AUSTRALIA PTY. LTD.
COOK IRELAND LTD.