EN
5**BiGopsy® Backloading Biopsy Forceps**

Instructions for Use

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
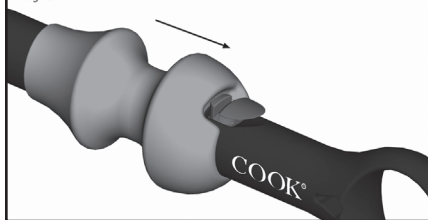
Fig 4b.

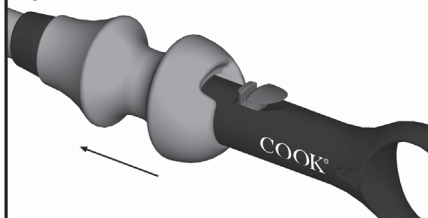BIGOPSY® BACKLOADING BIOPSY FORCEPS

CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

The BIGopsy® Backloading Biopsy Forceps is intended for obtaining tissue samples for pathological evaluation. The device may also be used for retrieval of specimens from the urinary tract.

DEVICE DESCRIPTION

The BIGopsy® Backloading Biopsy Forceps is a stainless-steel biopsy device capable of a 4mm³ sample size. The forceps shaft is 2.4Fr 115cm long with a removable handle.

CONTRAINDICATIONS

Contraindications include coagulopathy and those specific to the primary ureteroscopic procedure performed in gaining access to the desired biopsy or retrieval site.

POTENTIAL ADVERSE EVENTS

Potential adverse events include, but are not limited to: allergic reaction to nickel, bleeding, fever, infection, nausea/vomiting, pain/discomfort, perforation.

PRECAUTIONS

This device is intended for use by physicians trained and experienced in ureteroscopic procedures. Standard techniques for urology should be employed.

This device should be used with a ureteral access sheath. Ensure the biopsy cups are closed before surgical introduction. Do not depress the button during use, the handle will release from the forceps.

Do not use this device for any purpose other than the stated intended use.

Store in a dry location, away from temperature extremes.

WARNINGS

This device is designed for single use only. Attempts to reprocess, re-sterilize, and/or reuse may lead to device failure and/or transmission of disease.

This device is conductive. Avoid contact with any electrified instrument. Do not force the device during the procedure; proceed with caution if excessive resistance is encountered.

Visually inspect the product to ensure no damage has occurred. If the package is opened or damaged when received, do not use. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

INSTRUCTIONS FOR USE

Illustrations

1. Prior to attaching the handle, back load the BIGopsy® forceps into the working channel of the endoscope. **Note:** The BIGopsy® Backloading Biopsy Forceps component parts are indicated in **Fig. 1**.

2. Secure the handle to the forceps according to instructions as follows:
 - a) With the button depressed, retract the finger spool over the button.
 - b) Secure handle in this position by releasing the button (**Fig. 2**). The forceps shaft may now be introduced into the handle.
 - c) Feed the forceps shaft through the handle (**Fig. 3a**). When the forceps shaft is visible in the viewing window of the handle (**Fig. 3b**) press button to release finger spool (**Fig. 3c**). This secures the forceps shaft in the handle.
3. After the handle is attached, ensure the operation of the biopsy cups before surgical introduction. Biopsy cups default to the closed position. To open the biopsy cups pull the finger spool towards the thumb ring (**Fig. 4a**). To close, push the finger spool forward (**Fig. 4b**). **Note:** Biopsy cups need to be in the closed position before surgical introduction.
4. Through a previously placed ureteral access sheath, advance endoscope with the forceps to the desired biopsy or retrieval site and proceed with biopsy or specimen retrieval.
Caution: Do not depress the button during use, the handle will release from forceps.
5. Prior to handle removal, ensure that the biopsy sample is retrieved.
6. At the completion of the procedure, detach the handle according to instructions as follows:
 - a) With the button depressed, retract the finger spool over the button.
 - b) Secure handle in this position by releasing the button (**Fig. 2**). The forceps shaft may now be removed from the handle.
7. Remove BIGopsy® forceps from endoscope.
8. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages.

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© COPYRIGHT COOK 2019 ©
COOK IRELAND LTD.