

- EN 5** **Evolution® Duodenal Stent System – Uncovered**  
 Instructions for Use
- BG 10** **Система дуоденален стент Evolution® – без покритие**  
 Инструкции за употреба
- HR 17** **Sustav duodenalnog stenta Evolution® – otkriveni**  
 Upute za uporabu
- CS 22** **Systém duodenálního stentu Evolution® – nekrytý**  
 Návod k použití
- DA 28** **Evolution® duodenalt stentsystem – udækket**  
 Brugsanvisning
- NL 34** **Evolution® duodenumstentsysteem – niet-bekleed**  
 Gebruiksaanwijzing
- ET 40** **Kaksteistsõrmiksoole stendisüsteem Evolution® – katteta**  
 Kasutusjuhised
- FR 46** **Système d'endoprothèse duodénale Evolution® – non couverte**  
 Mode d'emploi
- DE 53** **Evolution® Duodenalstentsystem – Nicht ge covert**  
 Anleitung zum Gebrauch
- EL 59** **Σύστημα ενδοπρόσθεσης δωδεκαδακτύλου Evolution® – ακάλυπτο**  
 Οδηγίες χρήσης
- HU 66** **Evolution® nyombélsztentrendszer – fedetlen**  
 Használati utasítás
- IT 72** **Sistema con stent duodenale Evolution® – Non rivestito**  
 Istruzioni per l'uso
- LV 79** **Evolution® divpadsmiņpirkstu zarnas stenta sistēma — bez pārklājuma**  
 Lietošanas instrukcija
- LT 85** **„Evolution®“ dvylikapirštės žarnos stento sistema – nedengta**  
 Naudojimo nurodymai
- NO 91** **Evolution® Stentsystem for duodenum – utildekket**  
 Bruksanvisning
- PL 97** **System Evolution® do stentowania dwunastnicy – nieosłonięty**  
 Instrukcja użycia
- PT 103** **Sistema de stent duodenal Evolution® – Não revestido**  
 Instruções de utilização
- RO 110** **Sistem pentru stentare duodenală Evolution® – fără strat de acoperire**  
 Instrucțiuni de utilizare
- SK 116** **Systém duodenálneho stentu Evolution® – nekrytý**  
 Návod na použitie
- ES 122** **Sistema de stent duodenal Evolution® – sin recubrimiento**  
 Instrucciones de uso
- SV 128** **Evolution®-stentsystem för duodenum – utan täckning**  
 Bruksanvisning


1.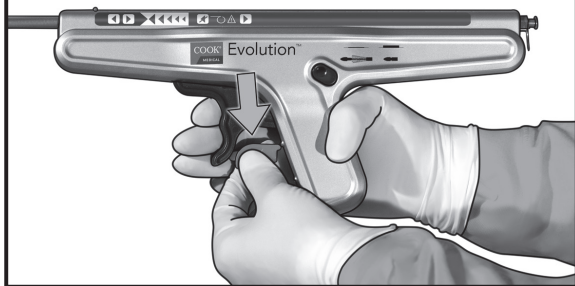


2.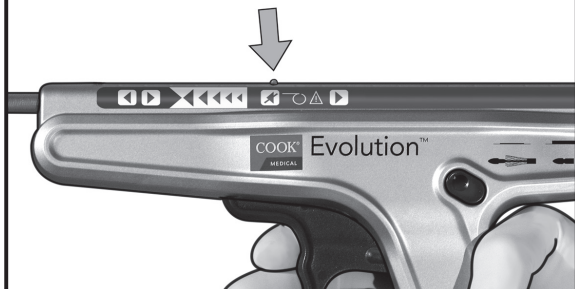


3.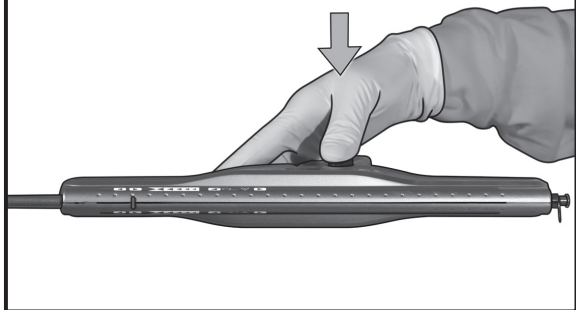

4.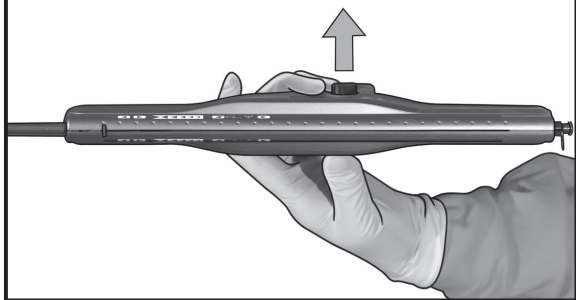

5.

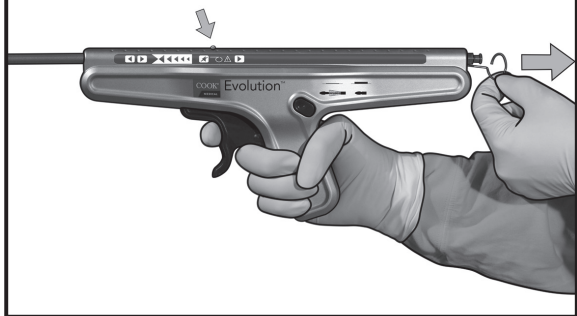## **EVOLUTION® DUODENAL STENT SYSTEM - UNCOVERED**

**CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

### **IMPORTANT INFORMATION**

Please review prior to use.

### **INTENDED USE**

This device is used for palliative treatment of duodenal or gastric outlet obstruction and duodenal strictures caused by malignant neoplasms.

### **STENT DESCRIPTION**

This flexible, self-expanding stent is constructed of a single, woven, nitinol wire. The stent has a flange at both ends to aid in preventing migration. The stent foreshortens due to its design. The total length of the stent is indicated by radiopaque markers on the inner catheter, indicating the actual length of the stent at nominal stent diameter. To aid in visibility under fluoroscopy, there are four tantalum marker bands at either end of the stent. The proximal and distal ends of the stent are designed with numerous rounded "crowns" to potentially reduce tissue trauma.

### **DELIVERY SYSTEM DESCRIPTION**

The stent is mounted on an inner catheter, which accepts a .035-inch wire guide, and is constrained by an outer catheter. An endoscopically and fluoroscopically visible yellow marker defines the proximal end of the stent when constrained in the catheter. A pistol-grip delivery handle allows controlled stent deployment or recapture. The position of the red marker on top of the handle in relation to the point-of-no-return position on the handle label indicates the point after which it is not possible to recapture the deployed stent.

### **CONTRAINDICATIONS**

Those specific to GI endoscopy and any procedure to be performed in conjunction with stent placement.

Additional contraindications include, but are not limited to: enteral ischemia • suspected or impending perforation • intra-abdominal abscess/perforation • inability to pass wire guide or stent through obstructed area • patients for whom endoscopic procedures are contraindicated • coagulopathy/patients with elevated bleeding times • benign disease.

## **POTENTIAL ADVERSE EVENTS**

Potential adverse events associated with GI endoscopy include, but are not limited to: airway obstruction • allergic reaction to contrast or medication • aspiration • biliary obstruction • cardiac arrhythmia or arrest • cholangitis • fever • hemorrhage • hypotension • infection • perforation • reflux • respiratory depression or arrest.

Additional adverse events include, but are not limited to: allergic reaction to nickel • bowel impaction • death (other than due to normal disease progression) • erosion of the luminal mucosa • foreign body sensation • inadequate expansion • intestinal perforation • nausea/vomiting • pain/discomfort • pancreatitis • pressure necrosis • septicemia • stent misplacement and/or migration • stent occlusion • tumor ingrowth or overgrowth • ulcerations.

## **PRECAUTIONS**

Refer to product package label for the minimum channel size required for this device.

A complete diagnostic evaluation must be performed prior to use to determine proper stent size.

If wire guide or stent cannot advance through obstructed area, do not attempt to place stent.

Stent should be placed using fluoroscopic monitoring with endoscopy.

The stent should only be placed with the Cook delivery system, which is provided with each stent.

This device is intended for palliative treatment only. Alternate methods of therapy should be investigated prior to placement.

Long-term patency of this device has not been established. Periodic evaluation is advised.

Stent cannot be retrieved after the deployment threshold has been passed. The position of the red marker on the top of the handle in relation to the point-of-no-return position on the handle label indicates when threshold has been passed.

Do not use this device for any purpose other than stated intended use.

Use of this device is restricted to a trained healthcare professional.

## **WARNINGS**

The stent is not intended to be removed or repositioned after stent placement and is intended to remain in the body permanently. Attempts to remove or reposition stent after placement may cause damage to surrounding tissue or mucosa.

The stent contains nickel, which may cause an allergic reaction in individuals with nickel sensitivity.

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

After stent placement, alternative methods of treatment such as chemotherapy and radiation should not be administered as this may increase risk of stent migration due to tumor shrinkage, stent erosion, and/or mucosal bleeding.

## **MR INFORMATION**

This symbol means the device is MR Conditional.

Non-clinical testing has demonstrated that this stent is MR Conditional according to ASTM F2503. A patient with this stent can be scanned safely immediately after placement under the following conditions.

### **Static Magnetic Field**

- Static magnetic field of 3 Tesla or less
- Maximum spatial magnetic gradient of 720 Gauss/cm or less

Non-clinical evaluation was conducted in a 3 Tesla MR system (Excite, General Electric Healthcare) with a maximum spatial magnetic gradient field of 720 Gauss/cm as measured with a gaussmeter in the position of the static magnetic field pertinent to the patient (i.e., outside of scanner covering, accessible to a patient or individual).

### **MRI-Related Heating**

- 1.5 and 3.0 Tesla Systems: It is recommended to scan in normal operation mode (whole body averaged specific absorption rate (SAR)  $\leq 2.0$  W/kg) ("Normal Operating Mode" is defined as the mode of operation of the MR system in which none of the outputs have a value that cause physiological stress to the patient) for 15 minutes of scanning (i.e., per scanning sequence).
- Non-clinical testing was conducted on the stent under the following conditions, and produced a maximum temperature rise of 3.7 °C:
  - a maximum whole body average specific absorption rate (SAR) of 2.9 W/kg (corresponding to a calorimetry measured value of 2.1 W/kg) for 15 minutes of MR scanning in a 1.5 Tesla Magnetom (Siemens Medical Solutions, Malvern, PA, Software Numaris/4) MR scanner.
  - a maximum whole body average specific absorption rate (SAR) of 2.9 W/kg (corresponding to a calorimetry measured value of 2.7 W/kg) for 15 minutes of MR scanning in a 3.0 Tesla Excite (GE Electric Healthcare, Milwaukee, WI, Software 14X.M5) MR scanner.

### **Image Artifacts**

MR image quality may be compromised if the area of interest is within the lumen of the Evolution Stent or within approximately 6 mm of the position of the Evolution Stent as found during non-clinical testing using the sequences: T1-weighted, spin echo pulse sequence and gradient echo pulse sequence in a 3.0 Tesla Excite (GE Electric Healthcare, Milwaukee, WI, Software 14X.M5) MR system with body radiofrequency coil. Therefore, it may be necessary to optimize MR imaging parameters for the presence of this metallic stent.

Additional Information: The safety of performing an MRI procedure in a patient with overlapping duodenal stents or other MRI-conditional device(s) in direct contact with this device has not been determined. Performing MRI in such situations is not recommended.

### **For US Patients only**

It is recommended that patients register the conditions under which the implant can safely be scanned with the MedicalAlert Foundation ([www.medicalert.org](http://www.medicalert.org)) or an equivalent organization.

### **EQUIPMENT REQUIRED**

Stent and delivery system

.035-inch wire guide

## SELECTION OF STENT

A stent length of at least 4 cm longer than the stricture is recommended (e.g. approximately 2 cm longer at either end of the stricture).

## PATIENT PREPARATION

With duodenoscope in place, introduce a wire guide until it is fluoroscopically visualized in position through stricture.

## INSTRUCTIONS FOR USE

### Illustrations

1. Remove stent delivery system from the package.
2. Introduce delivery system in short increments over wire guide, into accessory channel, until it is endoscopically visualized exiting scope.
3. To ensure that stent will bridge stricture after deployment, fluoroscopically position radiopaque markers on inner catheter beyond extremities of stricture to be crossed. **Note:** Yellow marker on the delivery system located at the proximal end of stent can also serve as an endoscopic/fluoroscopic reference for positioning of the proximal (nearest user) end of the stent relative to the stricture.
4. Confirm desired stent position fluoroscopically and deploy stent by removing red safety guard from the handle. **(Fig. 1)**
5. Continue deploying stent by squeezing trigger. **Note:** Each trigger squeeze will deploy stent by an equal amount.
6. If stent repositioning is required during deployment, it is possible to recapture stent. **Note:** It is **not** possible to recapture stent after passing point-of-no-return, indicated when red marker on top of the handle has passed the point-of-no-return position on the handle label. The stent can be recaptured a maximum of five times. **(Fig. 2)**
7. To recapture stent, push direction button on side of delivery system to opposite side. **(Fig. 3)** **Note:** Hold thumb on button when squeezing trigger for first time to recapture.
8. Continue squeezing trigger as required to recapture stent by desired amount.
9. To resume deployment, push button to opposite side again and hold thumb on button for first stroke while squeezing trigger. **(Fig. 4)**

10. When stent point-of-no-return has been passed, pull safety wire out of delivery handle near wire guide port. **(Fig. 5)**

11. Continue deploying stent by squeezing trigger.

12. After deployment, fluoroscopically confirm full stent expansion. Once full expansion is confirmed, delivery system can be safely removed.

**Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.**

#### **HOW SUPPLIED**

Store in a dry location, away from temperature extremes.

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© COPYRIGHT COOK 2020 ©
**COOK IRELAND LTD.**