EN 3 Amplatz Needle Holder with Silicone Inserts

Instructions for Use

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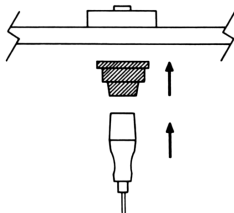
Figure C

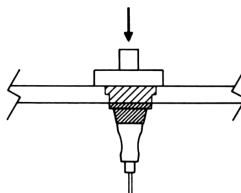
Figure D

AMPLATZ NEEDLE HOLDER WITH SILICONE INSERTS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Polycarbonate needle holder
- Silicone inserts

NOTE: Set components may vary.

INTENDED USE

This device is used to direct a Cook standard disposable needle during needle access procedures. It prevents the stylet from backing out of the cannula during puncture and also keeps the hands from radiation exposure during the puncture procedure.

CONTRAINDICATIONS

None known

WARNINGS

Needles used with this device should always be used under fluoroscopic guidance.

PRECAUTIONS

Always ensure the needle is securely placed within the silicone insert.

INSTRUCTIONS FOR USE

1. Secure the Cook trocar needle in the silicone insert within the clear plastic needle handle.
2. Position your hands to the outside of the device to ensure minimal radiation exposure during the procedure.
3. Upon completion of the procedure, remove needle and discard device.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd