- EN**
3 **Amplatz Needle Holder with Silicone Inserts**
Instructions for Use
- CS**
3 **Držák jehly Amplatz se silikonovými vložkami**
Návod k použití
- DA**
4 **Amplatz nåleholder med silikoneindsatser**
Brugsanvisning
- DE**
5 **Amplatz Kanülenhalter mit Silikoneinsätzen**
Gebrauchsanweisung
- EL**
6 **Βελονοκάτοχο Amplatz με ένθετα σιλικόνης**
Οδηγίες χρήσης
- ES**
6 **Portaagujas Amplatz con engastes de silicona**
Instrucciones de uso
- FR**
7 **Porte-aiguille d'Amplatz avec inserts en silicone**
Mode d'emploi
- HU**
8 **Amplatz tűtartó szilikonbetétekkel**
Használati utasítás
- IT**
9 **Porta-ago Amplatz con inserti in silicone**
Istruzioni per l'uso
- NL**
9 **Amplatz naaldhouder met silicone inzetstukken**
Gebruiksaanwijzing
- NO**
10 **Amplatz kanyleholder med silikoninnlegg**
Bruksanvisning
- PL**
11 **Uchwyt igły Amplatz z wkładkami silikonowymi**
Instrukcja użycia
- PT**
12 **Suporte de agulha Amplatz com acessórios de silicone**
Instruções de utilização
- SV**
12 **Amplatz nålhållare med silikoninsatser**
Bruksanvisning

BLUE INSERT

MODRÁ VLOŽKA
BLÅ INDSATS
BLAUER EINSATZ
ΜΠΛΕ ΕΝΘΕΤΟ
ENGASTE AZUL
INSERT BLEU
KÉK SZÍNŰ BETÉT
INSERTO BLU
BLAUW INZETSTUK
BLÅTT INNLEGG
WKŁADKA NIEBIESKA
ACCESSÓRIO AZUL
BLÅ INSATS

Figure A

Blue insert for Cook® Disposable Chiba Needle

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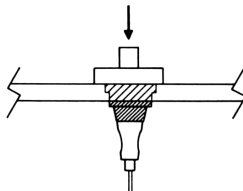
Figure D

AMPLATZ NEEDLE HOLDER WITH SILICONE INSERTS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Polycarbonate needle holder
- Silicone inserts

NOTE: Set components may vary.

INTENDED USE

This device is used to direct a Cook standard disposable needle during needle access procedures. It prevents the stylet from backing out of the cannula during puncture and also keeps the hands from radiation exposure during the puncture procedure.

CONTRAINDICATIONS

None known

WARNINGS

Needles used with this device should always be used under fluoroscopic guidance.

PRECAUTIONS

Always ensure the needle is securely placed within the silicone insert.

INSTRUCTIONS FOR USE

1. Secure the Cook trocar needle in the silicone insert within the clear plastic needle handle.
2. Position your hands to the outside of the device to ensure minimal radiation exposure during the procedure.
3. Upon completion of the procedure, remove needle and discard device.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.