**COOK**  
MEDICAL

**CE**  
0123

**EN**  
3

## Peel-Away® Introducer

Instructions for Use

**CS**  
4

## Zavaděč Peel-Away®

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**DA**  
5

## Peel-Away® indfører

Brugsanvisning

**DE**  
6

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**EL**  
7

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T - A P A I - R E V O

## Peel-Away® Introducer

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.**

### DEVICE DESCRIPTION

- Peel-Away Sheath
- Introducer

**NOTE:** Components may vary by size

### INTENDED USE

The Peel-Away Introducer is intended to be used to establish a conduit during endoscopic urological procedures (e.g. nephrostomy, cystoscopy, ureteroscopy, etc.) facilitating the passage of endoscopes and other instruments into the urinary tract.

### CONTRAINDICATIONS

This device is contraindicated for use in patients unsuitable for endoscopic urological procedures.

### WARNINGS

Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. The device is intended for one-time use.

### PRECAUTIONS

This device should only be used by qualified physicians experienced in endourological procedures. Physician should ensure patient has a full and palpable bladder for correct bladder position.

### POTENTIAL ADVERSE EVENTS

Complications associated with this procedure may include incorrect location at insertion, resulting in trauma to the surrounding tissue.

### INSTRUCTIONS FOR USE

1. Place wire guide in a manner consistent with sterile technique and standard protocol.
2. If needed, dilate tract with dilator(s) of choice up to size of Peel-Away Introducer.
3. Place Peel-Away Introducer assembly over wire guide.
4. Confirm that the sheath is properly placed by fluoroscopy.
5. Remove the dilator and wire guide from the Peel-Away sheath and place an appropriately sized endoscope or other instrument through the sheath.
6. Using the dual knobs of the Peel-Away sheath gently pull back and split the sheath into two parts while leaving the endoscope or instrument *in situ*.

Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd