COOK

MEDICAL

- EN**
3 Catheter for use with Amplatz Renal Dilator Set
Instructions for Use
- CS**
3 Katetr pro použití se soupravou renálního dilatátoru Amplatz
Návod k použití
- DA**
4 Kateter til anvendelse med Amplatz-nyredilatatorsæt
Brugsanvisning
- DE**
5 Katheter zur Verwendung mit dem Amplatz Nierendilatator-Set
Gebrauchsanweisung
- EL**
6 Καθετήρας για χρήση με το σετ νεφρικών διαστολέων Amplatz
Οδηγίες χρήσης
- ES**
6 Catéter para uso con los equipos de dilatadores renales Amplatz
Instrucciones de uso
- FR**
7 Sonde à utiliser avec le set de dilateurs rénaux d'Amplatz
Mode d'emploi
- HU**
8 Katéter Amplatz renalis dilatátorkészlethez
Használati utasítás
- IT**
9 Catetere per il set con dilatatori renali Amplatz
Istruzioni per l'uso
- NL**
9 Katheter voor gebruik met Amplatz renale dilatatorset
Gebruiksaanwijzing
- NO**
10 Kateter for bruk med Amplatz nyredilatatorsett
Bruksanvisning
- PL**
11 Cewnik do stosowania z zestawem rozszerzacza nerkowego Amplatz
Instrukcja użycia
- PT**
12 Cateter para utilização com o conjunto de dilatador renal Amplatz
Instruções de utilização
- SV**
13 Kateter för användning med Amplatz njurdilatatorset
Bruksanvisning

T _ A R C _ R E V O

CATHETER FOR USE WITH AMPLATZ RENAL DILATOR SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

The Catheter for use with Amplatz Renal Dilator Set is an 8.0 French radiopaque catheter, tapered on both ends.

INTENDED USE

This device is intended to be used over a pre-positioned wire guide for percutaneous dilation and access of nephrostomy tract.

CONTRAINDICATIONS

This device is contraindicated in the presence of conditions which create unacceptable risk during percutaneous nephrostomy.

WARNINGS

Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. The device is intended for one-time use.

PRECAUTIONS

This product is intended for use by physicians trained and experienced in Urological techniques. Standard techniques should be employed.

POTENTIAL ADVERSE EVENTS

- Tissue trauma
- Bleeding
- Perforation
- Infection
- Urine leakage
- Fistula
- Pain

INSTRUCTIONS FOR USE

1. After gaining access and achieving appropriate needle position, use fluoroscopic guidance to advance a wire guide through the needle into the renal pelvis and/or down the ureter.
2. Remove the needle, leaving the wire guide in place.
3. Under fluoroscopic guidance, load the Amplatz Renal Catheter over the wire guide and advance it into the renal pelvis. **NOTE:** If the catheter is advanced too far, it can cause trauma to the tissue.
4. Once appropriate position is achieved, continue with the procedure.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.