EN
3

Amplatz Renal Introducer
Instructions for Use

CS
4

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AMPLATZ RENAL INTRODUCER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

The Amplatz Renal Introducer consists of an Amplatz Sheath and a tapered Amplatz Dilator. The distal end of the sheath is cut at a 45-degree angle, and the proximal end is cut at a 90-degree angle. The device is available in a variety of lengths and French sizes.

INTENDED USE

This device is intended to be used for nephrostomy tract dilation and the placement of a sheath.

CONTRAINDICATIONS

This device is contraindicated in the presence of conditions which create unacceptable risk during percutaneous nephrostomy.

WARNINGS

Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. The device is intended for one-time use.

PRECAUTIONS

This product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of devices in angiographic procedures should be employed.

POTENTIAL ADVERSE EVENTS

- Tissue trauma
- Bleeding
- Perforation
- Infection
- Urine leakage
- Fistula
- Pain

INSTRUCTIONS FOR USE

1. Advance the catheter for use with Amplatz Renal Dilator Set (supplied separately) into the renal pelvis.
2. Load the Amplatz Dilator over the catheter.
3. Advance the dilator to dilate the nephrostomy tract. **NOTE:** Avoid advancing the dilator too far into the renal pelvis.
4. Load the Amplatz Sheath over the dilator.
5. While maintaining the dilator's position, advance the sheath into the calyx.
6. Use fluoroscopy to confirm location of the sheath in the calyx.
7. While holding the sheath in place, remove the dilator.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.