EN
3

Wire Guides

Instructions for Use

CS
3

Vodicí dráty

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DA
4

Kateterledere

Brugsanvisning

DE
4

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EL
5

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ZH
12

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WIRE GUIDES

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

INTENDED USE

Wire Guides are used to gain ureteral access, to establish a tract, and to assist in the placement, replacement, and exchange of medical devices during urological procedures. These wire guides are not intended for PTCA use.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- When exchanging or withdrawing an instrument over the wire guide, secure and maintain the wire guide in place under fluoroscopy in order to avoid unexpected wire guide displacement.
- Manipulation of wire guide requires appropriate imaging control. Use caution not to force or over manipulate the wire guide when gaining access.
- When using wire guide through a metal cannula/needle, use caution as damage may occur to outer coating.

INSTRUCTIONS FOR USE

1. Using aseptic technique, remove the wire from its outer packaging and place in the sterile field.
2. Remove the wire guide from the holder.
3. Advance the flexible end of the wire guide into the device and position to the desired anatomical location.

NOTE: Endhole size and length of the device must be taken into consideration to ensure proper fit between wire guide and device.

4. After wire guide use is complete, remove from patient and discard.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd