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Bruksanvisning

T _ B U D S _ R E V 2

BALLOON URETERAL DILATOR AND BALLOON URETERAL DILATOR SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

Set contains:

- Radiopaque polyethylene balloon catheter with radiopaque markers at the proximal and distal ends of the balloon.
- Stainless steel wire guide, appropriate diameter, 145 cm long with 3-cm flexible J-tip.
- Pin vise handle to stabilize the wire guide.
- Endoscopic cap
- Syringe

NOTE: Set components may vary. If no wire guide is included in this set, the following is recommended:

- 5.0 French balloons accept a .028" wire guide
- 7.0 French balloons accept a .038" wire guide

INTENDED USE

Balloon Ureteral Dilators and Balloon Ureteral Dilator Sets are used for ureteral dilation prior to ureteral stone manipulation or ureteroscopy, and dilation of the intramural ureter.

CONTRAINDICATIONS

- Do not use this device to treat obstruction that is due to extrinsic factors.
- This device is contraindicated in the presence of conditions which create unacceptable risk during catheterization.

WARNINGS

- Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
- Refer to product label for maximum inflation pressure. Do not exceed the maximum inflation pressure for this balloon device.
- Removal of the catheter is facilitated by rotating the shaft counterclockwise during withdrawal. Using excessive force to withdraw the balloon can inflict trauma to tissue and/or damage the cystoscope.

PRECAUTIONS

- Balloon dilation catheters are intended for use by physicians trained and experienced in techniques for balloon-catheter dilation.
- Do not pre-inflate the balloon prior to insertion.

POTENTIAL ADVERSE EVENTS

- Infection
- Tissue perforation
- Tissue trauma
- Hematuria

INSTRUCTIONS FOR USE:

1. Remove the protective sleeve from the balloon and discard. **CAUTION: Do not pre-inflate the balloon.**
2. Lock the syringe onto the side-arm stopcock of the balloon catheter and aspirate until all air is removed. Close the stopcock.

- Lubricate the deflated balloon with a water-soluble lubricant.
- Attach the endoscopic cap to the accessory channel of the cystoscope.
- Under X-ray or fluoroscopic control, cystoscopically pass the wire guide the desired distance into the ureter.
- Pass the balloon catheter over the wire guide. Position the tip of the balloon within the distal ureter, being sure to leave a small portion of the balloon in the ureteral orifice and extending into the bladder.

NOTE: Alternatively, the balloon and catheter can be passed through the cystoscope as a unit, following the instructions below:

- Attach the pin vise handle to the female Luer lock fitting of the balloon catheter.
 - Place the wire guide, flexible tip leading, into the central lumen and pass it the length of the catheter until only the 3-cm flexible tip extends beyond the catheter tip.
 - Maintaining position of the wire guide, twist the pin vise handle clockwise to lock the wire guide in place.
 - Cystoscopically pass the wire guide/catheter assembly so that the tip of the balloon is within the distal ureter, being sure to leave a small portion of the balloon in the ureteral orifice and extending into the bladder.
- Fill the syringe with dilute (30% solution) contrast medium and eliminate air from the syringe.
CAUTION: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
 - Open the stopcock on the inflation device and inflate the balloon. **NOTE:** To ensure proper regulation of balloon pressure, use of a balloon inflation device and pressure gauge is recommended. **CAUTION: Do not exceed 60 psi on the 7.0 French balloon ureteral dilators and 40 psi on the 5.0 French balloon ureteral dilators.**
 - Close the stopcock for at least 30 seconds to maintain pressure and allow adequate dilation.
NOTE: Time, rather than excessive pressure, is the key factor.
 - Deflate the balloon by syringe aspiration. Reposition the balloon at a higher level and repeat the process until the desired location is reached.
 - Aspirate the balloon completely, then gently remove the device. **CAUTION: Removal of the catheter is facilitated by rotating the shaft counterclockwise during withdrawal. Using excessive force to withdraw the balloon can inflict trauma to tissue and/or damage the cystoscope.**

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

A. B. Rutner: "Ureteral Balloon Dilatation and Stone Basketing," *Urology*, Special Issue, 23 (1984), 44-53.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd